



Product Reference Guide

ChemoClave®

Needlefree Closed System Transfer Device (CSTD)

Universal Vial Access Device (CH-70)

1. Spike Vial



- › Place vial on counter and align vial access device with center of vial closure.
- › Separate the locking mechanism using your thumb and forefinger and push down firmly until the vial access device is fully seated and locked onto the vial.

2. Attach Spiros (CH2000S)



- › Attach Spiros to the end of a syringe, rotating Spiros until it “clicks” and spins freely.
- › To ensure proper connection, confirm Spiros spins freely.

3. Swab



- › Swab Clave using aggressive circular motion for 3 seconds or per facility protocol.

4. Reconstitution (Optional)



- › If required, reconstitute per manufacturers instructions.
- › Place the vial in the upright position on a flat surface.
- › Connect and slowly infuse diluent into the vial.

- › To minimize a blocked filter, never infuse fluid or air into vial when inverted.

5. Withdraw



- › Invert to withdraw fluid from the vial.

Storage Tip:

Always store vial in upright position on flat surface to prevent filter occlusion or leaking.