mylife™ Control KONTROLLÖSUNG

DE

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EN

Intended Use

With the mylife™ Control control solution you can perform a quality control check of the mylife™ Pura® blood glucose meter and mylife™ Pura® blood glucose test strips. If the test result is in line with the control solution range the mylife™ Pura® blood glucose monitoring system is working correctly. The control solution range is printed on the test strip vial.

The mylife™ control solution is intended for in vitro diagnostics (outside the body) and for self testing.

In this user manual we hereafter refer to mylife™ Pura® blood glucose test strips as test strips, the mylife™ Pura® blood glucose meter as meter, the mylife™ Pura® blood glucose monitoring system as BGMS and the mylife™ control solution as control solution.

Summary and explanation

Control solutions are red liquids containing different levels of glucose (low, normal, high) that react with the test strips of the BGMS. When used following the same procedure as whole blood, the control solution should give results within the expected range printed on the test strip vial. Control solution should be used whenever you want to check if your BGMS is working correctly. The use of control solution is also recommended for practice testing and to check correct procedure.

Chemical composition

The control solution is an aqueous liquid which contains the ingredients below:

| Level | Low | Normal | High |
|---------------------------|----------------------------------|----------------------------------|----------------------------------|
| Ingredient | Approximate Concentration (w/v%) | Approximate Concentration (w/v%) | Approximate Concentration (w/v%) |
| Water | 85% | 84% | 69% |
| d-Glucose | 0.05% | 0.1% | 0.3% |
| Viscosity enhancing agent | 13.2% | 15.0% | 29.5% |
| Inorganic salts/Buffers | 1.8% | 1.0% | 1.0% |
| Dye | 0.08% | 0.08% | 0.08% |
| Preservative | 0.03% | 0.03% | 0.03% |

Precautions

- The control solution is not intended for human consumption.
- Do not dilute the control solution with any liquid.
- Keep the control solution away from children. If the control solution or the vial cap has been swallowed contact your physician immediately.

Storage and handling

- Store your unopened control solution at temperature between 2°C to 30°C (36°F to 86°F).
- Shake the control solution vial before use.
- When you open a new bottle of control solution, please write the opening date on the label. Control solution is good for 6 months after opening the bottle, or until the expiration date printed on the label (whichever comes first).
- Do not refrigerate.
- Check the expiration date and opening date before use of the control solution.
- Replace the vial cap of the control solution and close tightly right away after use.

Procedure

You can find detailed instructions for the performance of a quality control test with the control solution in the instructions for use of the BGMS. To perform a quality control test with the control solution you need the mylife™ Pura® meter, mylife™ Pura® test strips and mylife™ control solution.

Understanding control test results

The results of the quality control test should be within the specified control solution range. This range is printed on the test strip vial label. If the results are within the specified ranges, your meter and test strip work correctly.

Example of control solution range

**Control solution range
79 - 107 mg/dL
4.4 - 5.9 mmol/L**

Attention:

The values of the control solution mentioned in this leaflet are only for demonstration purpose. Please always refer to the values printed on the label of your test strip vial.

If control solution test results are out of the control range, your BGMS may not be working correctly. Repeat the quality control test considering these instructions and the user manual of the BGMS.

Do not use the BGMS to test your blood glucose if the control solution test results are still outside the range printed on the test strip vial.

The reason your control results could be out of the range:

- Your control solution is expired or has been opened more than 3 months ago.
- Your control solution has been diluted.
- Your test strip is expired. Check the expiration date on the vial of test strips.
- The used test strip is damaged.
- Your control solution or test strip vial was not properly closed or left open for a long time.
- You did not perform the test procedure correctly.
- Malfunction of the meter.
- Your control solution test has been performed out of normal temperature range (< 10°C and > 40°C resp. < 50°F and > 104°F).

Customer service

Please review all the instructions to make sure you are performing the steps correctly. If you have any questions or in case of problems with the mylife™ Pura® products, please contact your local Ypsomed customer service.

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Distribution UK:

Ypsomed Ltd., 1 Park Court, Riccall Road, Escrick, North Yorkshire, UK-YO19 6ED
Customer care: 0344 8567820, www.mylife-diabetescare.co.uk

Distribution India:

Ypsomed India Private Ltd., 10/61/1-F, Kirti Nagar Industrial Area, IN-New Delhi-110015
Customer care: 099999 75157, www.mylife-diabetescare.co.in

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Ypsomed Oy, Eteläinen Salmite 1, FI-02430 Masala
Asiakaspalvelu: +358 (0) 800 89998, www.mylife-diabetescare.fi

Distributør Danmark:

Ypsomed ApS, Smedeland 7, DK-2600 Glostrup
Kundeservice: +45 48 24 00 45, www.mylife-diabetescare.dk

BIONIME CORPORATION
No. 100, Sec. 2, Daqing St.,
South Dist., Taichung City,
40242, Taiwan

Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands
EmergoVigilance@ul.com

Manufactured for
Ypsomed AG
3401 Burgdorf, Switzerland
www.mylife-diabetescare.com

0197

10207523/V01
101-36/C560-0L1
de+fr+it+en+sv+fi+da

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Ypsomed S.A.S., 44 rue Lafayette, FR-75009 Paris