heel^{up}®

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Product description

The Heel Up® is designed to give complete heel pressure relief.

The Heel Up® is shaped with a deep channel to accommodate the Achilles tendon and with high sides to support the foot. Heel Up is an optimal positioning device used when bedbound patients require comfortable and stable heel pressure relief, whilst facilitating a certain amount of mobility. Even with the slightest movement, the Heel Up® will stimulate the microcirculation of the skin, which optimises circulation.

Heel Up® is made of a non-woven material, which makes the device soft, skin-friendly, heat and moisture absorbent, providing comfortable, optimal pressure relief.

Advantages

Single-patient-use ensures high optimal hygiene levels.

Easy to inflate and fit.

Made from a comfortable soft, skin-friendly material, which both absorbs moisture and transfers heat.

Comes in a clean, sealed flat package which includes an oral inflation device.

Tested and approved in accordance with ISO 10993-10:2010, test for irritation and hypersensitivity.

Indications / Purpose

Decubitus prophylaxis and therapy.
Existing pressure ulcer.
Decubitus risk.

Application Risks / Contraindications

Heel Up® can be used in beds with adjustable reclining surfaces.

When using the Heel Up® in beds with side guards, the use of Lateral grid pads are recommended to avoid pinching the foot in the side guard.

Be sure to follow guidelines to avoid shear.

Permissible operating conditions / locations

At the foot (heels, ankles)

Heel Up® can be used for the same patient for up to 8 weeks. Heel Up® is designed as a 'single-patient' only product and is therefore not to be reused on another patient. Single-patient use eliminates the risk of a possible bacterial spread between patients.

Cleaning

Note!

Do not clean with steam or in an autoclave. The product does not withstand extreme heat ..

Note!

Do not use caustic or abrasive cleaners or sharp objects to clean Heel Up®.

Use and washing results in wear on the outer fiber layer.

If the outer fiber layer is no longer intact or if Heel Up® loses air, Heel Up® can not be used and can be disposed of with ordinary waste*.

When Heel Up® is pumped up without Stop Up® sleeve, follow the instructions on page 5

Cleaning of Heel Up®:

Let out the air before washing.

Heel Up® can be washed in the machine or manually at 60-65 ° C (max 65 ° C)

Remove stubborn stains with a damp cloth, use a mild anti-allergic detergent.

Before using Heel Up®, it must be completely dry. Do not dry it in a tumble dryer.

Heel Up® disinfection:

If Heel Up® has been contaminated by body fluids, it can be disinfected to prevent infection.

Heel Up® can be sprayed with a suitable disinfectant or with alcohol / propyl alcohol on a damp cloth for wiping.

Heel Up® can then be washed in the washing machine or manually as described above, max. 65 ° C.

Before using Heel Up®, it must be completely dry. Do not dry it in a tumble dryer.

Guarantee

* The warranty is limited and covers only manufacturing defects. A non-intended use of the product will invalidate the warranty. Normal wear is excluded from the warranty.

GB

Inflation and adjustment See pictograms at the back

Heel Up® has been developed by Levabo® to prevent pressure ulcers and to support the healing process.

Please also ensure that the heel area is examined regularly and in accordance with the applicable clinical guidelines.

Heel Up® is designed for bedbound patients with limited mobility.

Carefully remove the Heel Up® from its packaging. Let the encircling Stop Up sleeve stay on the Heel Up.

Place the Heel Up® on a flat surface. (Fig. 1). The Stop Up® sleeve and air valve should point towards you. Open the white Velcro fastener (F).

To inflate:

Hold the device valve between the finger and thumb (D), insert the small blue pump nozzle (A) firmly until it is gently gripped. Insert the pump or oral inflation device (that comes with the product) (B) in the pump nozzle (FIG. 1). When using a Levabo manual pump the pump nozzle (A) is not needed because this pump fits directly into the valve.

Use the pump (Fig. 2) or the oral inflation device to inflate the Heel Up®.

When the sufficient air volume is reached, the Stop Up® sleeve bursts. Clearly visible (FIG. 3).

Stop pumping. Remove the pump and the pump connector (Fig.3).

Once you have inflated the Heel Up® to the required level close the valve (FIG. 4) and press the blue circular adhesive patch (C), over the valve cap (FIG. 6).

Open the Heel Up® by slightly bending the side walls and the bottom wall outwards.

Place the foot into the device (Fig. 5). Now close the white Velcro fastener (F).

Place the colored Velcro (G) around the calf and close the colored Velcro.

When using Velcro fasteners, make sure that the foot and support are tightly closed but not pinched.

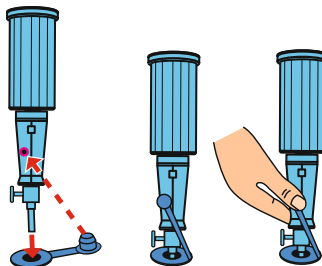
Then check the lifting height of the heel with your hand (Fig. 5). See note below.

To Deflate:

Hold the device valve between finger and thumb (E), insert the small blue pump nozzle (A) to release the air. When using a Levabo manual pump the pump nozzle (A) is not needed because a vent button is built into the pump (E).

Tip before inflation

Insert the pump into the valve.
Insert the closure plug into the hole on the pump handle.
Pump up.

If there is no hole, you can hold the closure plug with a finger.

GB

IMPORTANT NOTE:

The Heel Up® must not be over inflated. (Figure 5)! For optimal and comfortable positioning it should be soft and pliable, but not hard. The Heel Up® short, medium or long should be about 25 mm (two fingers) above the supporting surface, or 28-30 mm for the Heel Up® max (FIG. 5).

Check whether the lifting height is correct on a daily basis.

If inflating without a stop Up sleeve, it is important to follow the guidelines given above.

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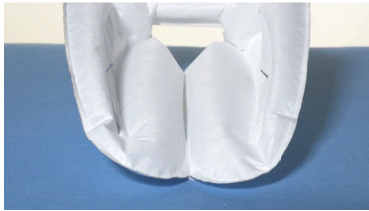
FR

The outer surface provides superior drainage of moisture and heat from the skin, it also ensures the skin doesn't stick to the cushion. It can be wiped with a damp cloth or alcohol.

The sail and air under the foot supports the front foot and minimizes the risk of foot drop.

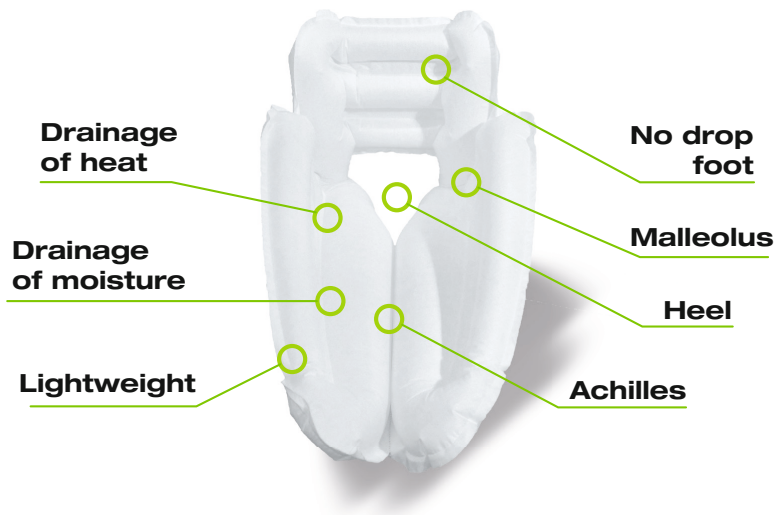
There is room for the Achilles tendon between the 2 bearing cells. The cells are beveled so that the lower leg and knee is not unnecessarily burdened.

It is open to the heel and there is room for the malleoli to ensure full relief.

Heel Up® has some important features that provide a number of advantages

Unfold Heel Up® and insert the pump into the self-closing blue valve until it is tight.

Pump up...

Inflate Heel Up® until the enveloping paper strap - Stop Up® jumps - and remove the pump.
Heel Up® is now ready to use.

Place Heel Up® on the patient's foot. Close the ribbon across the opening.

Check that the heel is raised about 1½ finger width from the ground.

Done...

**From zero to pressure relief
in less than 60 seconds....**

Visit
www.levabo.com
for more indepth product information.

Product videos and instructions. 

Anti decubitus information on **academy**

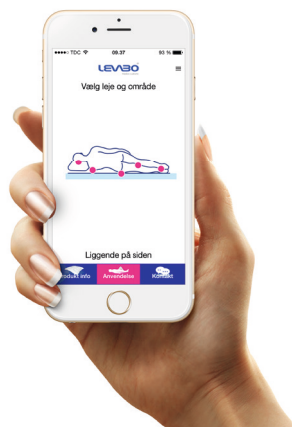
Home Care information on **dcubi kit**

Download:
Braden Scale Scheme,
product data sheets,
anti decubitus poster
and Levabo mobile app.

LEVABO[®]
medical cushions

14

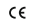
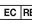
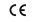
Heel Up® -Range of Heel Lifters

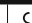
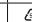
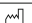
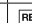
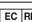
Product name	Size	Packaging	Prod-uct no.	Weight	Air cells	Operating / Storage temp.	Materials
Heel Up® Short	18 x 32 x 25 cm	The Heel Up® comes in an air-tight sealed package which includes a valve set, oral inflation device	70001	80 g	1	0-40° C	100% PE
Heel Up® Medium	18 x 39 x 25 cm		70007	90 g			
Heel Up® Long	18 x 48 x 25 cm		70002	105 g			
Heel Up® Max	18 x 50 x 30 cm		70018	115 g			
Heel Up® Max Medium	18 x 39 x 30 cm		70035	95 g			
Heel Up® FIX Medium	18 x 39 x 25 cm		70073	95 g			
Heel Up® FIX Long	18 x 48 x 25 cm		70079	110 g			
Heel Up® AIR Medium	18 x 39 x 25 cm		70036	90 g			
Heel Up® CUT Medium	18 x 39 x 25 cm		70037	90 g			

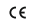
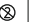
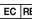
Heel Up -Accessories

Product name	Size in cm.	Packaging	Item no.	Air cells	Operating and storage
Velcrostrap	50x075	1 pcs	76011	N/A	0-40° C
Pump nozzle kit	12 x 18	10 pcs	76005		

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Explanation of the symbols on the packaging				
	Read the instructions		Expiry date	 Declaration of conformity
	Batch number		Single patient product	 Material in the packaging
	Manufacturer		Manufacturing date	 Order number
	Authorized representative in the European Community			

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Declaration of Conformity (CE)

The company Levabo ApS, Sverigesvej 20A, 8660 Skanderborg, declares that the products are designed and manufactured in accordance with the technical documentation and legal requirements.