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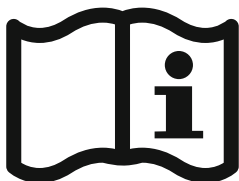
# Product description

The Seat All® cushions are designed to give complete relief in prevention and treatment of pressure ulcers.

The cushions ensures stability without hindering the patient's movements. Even with the slightest movement, the cushion will stimulate the microcirculation of the skin, which optimises circulation.

Seat All® is made of a non-woven material, which makes the device soft, skin-friendly, heat and moisture absorbent, providing comfortable, optimal pressure relief.

## Indications / Purpose

Decubitus prophylaxis and therapy.  
Existing pressure ulcer.  
Decubitus risk.

## Application Risks / Contraindications

Seat All® can be used in beds or chairs with or without adjustable reclining surfaces.

When using the Seat All® in beds or chairs with adjustable surfaces.

Be sure to follow guidelines to avoid shear.

## Permissible operating conditions / locations

Seat All® can be used everywhere on the body.

Seat All® can be used for the same patient for up to 8 weeks.

Seat All® is designed as a 'single-patient' only product and is therefore not to be reused on another patient.

Single-patient use eliminates the risk of a possible bacterial spread between patients.

# Inflation and adjustment

See pictograms at the back

The Seat All® cushions is designed to prevent and treat pressure ulcers.

The cushions are for single patient use only.

Please also ensure that the treated area is examined regularly and in accordance with the applicable clinical guidelines. When using Seat all with other products, please read the instructions for the other products.

Carefully remove the cushion from the outer packaging.

Let the encircling Stop Up sleeve stay on the cushion.

## To inflate:

Hold the device valve between the finger and thumb (D), insert the small blue pump nozzle (A) firmly until it is gently gripped. Insert the pump or oral inflation device (that comes with the product)(B) in the pump nozzle and start pumping (FIG. 1). When using a Levabo manual pump the pump nozzle (A) is not needed because this pump fits directly into the valve.

Then inflate to the required level. Please note that the device comes with a card sleeve (Stop Up sleeve) which bursts. Clearly visible (FIG. 3) as the device is inflated to a recommended level.

The device should be inflated to a level to suit the needs of your patient.

Once you have inflated the cushion to the required level close the valve (FIG. 4) and press the blue circular adhesive patch (C), over the valve cap (FIG. 6).

## To deflate:

Hold the device valve between finger and thumb (E), insert the small blue pump nozzle (A) to release the air. When using a Levabo manual pump the pump nozzle (A) is not needed because a vent button is built into the pump (E).

**GB**

### IMPORTANT NOTE:

The Seat All® must NOT be over inflated. (Figure 5)

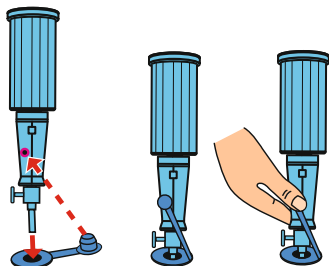
For optimal and comfortable positioning it should be soft and pliable, but not hard. The body part should be approximately 25 mm (two fingers) above the supporting surface.

- Check whether the lifting height is correct on a daily basis.

If inflating without a stop Up sleeve, it is important to follow the guidelines given above.

## Tip before inflation

Insert the pump into the valve.  
Insert the closure plug into the hole on the pump handle.  
Pump up.

If there is no hole, you can hold the closure plug with a finger.

## Cleaning

### Note!

Do not clean with steam or in an autoclave. The product does not withstand extreme heat ..

### Note!

Do not use caustic or abrasive cleaners or sharp objects to clean Seat All®.

Use and washing results in wear on the outer fiber layer.

If the outer fiber layer is no longer intact or if Seat All® loses air,

Seat All® can not be used and disposed of with ordinary waste \*.

When Seat All® is pumped up without Stop Up® sleeve, follow the instructions on page 4.

### Cleaning of Seat All®:

Let out the air before washing.

Seat All® can be washed in the machine or manually at 60-65 ° C (max 65 ° C)

Remove stubborn stains with a damp cloth, use a mild anti-allergic detergent.

Before using Seat All®, it must be completely dry. Do not dry it in a tumble dryer.

### Seat All® disinfection:

If Seat All® has been contaminated by body fluids, it can be disinfected to prevent infection.

Seat All® can be sprayed with a suitable disinfectant or with alcohol / propyl alcohol on a damp cloth for wiping.

Seat All® can then be washed in the washing machine or manually as described above, max. 65 ° C.

Before using Seat All®, it must be completely dry. Do not dry it in a tumble dryer.

## Guarantee

\* The warranty is limited and covers only manufacturing defects. A non-intended use of the product will invalidate the warranty. Normal wear is excluded from the warranty.

**GB**

Explanation of the symbols on the packaging					
	Read the instructions		Expiry date		Declaration of conformity
	Batch number		Single patient product		Material in the packaging
	Manufacturer		Manufacturing date		Order number
	Authorized representative in the European Community				

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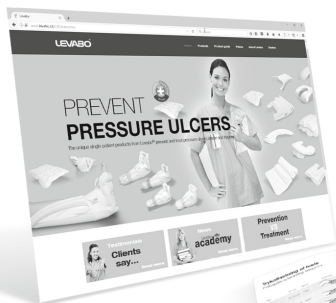
Visit

[www.levabo.com](http://www.levabo.com)

for more indepth product information.

Product videos and instructions. 

Anti decubitus information on  **academy**

Home Care information on  **dcubi kit**

Download:

Braden Scale Scheme,  
product data sheets,  
anti decubitus poster  
and Levabo mobile app.

**LEVABO**<sup>®</sup>  
medical cushions

## Seat All® -Range of positioning cushions

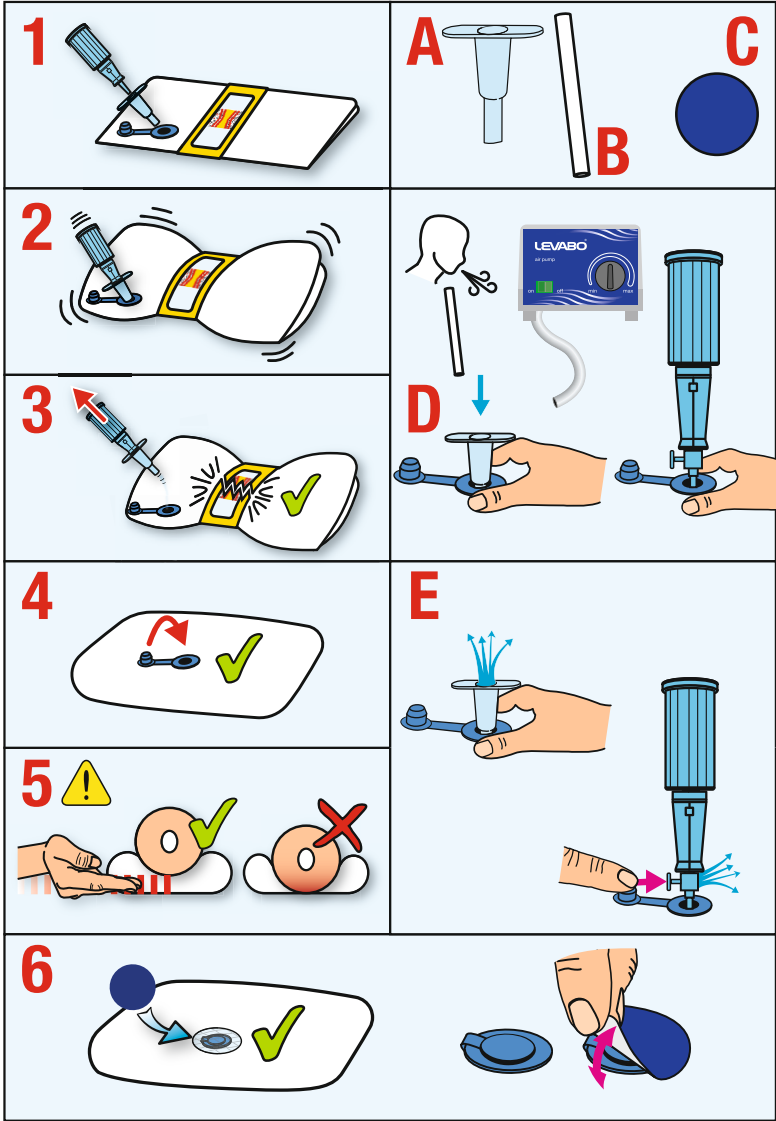
Product name	Size in cm.	Stop Up Sleeve	Packaging	Item no.	Weight	Air cells	Operating and storage	Materials
Seat All® Seat	42 x 42 x 5	+	Seat All® comes in an air-tight sealed package which includes a valve set, and oral inflation device.	70008	115 g	1	0-40° C	100% PE
Seat All® Seat Large	52 x 52 x 5	+		70019	180 g	1		
Seat All® Seat Max	52 x 62 x 5	+		70020	220 g	1		
Seat All® Seat Twin	48 x 52 x 5	-		70021	115 g	1		
Seat All® Seat Tailbone	42 x 42 x 5	+		70022	115 g	2		
Seat All® Back & Seat	27 x 45 x 5	-		70009	80 g	1		
Seat All® Donut	45 x 45 x 8	+		70017	115 g	1		
Seat All® Donut Small	35 x 35 x 8	+		70055	100 g	1		
Seat All® Bath	50 x 50 x 8	-		70013	115 g	1		
Seat All® Flexi	42 x 42 x 5	+	70070	125 g	1			

## Seat All -Accessories

Product name	Size in cm.	Packaging	Item no.	Air cells	Operating and storage	Materials
Seat cover	42 x 42	5 pcs	76009	N/A	0-40° C	100% PE
Seat cover	52 x 52	5 pcs	76010			
Pump nozzle kit	12 x 18	10 pcs	76005			

## Declaration of Conformaty (CE)

The company Levabo ApS, Sverigesvej 20A, 8660 Skanderborg, declares that the products are designed and manufactured in accordance with the technical documentation and legal requirements.

UNI userguide\_SA\_2019\_01\_15