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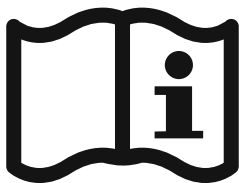
Product description

The All Up® cushions are designed to give complete relief in prevention and treatment of pressure ulcers.

The cushions ensures stability without hindering the patient's movements. Even with the slightest movement, the cushion will stimulate the microcirculation of the skin, which optimises circulation.

All Up® is made of a non-woven material, which makes the device soft, skin-friendly, heat and moisture absorbent, providing comfortable, optimal pressure relief.

Indications / Purpose

Decubitus prophylaxis and therapy.

Existing pressure ulcer.

Decubitus risk.

Application Risks / Contraindications

All Up® can be used in beds or chairs with or without adjustable reclining surfaces.

When using the All Up® in beds or chairs with adjustable surfaces,

be sure to follow guidelines to avoid shear.

Permissible operating conditions / locations

All Up® can be used everywhere on the body.

All Up® can be used for the same patient for up to 8 weeks.

All Up® is designed as a 'single-patient' only product and is therefore not to be reused on another patient.

Single-patient use eliminates the risk of a possible bacterial spread between patients.

Inflation and adjustment

See pictograms at the back

The All Up® cushions is designed to prevent and treat pressure ulcers.

The cushions are for single patient use only.

Please also ensure that the treated area is examined regularly and in accordance with the applicable clinical guidelines.

Carefully remove the cushion from the outer packaging.

Let the encircling Stop Up sleeve stay on the cushion.

To inflate:

Hold the device valve between the finger and thumb (D), insert the small blue pump nozzle (A) firmly until it is gently gripped. Insert the pump or oral inflation device (that comes with the product)(B) in the pump nozzle and start pumping (FIG. 1). When using a Levabo manual pump the pump nozzle (A) is not needed because this pump fits directly into the valve.

Then inflate to the required level. Please note that the device comes with a card sleeve (Stop Up sleeve) which bursts. Clearly visible (FIG. 3) as the device is inflated to a recommended level.

The device should be inflated to a level to suit the needs of your patient.

Once you have inflated the cushion to the required level close the valve (FIG. 4) and press the blue circular adhesive patch (C), over the valve cap (FIG. 6).

To deflate:

Hold the device valve between finger and thumb (E), insert the small blue pump nozzle (A) to release the air. When using a Levabo manual pump the pump nozzle (A) is not needed because a vent button is build into the pump (E).

GB

IMPORTANT NOTE:

The All Up® must NOT be over inflated. (Figure 5)!

For optimal and comfortable positioning it should be soft and pliable, but not hard.

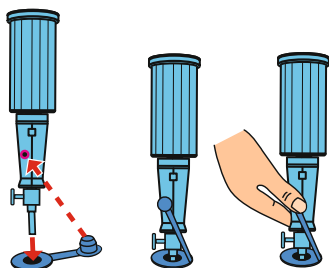
The body part should be approximately 25 mm (two fingers) above the supporting surface.

Check whether the lifting height is correct on a daily basis.

If inflating without a stop Up sleeve, it is important to follow the guidelines given above.

Tip before inflation

Insert the pump into the valve.
Insert the closure plug into the hole on the pump handle.
Pump up.

If there is no hole, you can hold the closure plug with a finger.

Cleaning

Note!

Do not clean with steam or in an autoclave. The product does not withstand extreme heat ..

Note!

Do not use caustic or abrasive cleaners or sharp objects to clean All Up®.

Use and washing results in wear on the outer fiber layer.

If the outer fiber layer is no longer intact or if All Up® loses air, All Up® can not be used and can be disposed of with ordinary waste*.

When All Up® is pumped up without Stop Up® sleeve, follow the instructions on page 4

Cleaning of All Up®:

Let out the air before washing.

All Up® can be washed in the machine or manually at 60-65 ° C (max 65 ° C)

Remove stubborn stains with a damp cloth, use a mild anti-allergic detergent.

Before using All Up®, it must be completely dry. Do not dry it in a tumble dryer.

All Up® disinfection:

If All Up® has been contaminated by body fluids, it can be disinfected to prevent infection.

All Up® can be sprayed with a suitable disinfectant or with alcohol / propyl alcohol on a damp cloth for wiping.

All Up® can then be washed in the washing machine or manually as described above, max. 65 ° C.

Before using All Up®, it must be completely dry. Do not dry it in a tumble dryer.

Guarantee

* The warranty is limited and covers only manufacturing defects. A non-intended use of the product will invalidate the warranty. Normal wear is excluded from the warranty.

GB

Explanation of the symbols on the packaging					
	Read the instructions		Expiry date		Declaration of conformity
	Batch number		Single patient product		Material in the packaging
	Manufacturer		Manufacturing date		Order number
	Authorized representative in the European Community				

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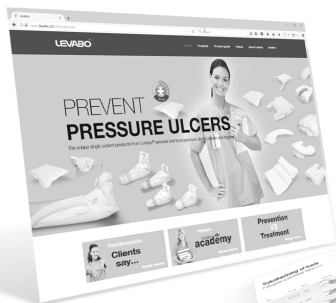
Visit

www.levabo.com

for more indepth product information.

Product videos and instructions. 

Anti decubitus information on  **academy**

Home Care information on  **dcubi kit**

Download:

Braden Scale Scheme,
product data sheets,
anti decubitus poster
and Levabo mobile app.

LEVABO[®]
medical cushions

All Up® -Range of positioning cushions


Product name	Size in cm.	Stop Up Sleeve	Packaging	Item no.	Weight	Air cells	Operating and storage	Materials
All Up® Multi Small	50 x 25 x 8	+	All Up® leveres i en forsejlet pakke, som indeholder et ventil sæt og pusterør.	70003	75 g	1	0-40° C	100% PE
All Up® Multi Medium	50 x 37 x 8	+		70004	100 g	1		
All Up® Multi Large	50 x 50 x 8	+		70005	145 g	1		
All Up® Multi Max	50 x 50 x 20	-		70012	240 g	1		
All Up® Universal	50 x 50 x 8	+		70010	135 g	1		
All Up® Universal Long	55 x 70 x 8	+		70015	150 g	1		
All Up® Universal Slim	45 x 75 x 8	-		70016	130 g	1		
All Up® Head	28 x 30 x 5	+		70011	50 g	1		
All Up® Head Fix	30 x 35 x 6	+		70057	55 g	1		
All Up® Head Intubate	29 x 32 x 5	-		70058	55 g	1		
All Up® Head Tube	29 x 32 x 5	-		70056	55 g	1		
All Up® Wing	50 x 85 x 8	+		70006	330 g	1		
All Up® Wedge 30 deg.	70 x 40 x 15	-		70024	290 g	1		
All Up® Multi Wedge	50 x 23 x 23	-		70053	130 g	1		
All Up® Lap	80 x 60 x 10	-		70025	165 g	1		
All Up® Lap Max	80 x 60 x 20	-		70026	180 g	1		
All Up® Arm Rest	55 x 25 x 8	-	70027	60 g	1			
All Up® Elbow	30 x 20 x 10	-	70028	80 g	1			
All Up® Elbow Medium	23 x 20 x 10	-	70048	70 g	1			
All Up® Elbow & Heel	16 x 19 x 10	-	70049	60 g	1			
All Up® Seat	42 x 42 x 5	+	Luftdichte Beutel mit einem Ventil-Set und einem Blasrohr.	70008	115 g	1		
All Up® Seat Large	52 x 52 x 5	+		70019	180 g	1		
All Up® Seat Max	52 x 62 x 5	+		70020	220 g	1		
All Up® Seat Twin	48 x 52 x 5	-		70021	115 g	1		
All Up® Seat Tailbone	42 x 42 x 5	+		70022	115 g	2		
All Up® Back & Seat	27 x 45 x 5	-		70009	80 g	1		
All Up® Donut	45 x 45 x 8	+		70017	115 g	1		
All Up® Donut Small	35 x 35 x 8	+		70055	100 g	1		
All Up® Bath	50 x 50 x 8	-		70013	115 g	1		
All Up® Belly	65 x 75 x 15	-		70023	225 g	1		
All Up® Kidney	50 x 20 x 20	-	70014	70 g	1			
All Up® Kidney Slim	50 x 16 x 16	-	70050	65 g	1			

All Up -Accessories

Product name	Size in cm.	Packaging	Item no.	Air cells	Operating and storage	Materials
Seat cover	42 x 42	5 pcs	76009	N/A	0-40° C	100% PE
Seat cover	52 x 52	5 pcs	76010			
Overlay	50 x 50	5 pcs	76012			
Overlay	50 x 100	5 pcs	76013			
Velcrostrap	50x075	1 pcs	76011			
Pump nozzle kit	12 x 18	10 pcs	76005			

Declaration of Conformity (CE)

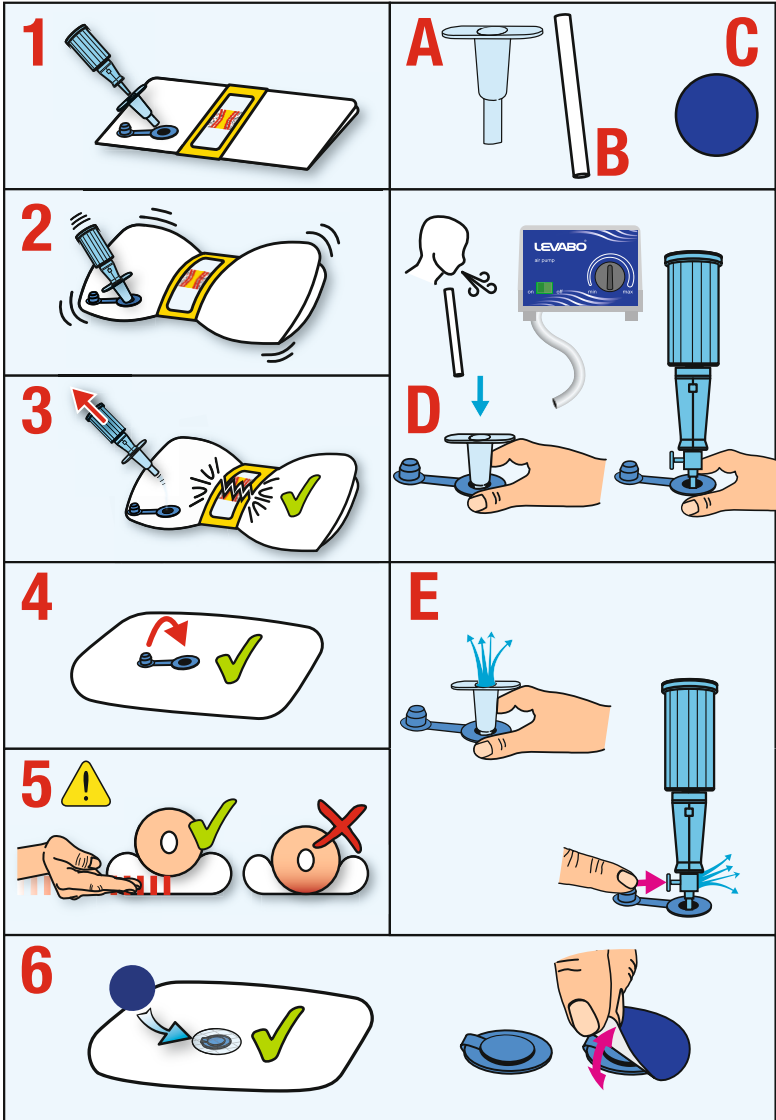The company Levabo ApS, Sverigesvej 20A, 8660 Skanderborg, declares that the products are designed and manufactured in accordance with the technical documentation and legal requirements.

UNI Userguide AU 2019 01_15