Product description

Cutimed® Silitec® PLUS dressings are an assortment of silicone coated foam dressings. Cutimed® Silitec® PLUS is a sterile, absorbent polyurethane foam dressing for atraumatic dressing changes that contains superior-absorbent strips which absorb and lock wound exudate. The wound contact surface is a perforated silicone layer that allows adherence to the per-wound skin but not to the moist wound bed or to newly formed epithelial tissue. This minimizes trauma and pain during dressing changes. The outer film is water-repellent, yet permeable to oxygen and vapour. The film dressing and its top-surface absorbent strips are designed to absorb and to lock away excess wound fluid inside the dressing, preventing the dressing from becoming saturated and the risk of maceration. Cutimed® Silitec® PLUS is highly conformable and provides protection of the wound site. Cutimed® Silitec® PLUS leaves no residues in the wound. The dressing reliably retains wound exudate, even under compression. If necessary, Cutimed® Silitec® PLUS may be cut to size. Cutimed® Silitec® PLUS gently adheres to the per-wound skin and stays in place while secondary fixation is applied.

Indications

Cutimed® Silitec® PLUS assortment is indicated for exuding wounds such as: venous and arterial ulcers, pressure ulcers, diabetic gangrenes, surgical incisions, skin grafts and donor sites, lacerations or abrasions. Cutimed® Silitec® PLUS is recommended for wounds with low to high exudate levels.

Contraindications

Do not use on patients with a known sensitivity to any of the product components.

Warnings and precautions

Cutimed® Silitec® PLUS is packaged for single use. Do not re-use or re-sterilise as there is a risk of transmission of body fluids or contaminated tissue between patients. Do not use if pouch is open or damaged. Discard after or when the pouch is empty. The wound should be inspected for signs of infection or treated according to clinical practices if required.

For external use only. Do not use with oily dressings. Do not use with hydrogel products. Do not use with hydrogel products. Do not use with hydrogel products. Do not use with hydrogel products.

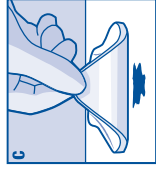
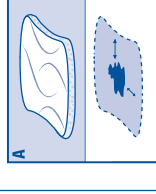
Directions for use

1. Clean the wound according to good clinical practice.
2. Make sure, the surrounding skin is dry.
3. Choose an appropriate dressing size that covers the wound and overlaps the wound edges. **(A)**
4. Open the pouch and take out the dressing. Ensure sterility.
5. Remove protective film **(B)** and apply with the adherent side towards the wound. **(C)**
6. Avoid stretching the dressing.
7. Apply additional dressing fixation.
8. If necessary, Cutimed® Silitec® PLUS can be used in combination with other dressings e.g. Cutimed® Sorbact® (in case of infected wounds)
9. Cutimed® Silitec® PLUS can be used on an infected wound in addition to antimicrobial therapy under medical supervision.
10. Cutimed® Silitec® PLUS can remain in place for up to seven days, depending on wound conditions and exudate levels. It is recommended to change the dressing every 24 hours initially, moving to less frequent changes as necessary.
11. Carefully remove any additional dressing fixation and gently remove Cutimed® Silitec® PLUS from the wound and discard.
12. If required, cleanse the wound prior to application of a new dressing.

Storage

Cutimed® Silitec® PLUS should be stored in dry conditions. Do not expose to direct sunlight.

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