**BAIRD**

**Pop-On**  
Self-Adhering Male External Catheter

**WideBand**  
Self-Adhering Male External Catheter

**SPIRIT**  
Style 2 / Style 3  
Hydrocolloid Adhesive Sheath

**DESCRIPTION/INDICATION**  
The self-adhering male external catheter is designed for the management of male urinary incontinence.

**CONTRAINDICATION**  
Do not use on irritated or compromised skin.

**PRECAUTION**  
Do not use if allergic reaction occurs. For good hygiene, change catheter daily. Use of a single device for longer periods than 24 hours may increase the risk of complications.

**DIRECTIONS TO APPLY**

- 1) Wash penis with mild soap and warm water. Dry thoroughly.
- 2) Trim pubic hair if necessary.
- 3) Open package at perforation.
- 4) To remove plastic insert, squeeze catheter at the top of the white cone and pull to release.

5) Urinal self-adhering catheter over penis.

6) Gently squeeze the catheter to properly seal adhesive to the skin.

7) Connect to collection bag.

**IMPORTANT:**  
Wear time may be significantly reduced if adhesive is not properly sealed to the skin.

**DIRECTIONS TO REMOVE**  
Gently roll catheter off the penis.

Note: If necessary, apply a warm wet compress (such as a wet washcloth) around the catheter to help loosen the adhesive.

**REF** Catalog Number

**LOT** Lot Number

**EC REP** Authorized Representative

**USA** Use by

**NR** Not made with natural rubber latex

**Manufacturer:**  
Rochester Medical Corporation  
One Rochester Medical Drive  
Stearnsville, MN 55976 USA  
(800) 528-4455 www.bairdcare.com

**CAUTION** consult accompanying documents

**CE**

Baird, Pop-On, Spirit and WideBand are trademarks and/or registered trademarks of C. R. Bard Inc. ©2015 C. R. Bard Inc. All rights reserved. PCH04701 10/2015

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Baird, Pop-On, Spirit, WideBand are trademarks or registered trademarks of R. Baird, Inc. ©2015 © R. Baird, Inc. Tıkakları kaldırarak kullanılmamalıdır.
© Baird, Pop-On, Spirit, WideBand are trademarks or registered trademarks of R. Baird, Inc. ©2015 © R. Baird, Inc. Tıkakları kaldırarak kullanılmamalıdır.