TRACHI-BRUSH®

Rensbørster til tracheostomirør

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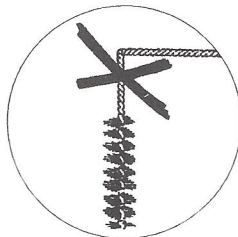
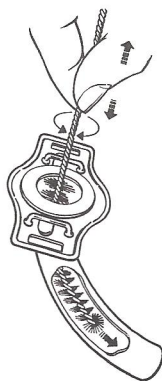
30

DO NOT CLEAN THE TRACHEOSTOMY TUBE WHILE IN SITU

Contra-indications: Do not perform cleaning while tracheostomy tube and/or inner cannula are in the patients tracheostoma. Always ensure the tracheostomy tube and inner cannula are clear before inserting into the patients tracheostoma.

Storage: Store in a cool, dry place, out of sunlight.

Disposal: In general household waste in a home environment & clinical waste in a healthcare environment.

To report a Serious Incident: Contact Kapitex Healthcare Ltd and the Competent Authority in your country of use.

Contact a healthcare professional: If you are unsure how to use the device after reading the instructions for use.

TRACHI-BRUSH®

INSTRUCTIONS FOR USE

Disclaimer: Instructions for use may be periodically updated, always consult them prior to each use.

Description: Trachi-Brush is a cleaning brush for tracheostomy/laryngectomy tubes. It is a wire brush with polypropylene bristles and a plastic handle.

Intended use: Trachi-Brush is indicated for use in patients who have a tracheostoma and use a device to maintain stoma patency. Trachi-Brush is intended for cleaning (removing mucus and debris) from Tracheostomy/Laryngectomy tubes and Inner Cannulas after removal from the patient. These devices shall be further referred to generically as 'tracheostomy tubes' in this instruction sheet.

1. Carefully remove the inner cannula from the tracheostomy tube or remove the complete tracheostomy tube from the patient's tracheostoma.

DO NOT CLEAN IN SITU.

2. Always refer to the tracheostomy tube manufacturer's Instructions for Use for cleaning instructions. Please follow device manufacturer's cleaning instructions. For a general guide follow points 3 & 4.

3. Gently insert the Trachi-Brush into the tracheostomy tube.

4. Using a sliding motion in the tracheostomy tube, remove mucus and debris with the Trachi-Brush.

5. Always ensure the tracheostomy tube and inner cannula are clear before inserting into the patient's tracheostoma.

6. Trachi-Brush may be refreshed by hand washing in warm (30°C) and dried thoroughly before next use for up to 30 times. Discard if damaged, soiled (phlegm or saliva) or shows signs of wear and tear.

WARNINGS AND PRECAUTIONS:

- Do not clean the Tracheostomy tube while in situ.
- Do not attempt to sterilise Trachi-Brush.
- Do not use Trachi-Brush if it is damaged or shows signs of wear and tear.
- Do not excessively bend the shaft of the Trachi-Brush.
- Do not leave brush in any solution.
- Do not pour boiling water over the Trachi-Brush.
- Always ensure the tracheostomy tube and inner cannula are clear before re-use.
- Do not use for any other purpose other than what is stated in the instructions.
- Do not use Trachi-Brush if the packaging is open, or if the device or packaging are damaged.
- Do not use Trachi-Brush on more than one user, since this presents a risk of infection.