Accordian Fold

Accordian Fold

Accordian Fold


Accordian Fold

Accordian Fold

Accordian Fold

For Blood Collection » За ваземан на крв » Pro odbrk krvi » Til blodprøvetagning » Zur Blutentnahme mit Spritze » Para extracción de sangre » Vere vótmeškes » Verinýtveny otamatis varten » Pour le prélèvement sanguin » Za vadenje krvi » Vértételhez » Fyrir blóðsöfnun » Прелиев на крв » 血液採取用 » Kan anyu yuini » Kraujai imti naudojant » Voor bloedafname » For blodprøtning » Do pobiranja krvi » Para coleta de Sangue » Pentru recoltarea sângelui » Для взятия крови » Pre odbrav krvi » Za odvzem krvi » Inför blodprovtagning » Kan Almak için » Для взятия крови » 血液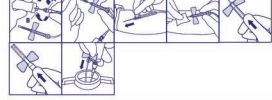

For Blood Collection Using a Syringe » За ваземан на крв с използване на спринцовка » Pro odbrk krvi s použitím stříkačky » Til blodprøvetagning ved brug af en sprøjte » Zur Blutentnahme mit einer Spritze » Para extracción de sangre con una jeringa » Vere vótmeškes sistaia abili » Verinýtveny otamatis stříkačkou » Pour le prélèvement sanguin au moyen d'une seringue » Za vadenje krvi uz pomoć štrcaljke » Vértételhez fecskendő használatával » Fyrir blóðsöfnun með sprautu » Перелиев на крв с използване на шприца » 血液採取用シリンジ » Kraujai imti naudojant švirką » Voor bloedafname met een spuit » For blodprøtning med sprøjte » Do pobiranja krvi za pomoć štrkavčice » Para coleta de Sangue com Seringa » Pentru recoltarea sângelui cu o seringă » Для взятия крови с использованием шприца » 注射器採血

ENGLISH: EN Note: Venipuncture should be performed aseptically. For Blood Collection 1. Check to ensure that the Female Luer Adapter (A) is securely attached to the Male Luer Adapter (B) and that the Male Luer Adapter (B) is securely attached to the holder. 2. Remove needle sheath. 3. Foretæt vespunktur ved at holdevingerne 4. Withdraw Safety-Lok™ Blood Collection Set: Apply pressure to site using your fingers as shown. » Grasp the translucent yellow safety shield grip area with the thumb and index finger while at the same time grasping the tubing. 5. Safety-Lok™ Shield Activation: One-hand technique: Hold tubing in palm with fingers and thumb and advance translucent yellow safety shield until it clicks in place and then hold firmly. 6. Dispose of all used materials in an appropriate container and wash hands.

For Short Term IV Administration (up to 2 hours) 1. Remove Luer cap. 2. Attach IV line or syringe to the Female Luer Adapter. Remove needle sheath. 3. Prime set in accordance with recommended procedure. 4. Perform venipuncture by holding syringe. Administer medication. » It is recommended that this device be changed per the institution's procedures, but no longer than 2 hours. 5. Safety-Lok™ Shield Activation: See Section 5 under Blood Collection (above). 6. Dispose of all used materials in appropriate container and wash hands.

UNIVERSAL PRECAUTIONS: Handle all biological samples and blood collection "kangas" (bags) as if they are HIV-infected, and blood collection sets in accordance with the policies and procedures of your facility. Obtain appropriate medical attention in the event of any exposure to biological samples (e.g., through a puncture injury) such samples may transmit viral hepatitis, HIV (AIDS), or other infectious diseases. Unless any built-in user resealable protector of the blood collection device provides one. Discard all blood collection "kangas" in biohazard containers approved for their disposal.

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Table with 3 columns: Gauge, Tubing (inch), Volume (ml). Rows for 21, 23, 25 gauges.

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