- STERILE SINGLE-USE LANCING DEVICE / STERILE EINMALSTECHHILFE / AUTOPIQUEUR STÉRILE À USAGE UNIQUE / PUNGIDITO STERILE MONOUSO**

**(EN) Sterile single-use lancing device for collection of capillary blood from the fingertip or, if the patient is a child under 1 year, from the heel.**

**Components of the single-use lancing device** (see illustration **●**)

- Description**
- Release button
- Housing
- Penetration depth adjuster
- Sterility cap
- Penetration depth indicator
- Low penetration depth (approx. 1.3 mm)
- Medium penetration depth (approx. 1.8 mm, pre-set)
- High penetration depth (approx. 2.3 mm)

**Preparing to collect blood**

- Children younger than 1 year**

With children younger than 1 year of age, the capillary blood must be collected from the heel. Blood collection must occur from the lateral areas of the sole (see black areas in illustration **●**).

The puncture must occur vertically to the sole (see arrow in the enlargement). During blood collection, the child should be held with its foot vertically facing the floor. A site which has already been punctured must not be punctured a second time.

To improve blood flow at the heel before collecting blood, you can carefully warm the heel, for example by wrapping it in a warm, moist cloth at a maximum 42°C.

The heel must be disinfected and then dried completely.

▲ To collect capillary blood from the heel of children **under 1 year, use only the low or medium penetration depth.** If the high penetration depth is used, the calcaneus (heel bone) is at risk of being injured.

- Children as of 1 year and adults**

We recommend collecting capillary blood from the side of the fingertips, as these puncture sites register the least amount of pain (see black areas in illustration **●**). Any finger can be used for the puncture.

The patient's hands should be washed with warm water and soap, and then dried well. This ensures that the puncture site is clean and stimulates the blood flow. If alcohol wipes are used, the finger must also be completely dry before continuing.

**Collecting blood**

- Removing the sterility cap**

- Twist the sterility cap and remove it.

▲ Do not use the single-use lancing device if the sterility cap has been previously removed.

- Setting the penetration depth**

- Turn the penetration depth adjuster to the desired penetration depth.

▲ Do not set the penetration depth before removing the sterility cap.

▲ To collect capillary blood from the heel of children under 1 year, **use only the low or medium penetration depth.** If the high penetration depth is used, the calcaneus (heel bone) is at risk of being injured.

- Obtaining blood**

- Hold the single-use lancing device between the index finger, middle finger and thumb.
- Press the single-use lancing device firmly against the chosen puncture site.

- Using your thumb, press the release button down completely.

- Gently massage the finger towards the fingertip or exert slight pressure on the heel to encourage a drop of blood to form.

▲ For some diagnostic tests the fingertip/heel must not be vigorously massaged or compressed, as doing so may produce incorrect results.

- Always dispose of used single-use lancing devices in accordance with your facility's disposal regulations.

**Last update**  
2009-03

**Customer Support and Service Centre**  
Roche Diagnostics Limited  
Charles Avenue, Burgess Hill  
West Sussex, RH15 9RY, **United Kingdom**  
+44 (0) 1444 256000  
Accu-Chek Customer Careline\*  
Freephone: 0800 701 000 (UK)  
Freephone: 1 800 709 600 (Ireland)  
\*calls may be recorded for training purposes

www.accu-check.co.uk

**Patents**  
US 5,755,733 licensed by Asahi, US 7,238,192

For an explanation of symbols used, please refer to the end of these instructions for use.

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© Roche Diagnostics 2010
ACCU-CHEK and SAFE-T-PRO are trademarks of Roche.