
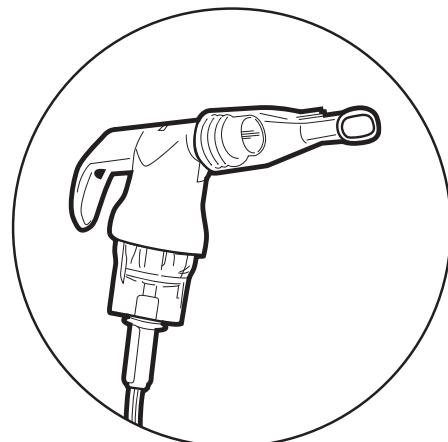
PHILIPS


RESPIRONICS

InnoSpire Deluxe

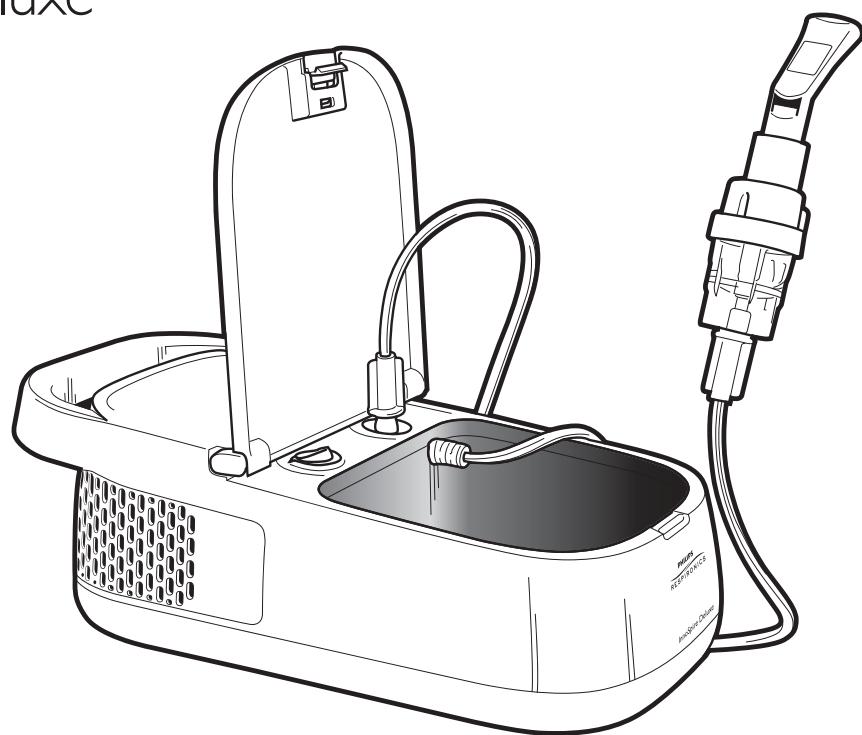InnoSpire Deluxe

Index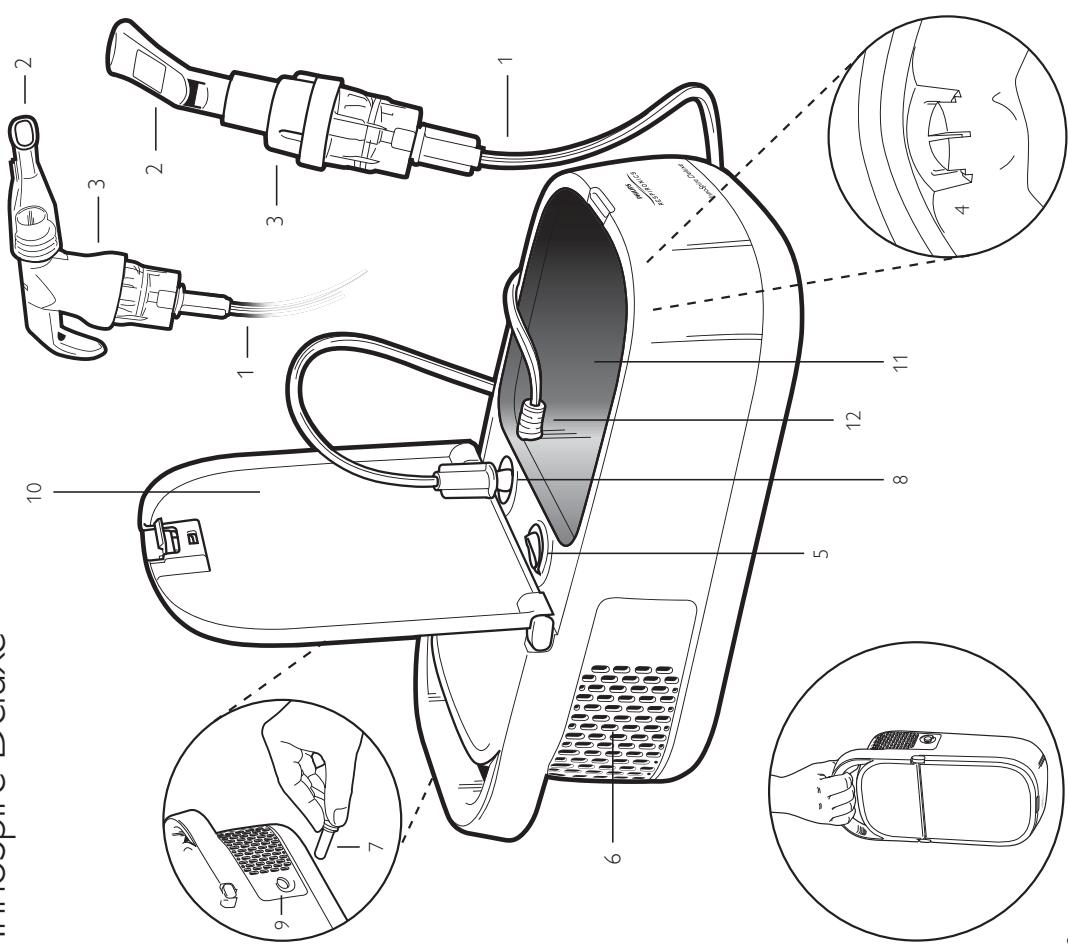

English	EN	4	10
Български	BG		16
Česky	CZ		22
Hrvatski	HR		28
Dansk	DA		34
Deutsch	DE		40
Español	ES		46
Eesti	EE		52
Français	FR		58
Ελληνικά	EL		64
Indonesia	ID		70
Italiano	IT		76
Latviski	LV		82
Lietuviškai	LT		88
Magyar	HU		94
Nederlands	NL		100
Norsk	NO		106
Polski	PL		112
Português	PT		118
Română	RO		124
Srpski	SR		130
Slovenčina	SK		136
Slovenščina	SL		142
Suomi	FI		148
Svenska	SV		154
ภาษาไทย	TH		160
Tiếng Việt	VI		166
Türkçe	TR		172
العَرَبِيَّةُ	AR		178
한국어	KO		184
עברית	IL		190
EMC data sheets (EN)			

1. Nebulizer tubing
2. Nebulizer mouthpiece
3. Nebulizer handset
4. Nebulizer holder (inside view)
5. On/Off power switch
6. Compressor air openings
7. Air filter (side view)
8. Compressor air outlet
9. Air inlet for filter (side view)
10. Lid
11. Storage compartment
12. Power cord location

Instructions for use

InnoSpire Deluxe

Important: Read these instructions carefully before using this device. If you have any questions or problems with the product, please contact your healthcare provider or Philips Respironics Customer Service at +1 724 387 4000.

General information

Intended use: This product is intended to provide a source of compressed air for medical purposes. It is to be used with a pneumatic (jet) nebulizer to produce aerosol particles of medication for respiratory therapy for both children and adults.

WARNINGS

- 1** Use the device for its intended purpose as described in this instruction manual as a compressor nebulizer system for aerosol therapy. Follow your doctor's instructions. Any other use constitutes improper use and may be dangerous. The manufacturer cannot be held liable for any damage caused by improper, incorrect or unreasonable use or if the unit is connected to electrical installations not complying with safety regulations.
- It is recommended to have a backup device (e.g. MDI or battery operated compressor) for respiratory delivery in case a situation arises when your nebulizer cannot be used (e.g. during a power outage or if your compressor becomes inoperable for any reason).
- If the device has been outside of the operating conditions, allow to come back to normal operating specifications before use.
- Any liquid spilled on or inside the unit should be allowed to dry before operating.
- 2** Keep this manual for future reference.
- 3** Use the product in a clean environment; one that is free from dirt, dust, pet hair, etc.
- 4** Device should not be used around flammable substances, or in the presence of a flammable anesthetic mixture with air, oxygen or nitrous oxide.
- 5** No modification of this equipment is allowed.
- 6** It is normal for the device to be warm to the touch when in use.
- 7** Do not stack the compressor with other equipment.
- 8** **Electromagnetic information:** Portable and RF communication devices such as cellular phones, pagers, etc. can interrupt operation of electrical medical equipment. For this reason, your compressor must be placed far enough away from these devices to prevent interference. This device meets IEC60601-1-2 standard for electromagnetic compatibility (EMC). EMC data sheets are available at the back of this document. EMC data sheets in other languages are available from Philips Respironics customer service at +1 724 387 4000. The CE mark on the product denotes compliance with all applicable EU Directives. Note that the Notified Body number does not apply to the RoHS (restriction of the use of certain hazardous substances in electrical and electronic equipment) Directive.
- 9** In case of failure and/or malfunction, read the "Troubleshooting" section. Do not open the compressor housing as this is not a serviceable product.
- 10** The proper function of the device can be compromised if original replacement parts (e.g., filters) are not used.
- 11** To reduce risk of electric shock, DO NOT DISASSEMBLE.
- 12** Indoor use only.
- 13** Do not use the device without the filter.
- 14** When using any electrical appliance, certain important safety measures must always be followed, including the following:
 - Use only manufactured-supplied original accessories and components;
 - Never submerge the unit in water as the device is not protected against water penetration;
 - Never touch the unit when your hands are wet or moist;
 - Do not leave the unit outdoors;
 - Place on a stable and horizontal surface when operating;
 - Make sure the air vent openings are not obstructed;
 - Do not allow unsupervised children or infirm individuals to use the unit;
 - Do not turn off the unit by simply pulling out the plug from the electrical outlet;
 - Do not use in a ventilator circuit;
 - Do not use if the device has cracks or is damaged.
- 15** Make sure that the electrical rating shown on the rating label on the bottom of the unit corresponds to your main voltage and frequency before plugging in the device.
- 16** Do not use any adapter, single or multiple, and/or extension cord.
- 17** Do not leave the unit plugged in when not in use. Unplug the device from the electrical outlet when not in operation.
- 18** Follow the manufacturer's instructions for installing the device. The manufacturer cannot be held responsible for any damage caused by incorrect installation.

19 The power supply cord cannot be replaced by the user: In case the power supply cord becomes damaged, contact Philips Respironics customer service for replacement (there is no service option).

20 Do not over fill the storage compartment with accessories.

21 Before performing any cleaning operation or filter replacement, disconnect the device from the main supply by unplugging it.

22 Small parts can be inhaled, swallowed or dangerous. Cable, due to its length, may result in provoking strangulation or asphyxiation. Do not leave the device alone with a small child or physically challenged individual or an individual with learning difficulties.

23 If you no longer use the device, dispose of according to the local regulations in force.

24 Remember to:

- Use this device only with medications prescribed by your doctor;

- Use the accessories recommended by your doctor.

Instructions for use

⚠ Remove the compressor from carrying case if supplied before use.

1 After unpacking the device, check the compressor and accessories for visible damage or defects such as cracks in the plastic housing which could expose electrical components. Contact Philips Respironics customer service or your product distributor to report any damaged product.

2 When taking a treatment, sit upright and relaxed.

3 Plug the compressor into the electrical outlet.

4 Refer to the nebulizer and accessory instructions for use before treatment.

5 To start the treatment, switch the device ON by pressing the ON / OFF switch onto the "I" position.

6 When the treatment has been completed, switch the unit OFF by setting the switch position to "O" and disconnect the plug from the electrical outlet.

7 Do not close the lid when the nebulizer tubing is attached.

Cleaning and maintenance

⚠ During cleaning, make sure the power cord is not plugged in the electrical outlet.

Cleaning of the compressor, nebulizer and accessories

Use a damp cloth to wipe down the outside of the compressor at least once each month.

Refer to nebulizer and accessory instructions for use for cleaning of the accessories and nebulizer.

Replacement of the nebulizer

The nebulizer and other accessories should be replaced according to its instructions.

Replacement of the filter

The filter is to be replaced when it appears discolored or wet.

Troubleshooting

The device does not turn on

- Make sure the plug is firmly fitted to the electrical outlet.

- The thermal protector may have been activated by the following reasons:

- a) the device was used near a heater or in a hot environment

- b) the air vent openings (6) are obstructed and do not allow for the motor ventilation

If this is the case set the ON/OFF switch to 'O' and let it sit for at least 60 minutes. In the event of malfunction (insufficient pressure and flow) turn the unit off for 10 seconds and turn back on.

After each cleaning, check that the unit is working properly by verifying that air still comes out of the nozzle when the unit is turned to the "ON" position.
Unauthorized repair voids warranty.

If the device does not work properly after attempting these troubleshooting steps, contact your authorized product distributor or Philips Respironics Customer Service at +1 724 387 4000.
Maintenance and repairs

Never open the device. There are no serviceable parts within the unit. The compressor requires no lubrication or maintenance.

Technical specifications

Electricity supply	230V/50Hz, 1 Amp
Thermal overload protector	Resettable thermal fuse; functioning temperature 150 °C
Compressor max. pressure	360 kPa (3.6 bar)
Average flow rate	7 LPM 70 kPa (0.7 bar)
Max flow rate	9.8 LPM
Weight	2.2kg
Size	380mm x 170mm 110mm
Noise level	58±3dB(A)

Technical specifications

Electricity supply	220V/60Hz, 1.2 Amp
Thermal overload protector	Resettable thermal fuse; functioning temperature 130 °C
Compressor max. pressure	268 kPa (2.68 bar)
Average flow rate	6 LPM 70 kPa (0.7 bar)
Max flow rate	10.7 LPM
Weight	20kg
Size	380mm x 170mm x 110mm
Noise level	58±3dBA

Class II device, Type BF device (device with specific protection against electrical hazards).

DOUBLE INSULATED**Reference to standards**






Electric safety standards IEC 60601-1

Electromagnetic compatibility according to IEC 60601-1-2

Ambient conditions

Storage conditions	MIN -25 °C – MAX 70 °C (-13 °F – 158 °F)
Temperature	MIN 10% RH – MAX 95% RH
Humidity	MIN 10% RH – MAX 95% RH
Operating conditions	MIN 5 °C – MAX 40 °C (41 °F – 104 °F)
Temperature	MIN 10% RH – MAX 95% RH
Humidity	Performance may vary depending on altitude above sea level, barometric pressure and temperature

Symbol glossary

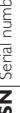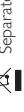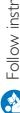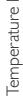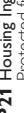-  ON (power)  ATTENTION
-  OFF (power)  Alternating current
-  Serial number  Follow instructions for use
-  Atmospheric pressure  Manufacturer
-  Humidity  Indoor use only
-  Temperature limitation  Separate collection
-  Type BF applied parts  Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the good repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.
-  Statement is provided to a Buyer who is a 'consumer' under the Consumer Guarantees Act 1993, New Zealand: Our goods come with guarantees that cannot be excluded under the Consumer Guarantees Act 1993. This guarantee applies in addition to the conditions and guarantees implied by that legislation.

IP21 Housing Ingress Protection rate: IP21 (protected from the penetration of solid bodies with dimensions greater than 12 mm, Protected from the penetration of vertically falling water drops).

Warranty

Respironics, Inc warrants the compressor to be free from defects in materials and workmanship under normal use and operation for a period of 3 years from the date of purchase from Respironics, Inc. The warranty is limited to repair or replacement, at Respironics, Inc's sole option, of any such component or equipment claimed to be defective when claim is shown to be bona fide by evaluation by Respironics, Inc. This warranty does not extend to any components or equipment subjected to misuse, improper operation, accidental damage, or unauthorized repairs, and is not extended to changes of, or for, labor repairs. All items returned must be properly packaged and shipped, prepaid by the product distributor servicing the unit. Respironics, Inc shall not be liable to purchaser or others for loss of use of equipment or for indirect, or incidental or consequential damages that might arise.

Limited Warranty for Australia and New Zealand only

Note: For Australian and New Zealand customers this warranty replaces the warranty contained above.
 Respironics, Inc., a Philips Healthcare company warrants that the Products shall be free from defects of workmanship and materials and will perform in accordance with the Product Specifications. 2. This warranty is valid for a period of 3 years from the date of purchase from an authorised Respironics dealer. 3. If the Product is found to contain a defect of workmanship or materials or fails to perform in accordance with the Product specifications during the applicable warranty period, Respironics will repair or replace, at its option, the defective material or part. 4. The customer is responsible for returning the product to an authorised Philips Respironics dealer, and collecting the product from the authorised Philips Respironics dealer after repair or replacement, at its own cost. Philips Respironics is responsible only for the freight cost of transporting the product between the authorised Philips Respironics dealer and Respironics. Respironics reserves the right to charge an evaluation and postage fee for any returned Product as to which no problem is found following investigation. 5. This warranty does not cover damage caused by accident, misuse, abuse, alteration, and other defects not related to materials or workmanship. 6. The warranty provided by Respironics herein is not transferrable by the Buyer in the event of any sale or transfer of Products purchased by the Buyer from an authorised Respironics dealer. 7. To the extent permitted by law, where the Buyer has the benefit of an implied guarantee under the Australian Consumer Law, but the Product is not of a kind ordinarily acquired for personal, domestic or household use or consumption Respironics' liability shall be limited, at the option of Respironics, to the replacement or repair of the Product, or the supply of an equivalent Product. 8. To exercise your rights under this warranty, contact your local authorised Philips Respironics dealer. A list of all authorised dealers is available at the following link: http://www.healthcare.philips.com/au_en/homehealth/distributors_index.wpd. Alternatively, you can make a claim under this warranty by contacting Respironics directly at: Philips Electronics Australia Limited, 65 Epping Road, North Ryde NSW 2113, Australia. Tel: 1300 766 488, Email: prcontact@philips.com. 9. The following statement is provided to a Buyer who is a "consumer" under the Australian Consumer Law: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the good repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. 10. The following statement is provided to a Buyer who is a 'consumer' under the Consumer Guarantees Act 1993, New Zealand: Our goods come with guarantees that cannot be excluded under the Consumer Guarantees Act 1993. This guarantee applies in addition to the conditions and guarantees implied by that legislation.

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- A. Instructions for use**
1. Innospire Deluxe needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in this document.
 2. Portable and mobile RF communications equipment can affect Innospire Deluxe.

4.

Guidance and manufacturer's declaration – electromagnetic immunity			
The Innospire Deluxe is intended for use in the electromagnetic environment specified below. The customer or the user of the Innospire Deluxe should assure that it is used in such an environment.			
Electromagnetic environment – guidance			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines and dc lines	±2 kV for power supply lines and dc lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) and neutral ±2 kV line(s) to earth	±1 kV line(s) and neutral ±2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	<5 % U_T (>95 % dip in U_T) for 0.5 cycle	<5 % U_T (>95 % dip in U_T) for 0.5 cycle	Mains power quality should be that of a typical commercial or hospital environment. If a dip or an interruption of mains power occurs, the current of the Innospire Deluxe may be dropped off from normal level, it may be necessary to use uninterruptible power supply or a battery.
RF emissions CISPR 11	Group 1	The Innospire Deluxe uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30% dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5s
Harmonic emissions IEC 61000-3-2	Class B	The Innospire Deluxe is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.	70 % U_T (30% dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5s
Voltage fluctuations/flicker emissions IEC61000-3-3	Complies (applicable to 50Hz rated devices only)	Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3 A/m
NOTE U_T is the ac. mains voltage prior to application of the test level.			Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and manufacturer's declaration – electromagnetic immunity			
The Innospire Deluxe is intended for use in the electromagnetic environment specified below. The customer or the user of the Innospire Deluxe should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3Vrms 150kHz to 80MHz	3Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the Innospire Deluxe, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter: $d = 12\sqrt{P}$
Radiated RF IEC 61000-4-3	3V/m 80MHz to 25GHz	3V/m	$d = 1.2\sqrt{P}$ 80MHz to 800MHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).

Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b. Interference may occur in the vicinity of equipment marked with the following symbol:

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Innospire Deluxe is used exceeds the applicable RF compliance level above, the Innospire Deluxe should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Innospire Deluxe.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the Innospire Deluxe			
The Innospire Deluxe is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Innospire Deluxe can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Innospire Deluxe as recommended below, according to the maximum output power of the communications equipment.			
Separation distance according to frequency of transmitter			m
Rated maximum output power of transmitter W	150 kHz to 80 MHz $d = 12\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

InnoSpire Deluxe is a trademark of Koninklijke Philips N.V.
and its affiliates, all rights reserved.

© 2015 Koninklijke Philips N.V. All rights are reserved.

CE 0123

Manufactured for
Respironics Respiratory Drug Delivery (UK) Ltd
Chichester Business Park, City Fields Way
Tangmere, Chichester, PO20 2FT UK

Made in China

www.philips.com/respironics

PN 1113054 R02