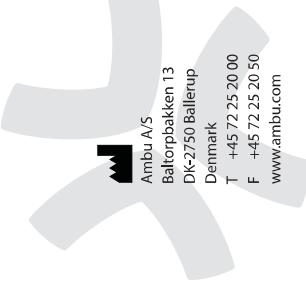
Ambu® Snap Clip
Clip Connector for ECG tab electrodes
Instructions for Use

Инструкции за ползване	Lietošanas pamācība
Návod k použití	Gebruiksaanwijzing
Brugsanvisning	Bruksanvisning
Bedienungsanleitung	Wskazówki do stosowania
Oδηγίες χρήσης	Instruções de utilização
Instrucciones de uso	Instrucțiuni de utilizare
Kasutamisjuhend	Инструкция по применению
Käyttöohjeet	Návod na použitie
Instructions d'utilisation	Navodila za uporabu
Upute za upotrebu	Bruksanvisning
Hásználati útmutató	Kullanım kılavuzu
Istruzioni d'uso	Instructions for use
使用说明书	使用说明
Naudojimosi taisyklės	使用说明

8690404 • 2013/03 • V02

CE

USA Ambu Inc.
6740 Baymeadow Drive
Glen Burnie, MD 21060
Tel: +1 410 768 6464
+1 800 262 8462
Fax: +1 410 760 4907
www.ambusa.com

Spain Firma Ambu S.L.
Alcalá, 261-265, Edif. 2-3º dcha.
28027 Madrid
Tel: +34 91 411 68 30
Fax: +34 91 564 50 82
www.ambus.es

Netherlands Ambu B.V.
Schiphol Boulevard 127
1118 BG Schiphol Airport
Tel: +31 0182 52 60 60
Fax: +31 0182 52 70 73
www.ambu.nl

France Ambu S.A.R.L.
Bureaux du Parc
Avenue Jean Gabriel Domergue
33000 Bordeaux
France
Tél: +33 (0)5 57 92 31 50
Fax: +33 (0)5 57 92 31 59
www.ambu.fr

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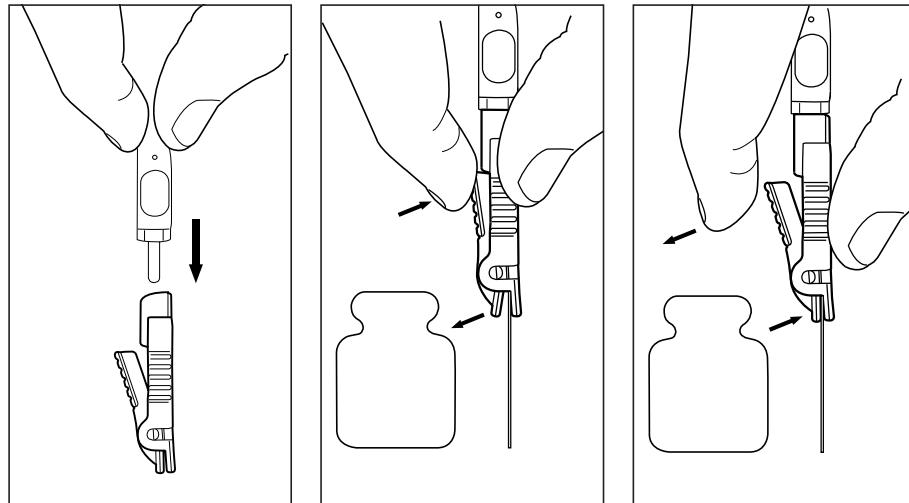
Ambu® is a registered trademark of Ambu A/S, Denmark
Ambu A/S is certified according to ISO 13485

Instructions for use

EN

The Ambu Snap Clip is designed to provide a secure connection from the ECG electrode to the cable. The Ambu Snap Clip can be connected to a cable with a 3 mm fitting.

Instructions for use

1. Before each use carefully inspect the Ambu Snap Clip. Do not use broken or damaged clips.
2. Ensure good contact of snap connection prior to use.
3. The Ambu Snap Clip must only be used with devices in compliance with IEC 60601-1.

Cleaning instructions

- To clean the Ambu Snap Clip use warm, soapy water and a soft bristle brush.
- Rinse with demineralised water.
- Ensure that the Ambu Snap Clip is dried thoroughly before using.
- Do not use ultrasound cleaning, as this will damage the connections.
- Do not autoclave the Ambu Snap Clip as this will damage the device.
- Do not clean the Ambu Snap Clip with solvents

Standards applied

- The Ambu Snap Clip for ECG tab electrodes function conforms with:
- IEC 60601-1 Medical electrical equipment – Part 1: General requirements for basic safety and essential performance.

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