SE

BESKRIVNING AV PRODUKTEN

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EN

DESCRIPTION OF THE PRODUCT

Single use hydrophilic urinary catheter, designed for female users.

MATERIAL

Polyolefin-based elastomer (POBE) with PVP coating. Not manufactured with PVC, Phthalates and latex.

INDICATIONS FOR USE

For short and long term bladder management with intermittent urinary catheterization.

WARNINGS

This is a single-use product. Once used the surface coating will deteriorate and the catheter will no longer be sterile. Intermittent urinary catheterization therapy is associated with an increased risk of urethral bleeding/trauma and/or urinary tract infection. User instructions and recommendations from healthcare professionals should be followed.

PRECAUTIONS

LoFric catheters must be prescribed by a healthcare professional, selecting the most suitable catheter type and size based on the patient individual needs. Please consult your healthcare professional if you experience any difficulties with the catheterization procedure or the use of LoFric Elle.

ADVERSE REACTIONS

There are no adverse reactions known specifically for LoFric catheters. Common adverse reactions (>1/100) related to the intermittent urinary catheterization therapy includes urethral trauma and urinary tract infection. If any discomfort or sign of trauma or infection occurs, discontinue use and consult your healthcare professional. Any serious incident occurring in relation to use of these catheters should be reported to the Manufacturer and your local health authority.

PACKAGE AND STORAGE

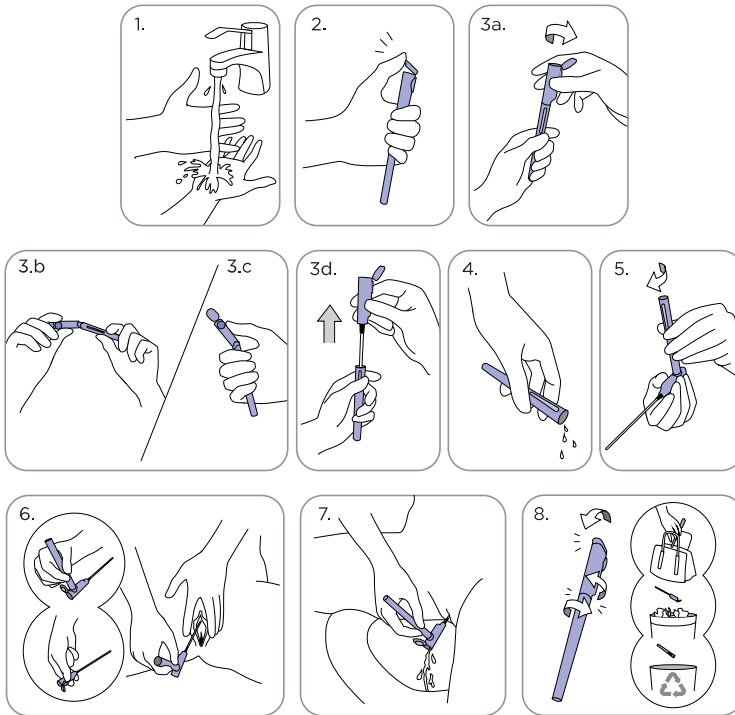
The package consists of 30 single use products, sterilized by irradiation.

The components must not be used beyond the expiry date. The components must be stored, in their sealed package, in a dry place, at room temperature.

Do not use the product if the sterile package is broken or damaged.

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EN

INSTRUCTIONS FOR USE:

1. Wash your hands thoroughly with soap and water.
2. Break the upper sterile barrier lid.
3. Break the lower sterile barrier by holding the container, and
 - a) twist or gently bend the upper part (image b or c).
 - b) Hold the product vertically and pull the catheters straight up out of the container.
4. Empty the remaining fluid from the container (in the toilet or sink).
5. OPTION: By attaching the container to the upper part, it can be used as a handle.
6. Tilt your pelvis forward, spread the labia, pull gently upwards, the urethra is located just above the vaginal opening. With the other hand, insert the catheter slowly into your urethral opening, until urine starts to flow, insert slightly more to ensure both eyelets are inside the bladder.
7. When the urine flow slows to a drip, withdraw the catheter slowly. If urine starts to flow again, stop the withdrawal process and wait until the urine flow stops, to ensure complete bladder emptying.
8. Put the catheter back inside the container. Dispose, or carry it in your bag until disposal. Please note! The container is recyclable.

ADDITIONAL INFO FOR CUSTOMERS IN THE UNITED KINGDOM:
If you are looking for any advice or information about our products or services, please contact our friendly support team on:
Tel: +44 (0)1453 791763

We can offer:

- FREE magazine, full of hints, advice and user stories.
- Information about current and new products available
- Experienced Nurse advisors to offer advice and support.
- FREE lifestyle accessories, such as discreet pouches/purses

Please contact us for more details by either visiting our website www.wellspect.co.uk or calling the number above.

PD-2981E-202104-006

UK:

For more information please visit wellspect.co.uk or call (0)1453 791763

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