
PLEASE KEEP THIS INSTRUCTION FOR FURTHER REFERENCE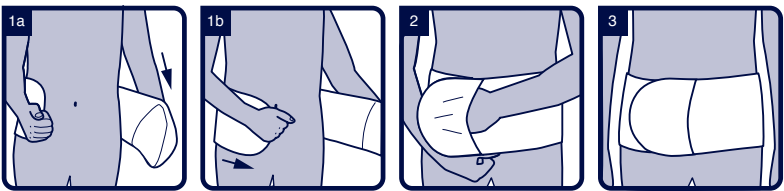

EN:

Indication:

Corsinel support belt is specially designed for parastomal hernia, umbilical hernia, epigastric hernia or abdominal scar hernia. The support from the belt offers relief, alleviation, assists mobility and counteracts sense of gravity, thereby providing pain relief. The Corsinel belt helps fixate the ostomy pouch, provides discretion and a smooth appearance. Furthermore, the belt may help reduce unpleasant smells and noises from a stoma as well as it may reduce the risk of hernia development. The Corsinel belt is available in three heights: 15, 20 and 26 cm. For recommendation on height and use, please contact your doctor or nurse.

Features:

The Corsinel belt supports a hernia, and the wide hook and loop offers adjustment of support level. The silicone anti-roll pattern on the elastic bands prevents the Corsinel belt from rolling. The 20 and 26 cm belts have a lower back, which improves the anatomic fit. The belt has an easy-to-apply pocket to facilitate application. The belt is made of air- and moisture permeable material. There is no difference between up and down.

How to measure yourself:

With a hernia: Measure yourself when lying down and when the hernia has retracted, if possible. Measure yourself at the widest abdominal circumference covered by the belt.

Without a hernia:

Measure yourself in standing position and at the widest abdominal circumference covered by the belt.

NOTE: Sizes overlap; if abdominal measurement lies between two sizes, choose the smaller size if you prefer an extra tight-fitting belt, and the larger size if you prefer the belt less tight.

Ensure that the belt feels comfortable to wear, when sitting and standing.

Instructions for use:

- For optimal support and if possible, apply the belt when laying down. Place the belt under your back (1a). Wrap and tighten it around the abdomen by using the easy-to-apply pocket (1b). If applied when standing, put one hand in the easy-to-apply pocket and wrap the belt around your back with the other hand (1a) so that the pocket-less end is located on your stomach (1b)
- Use the strength of your fingers, hand and arm to fixate the easy-to-apply pocket on the hook and loop. The wide hook and loop offers adjustment of support level. Ensure that the support level feels comfortable. Close the belt outside the stoma area to prevent uncomfortable pressure on the stoma. If you have a 20 or 26 cm belt, make sure that the low-cut part is centered at the back.
- Make sure the belt is not too loose, tight or has folds. If so, open and apply again.

Precautions:

Make sure the product is not compromising stoma and/ or lung functions. We advise against using the belt during pregnancy, by acute abdomen, abdominal compartment syndrome, presence of ascites fluid or general discomfort/pain in the abdominal region. Please consult your doctor if you suffer from Chronic Obstructive Pulmonary Disease, if you experience unexpected discomfort, stoma or skin irritation, or if you have any questions regarding use. By urgent need for removal of product, open with scissors.

Care handling:

Close hook and loop before washing. Wash with similar colours in a washing bag. Use of chlorine-based bleach or chlorine-based disinfectant will decrease durability and function of the product.

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