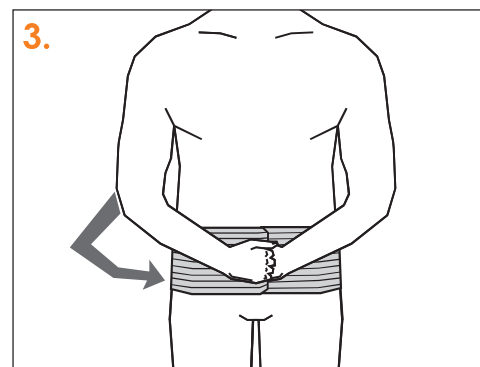
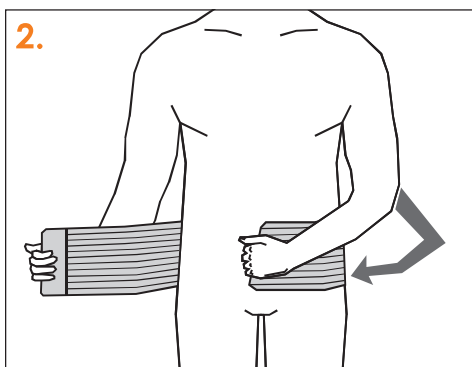
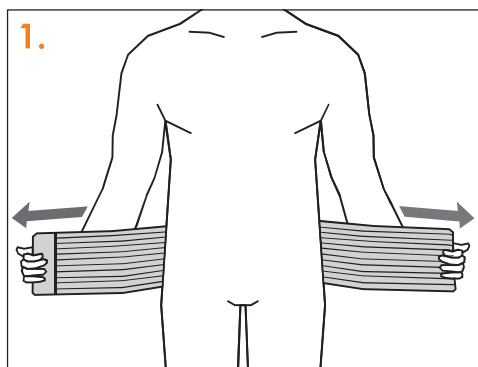
INSTRUCTIONS FOR USE

Mika – FIXATION BAND

art no 50224

Application

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EN To be used as a wide fixation band after different types of surgery and for compression. For example, for fixation of dressings, ostomy pouches or as a wide breast band after various types of breast surgery for compression over the chest, back and armpit.

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EN Machine wash at 60°C. Secure the hook closures before washing. Use a washing bag. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

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Material

Polyamide 80%, Polyester 10%, Elastane 10%. Latex free.

Please consult our product catalogue for more information.

For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.

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EN Soft fixation band for fixation of dressings, ostomy pouches or to be used as a breast band. The fixation band is discreet and invisible under clothing. Mika comes in two flexible sizes, easily to be adjusted as the Velcro attaches to the surface of the entire band. Mika is available in white. Height 15 cm.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Check that the Velcro is are clean and attaches properly. Consult medically trained personnel if using this product during medication and/or in combination with aids of assistance.

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