
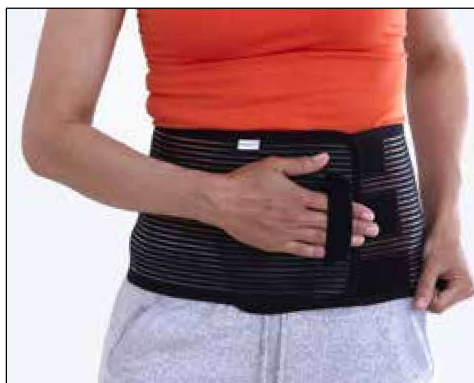
# INSTRUCTIONS FOR

# Smidig - SUPPORT BELT

art no 50247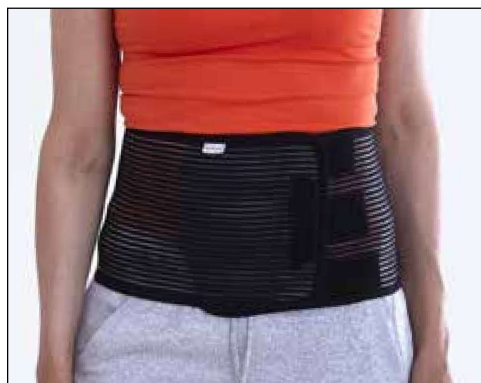

## Application

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**EN** 1. Choose the correct size and put on the binder around desired placement. Close the Velcro at centre front.  
2. Adjust until comfortable. Make sure compression is not too light.

## Material Composition

37% Spandex, 54% Polyester, 5% Polyamide, 4% Nylon. Latex free.

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**EN** Measure circumference around intended placement. Pick the correct size according to the measurement table.

Size	Circumference
X Small	60-75 cm
Small	70-85 cm
Medium	80-100 cm
Large	95-115 cm
X-Large	110-125 cm
2X-Large	120-135 cm

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**EN** When compression and light support are desired around the abdomen/torso. For both post-operative care and permanent use. Suitable as a wide breast band after surgery on the chest, e.g. gynecomastia or thorax operation. Can be used after abdominal surgery, to support hernia or ostomy. For fixation of ostomy pouch. The binder should not be cut. Can be used as a comfortable support after pregnancy / delivery or as light back support.

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**EN** If skin irritation, chafing or changes in skin color occur, discontinue use and contact your doctor or medical practitioner. Prescription and recommended use for the product should be made by a medical practitioner. el evaluador.

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**EN** Periodically check the condition of the Velcro to ensure they are free from grease and damage. A qualified medical practitioner should supervise use in combination with other medical products.

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**EN** Machine wash at 40°C. Can be washed at a higher temperature if necessary, max. 60°C. The product will last longer if washed at a lower temperature. Use a laundry bag. Dry flat. Do not tumble dry. Seal the Velcros before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. del material.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se).  
For further questions, contact your doctor, health practitioner, medical equivalent or  
Nordicare: +46 42-35 22 20/ DK 7070-1907, [info@nordicare.se](mailto:info@nordicare.se).