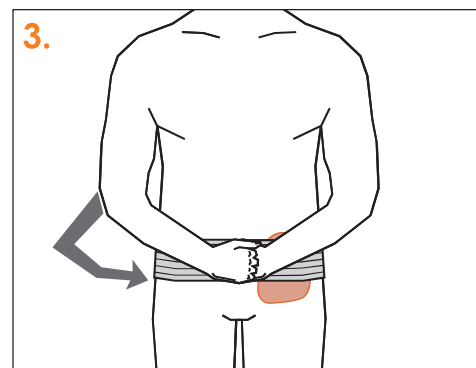
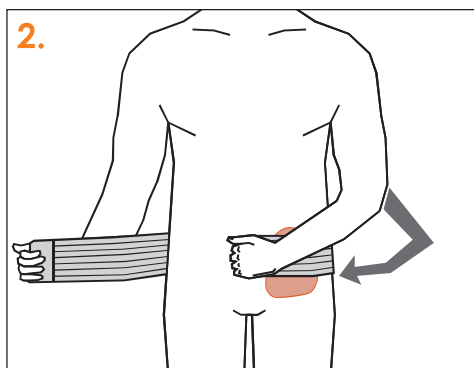
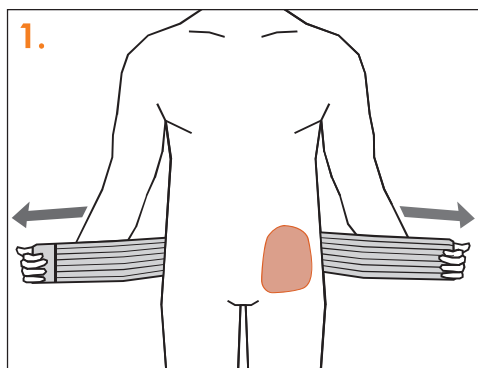
INSTRUCTIONS FOR USE

StomiMini – STOMA BINDER

art no 50213

Application

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EN 1. Apply the fixation band around the intended placement.

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EN Circumference of intended placement. Universal size, fits circumference measuring up to 120 cm.

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EN To be used as a stoma band for fixation of ostomy pouches. Suitable to use for ileo-, colo- or urostomy. Not suitable as support for parastomal hernias.

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Material

Polyamide 55%, Polyester 35%, Elastane 10%. Latex free.

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EN Machine wash at 60°C. Secure the hook closures before washing. Use a washing bag. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

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Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.

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EN Soft fixation band for fixation of ostomy pouch. Discreet and invisible under clothing. One universal size fits circumference up to 120 cm, easily to adjusted as the Velcro attaches to the surface of the entire band. Suitable for ostomy on either right or left side. StomiMini is available in black and white. Height 7,5 cm.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Check that the Velcro is are clean and attaches properly. Consult medically trained personnel if using this product during medication and/or in combination with aids of assistance.

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