INSTRUCTIONS FOR USE

StomiTube – STOMA BINDER

art no 50214

Application

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EN 1. Put on the tube shaped binder like regular underwear. Make sure the narrower shaped part of the binder is placed in the back and that the pouch is fixated in the pockets in the lower rib.

2. Secure the pouch in the folded edge at the bottom of the binder. Wear your regular underwear underneath or on top of the StomiTube.

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| Size | Hip Circumference | |
|-----------------|-------------------|--|
| X Small - Small | 65 - 90 cm | |
| Small - Medium | 85 - 110 cm | |
| Medium - Large | 105 - 125 cm | |
| Large - X Large | 120 - 140 cm | |

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EN In order to determine the correct size, measure the hip circumference (at its widest point) in centimeters.

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EN For safe ostomy pouch fixation. Suitable to use for ileo, colon or uro stoma.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of stoma binders. For best results, follow instructions for use.

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EN Machine wash in 60°C (140°F). The product lasts longer if washed in 40°C (104°F). Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent -or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.

NordiCare Ortopedi & Rehab
Solrosvägen 1, SE-263 62 Viken, Sweden.
Tel +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se

2020-02-17

Available colors:

MD Class 1 