
# INSTRUCTIONS FOR

# StomiSoft - STOMA BINDER


art no 50218

## 3 possible applications



1. With cut hole for stoma bag for strong compression around stoma but no compression on stoma bag.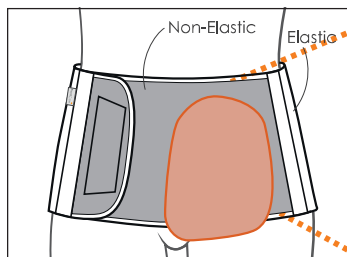

2. Stoma bag under non-elastic front for stronger compression on stoma bag.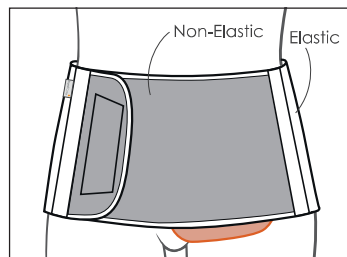

3. Stoma bag under elastic material for less compression on stoma bag.

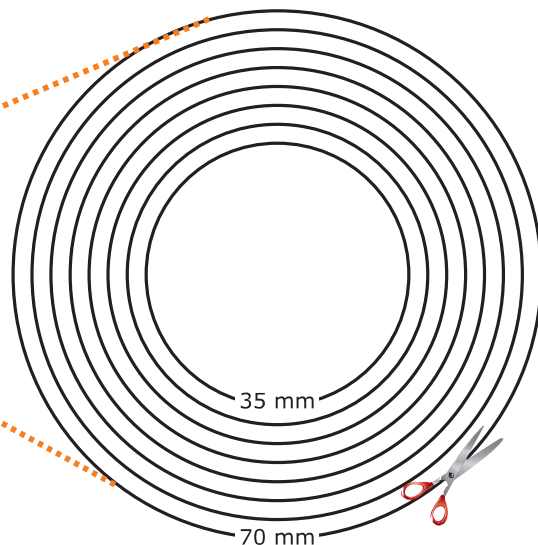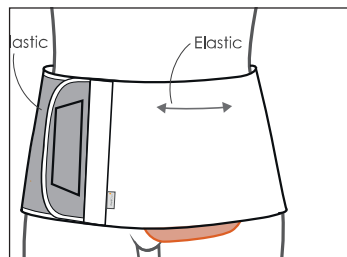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**EN** Velcros fastens on the entire surface of the non-elastic front material and makes the binder flexible in size and circumference, the binder can be adjusted to attain desired pressure. Reversible to suit stoma on both right and left side. Suitable to use with or without cut hole.

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**EN** Measure circumference around intended placement. Pick the correct size according to the measurement table.

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**EN** For stoma and non-complicated parastomal hernia. Suitable as permanent binder.

## Material Composition

Polyester 49%, Polyamide 29%, Elastane 11%, Polyurethane 11%. Latex free.

NordiCare®

2019-10-30

NordiCare Ortopedi & Rehab AB  
Solrosvägen 1, SE-263 62 Viken, Sweden  
Phone +46 42 35 22 20, Fax +46 42 35 22 21  
info@nordicare.se, www.nordicare.se

This product meets the requirements of Medical Devices class 1, non sterile.

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use.

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**EN** Periodically check the condition of the Velcro to ensure they are free from grease and damage. A qualified medical practitioner should supervise use in combination with other medical products.

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**EN** Machine wash at 40°C. Can be washed at a higher temperature if necessary, max. 60°C. The product will last longer if washed at a lower temperature. Use a laundry bag. Dry flat. Do not tumble dry. Seal the Velcros before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se).  
For further questions, contact your doctor, health practitioner, medical equivalent or  
Nordicare: +46 42-35 22 20/ DK 7070-1907, [info@nordicare.se](mailto:info@nordicare.se).