# INSTRUCTIONS FOR

## Vira x2 - BINDER

art no 50228

### Application

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**EN** Choose size and place the binder over the surged area. Begin by fastening the lower band **1** at the bottom of the hip. Then fasten the top band **2**. Adjust until it feels comfortable. The girdle should sit tight but not uncomfortably tight.

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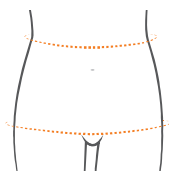
**EN** Choose the size by the hip circumference, if the hip size is within an overlapping range, check the circumference of your waist:

- Narrow waistline: choose the smaller size.
- Straight body shape: choose the larger size.

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**EN** The initial fitting, prescribing and recommendations for use should be done by/under the direction of qualified medical practitioner or other professionals. Follow the instructions for best result. If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person.

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**EN** For postoperative care after abdominal surgery, abdominoplasty and/or liposuction. Can also be used for various types of hernia problems. Relieves wounds, provides pain relief, and enables early mobilization of the patient.

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**EN** Periodically check the condition of the velcro to ensure they are free from dirt and that their fastening capabilities have not deteriorated. A qualified medical practitioner should supervise use in combination with other medical products.

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**EN** Machine wash at 40°C. Can be washed at a higher temperature if necessary, max. 60°C. The product will last longer if washed at a lower temperature. Use a laundry bag. Dry flat. Do not tumble dry. Seal the velcros before washing. Do not use bleach, chlorine or fabric softener as this could impair the function of the material.

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