© 2005-19 eekin dot s.p.a. All rights reserved.
TG Eekin Limited,
15 Ballystock Road,
County Down, BT23 5QY, U.K.
T: +44 (0)28 9187 1000
F: +44 (0)28 9187 1111
W: www.eekin.eu

www.eekin.eu
TGEekin
@TGEekin
tgeekinhd

Product information in Italian, Spanish, French, German, Dutch, Polish, Czech, Slovak, Hungarian, Greek, Turkish, and Russian. Includes a list of countries: Italy, Spain, France, Germany, Netherlands, Poland, Czech Republic, Slovakia, Hungary, Greece, Turkey, and Russia.

Product information in Danish, Norwegian, Swedish, Finnish, and Estonian. Includes the eekin dot logo and contact information.

Product information in Slovenian, Croatian, Serbian, Macedonian, Bulgarian, and Romanian. Includes icons for medical device, first aid, and recycling.

Product information in Portuguese, Spanish, Catalan, and Basque. Includes icons for medical device, first aid, and recycling.

Product information in Galician, Catalan, and Basque. Includes icons for medical device, first aid, and recycling.

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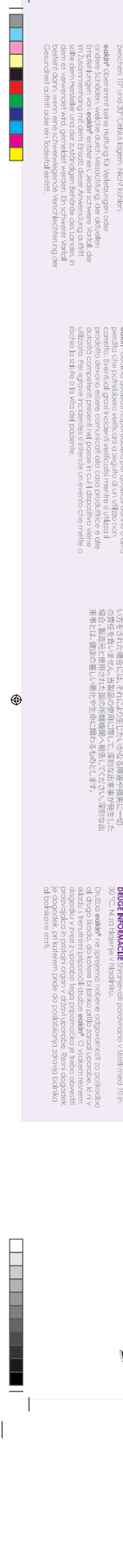
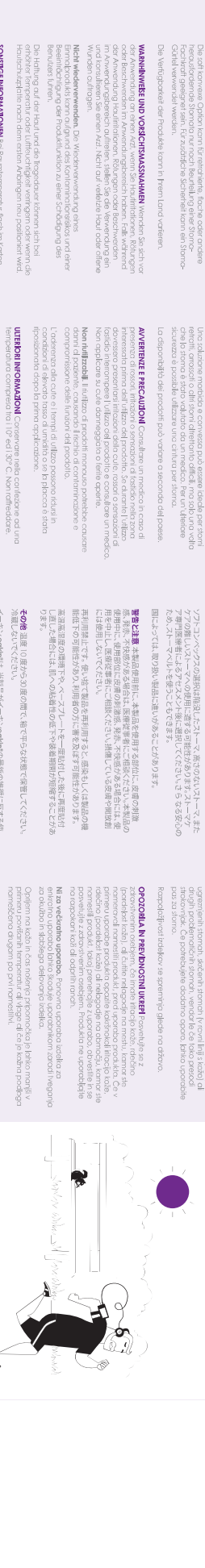
Instructions for use

1. Preparation: Ensure site is clean and dry before application. (1a) For use on the skin, gently wash and dry the skin. Use the provided measuring guide to measure the size and shape of the abrasion. Mark the location of the wound on the skin with a waterproof pen. (1b) If the wound extends to the face, use the forehead side of the dressing. (1c) If the wound extends to the arms or legs, use the other side. (1d) An accurate measurement of the skin defect is an aid.

2. Application: Peel the protective film off the backing (2a). Store the backing of the complete dressing gently against a clean surface. (2b) Cut the dressing to the correct size according to the measuring guide. (2c) Peel the protective film off the backing and place the adhesive side on the skin. (2d) The cover is protected with a thin white film. Do not touch the adhesive side. (2e) The dressing can be used when bonding or to prevent the dressing from peeling off. (2f) After application, the dressing can be used as a bandage. (2g) The dressing can be used to prevent the dressing from peeling off. (2h) The dressing can be used to prevent the dressing from peeling off. (2i) The dressing can be used to prevent the dressing from peeling off. (2j) The dressing can be used to prevent the dressing from peeling off.

3. Removal: Gently remove the dressing by peeling the adhesive side away from the skin. (3a) Peel the adhesive side away from the skin. (3b) Peel the adhesive side away from the skin. (3c) Peel the adhesive side away from the skin. (3d) Peel the adhesive side away from the skin. (3e) Peel the adhesive side away from the skin. (3f) Peel the adhesive side away from the skin. (3g) Peel the adhesive side away from the skin. (3h) Peel the adhesive side away from the skin. (3i) Peel the adhesive side away from the skin. (3j) Peel the adhesive side away from the skin.

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