

Instruction for Use

Rev. A/1 Effective: 2020-04-01

Description:

The anti-fog face shield is composed of a piece of PET film with a piece of protection film, a foam support and an elastic band.

Indication for Use:

The anti-fog face shield is used to help protect the whole face against potential contamination from blood pathogens, body fluids, or harmful chemical splash during inspection or treatment.

Contraindications:

None.

Product side effects:

This product has no side effects.

Warning:

1. Exposing in high humid environment may cause losing efficacy of anti-fog effect.
2. Suspension of use in case of damage or slackness during use.
3. During use, please try not to touch the center of the lens.

Operation:

Step 1: Remove the inner protective film.












Step 2: Pull the elastic band and put on the head.

Storage:

Store product in a dry place at 0– 30 °C.

Keep away from sunlight.

Explanations for symbols:

	USE BY		DO NOT REUSE
	BATCH CODE		DO NOT USE IF PACKAGE IS DAMAGED
	DATE OF MANUFACTURE		KEEP AWAY FROM SUNLIGHT
	CE MARK		KEEP DRY
	MANUFACTURER		AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY
	CATALOGUE NUMBER		

Manufactured by:

Name: Zhejiang Wellong Medical Technology Co., Ltd.

Address: Lvshan Industrial area, Changxing Country, Huzhou City, Zhejiang Province, P.R.China

ZIP Code: 313105

Under License by:

Name: s.a.r.l. Medinova International

Address: 3 cours Charlemagne, 69002 Lyon, France

Authorised Representative in the European Community:

Name: Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestrasse 80, 20537 Hamburg, Germany

Contact Person: Xufeng Chen

Telephone: +49-40-2513175