

# WOUND CARE DryMax<sup>®</sup>

## NON STERILE


DryMax Non Sterile is a conformable, superabsorbent dressing for use on exuding wounds. DryMax Non Sterile absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core forming a gel, thus minimizing the risk of maceration.

## DryMax Non Sterile


DryMax Non Sterile should be used as a secondary dressing. The design allows the dressing to be applied with either side facing the primary dressing. This feature also makes it possible to stack dressings on top of each other if extra absorption capacity is required.

The dressing remains dry and comfortable for the patient. Simultaneously it contributes to a moist wound environment. The hydrophilic layer transports fluid effectively away from the primary dressing on the wound and distributes it evenly into the ABSORBEST core.


The ultra-thin and highly conformable features allows the dressing to be used on all parts of the body. DryMax Non Sterile is designed to work well together with compression therapy.

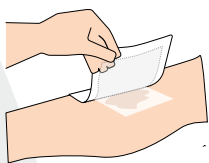

The dressing must be placed in close combination with a suitable primary dressing.

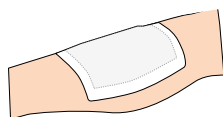

Secure the dressing with suitable bandage, film or tape.

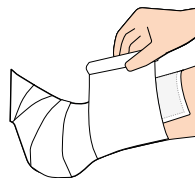

DryMax Non Sterile can be used in conjunction with compression therapy.



- Cost effective
- User friendly
- Increased patient comfort
- Improved wound healing



Item	REF	Pcs/box	Product size (cm)	Absorption capacity*
DryMax Non Sterile 10x10 cm	F20011/10	10	10x10	50 ml
DryMax Non Sterile 10x15 cm	F20015/10	10	10x15	95 ml
DryMax Non Sterile 10x22 cm	F20021/10	10	10x22	150 ml
DryMax Non Sterile 15x20 cm	F20031/10	10	15x20	170 ml
DryMax Non Sterile 20x22 cm	F20041/10	10	20x22	300 ml
DryMax Non Sterile 20x30 cm	F20051/10	10	20x30	400 ml
DryMax Non Sterile 37x56 cm	F20071/10	10	37x56	1600 ml

\*) According to international standard EN 13726



Manufactured by Absorbest AB Sweden  
www.absorbest.se | info@absorbest.se