


Mestopore[®]

Absorbent continent stoma dressing

Mestopore is a self-adhesive absorbent dressing for continent stomas.

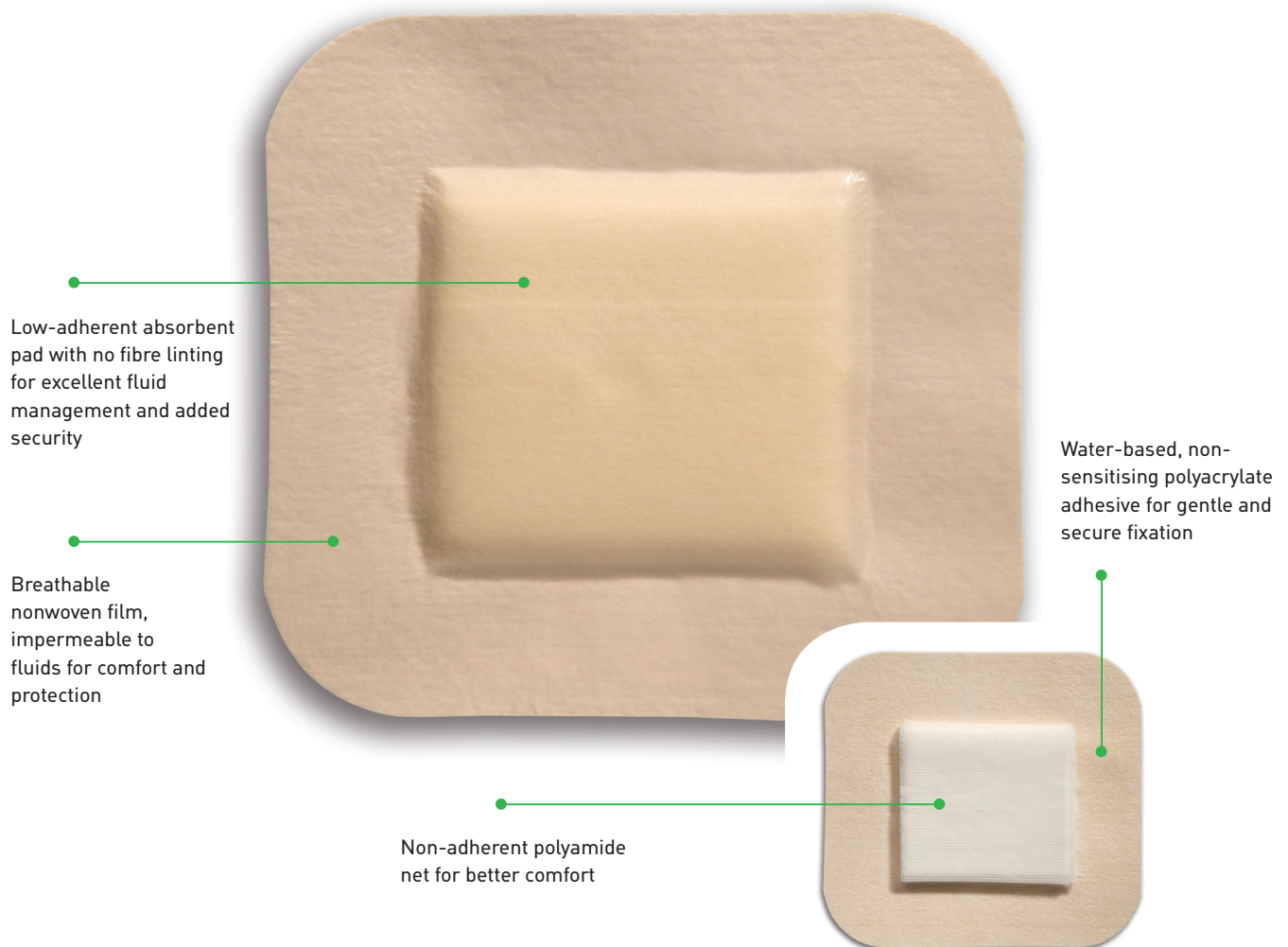

How Mestopore works


Mestopore is a self-adhesive absorbent stoma dressing for continent stomas. The stoma tissue is protected from injuries by a non-adherent contact layer. The stoma secretion is absorbed and retained by a highly absorbent swelling fibre matrix. The self-adhesive border is non-irritant, even with frequent dressing changes, and prevents

the dressing from leaking. The microporous backing material is skin coloured and allows the skin to breathe.


Areas of use

Mestopore is specially developed to be used on patients with continent stoma.


How to use Mestopore

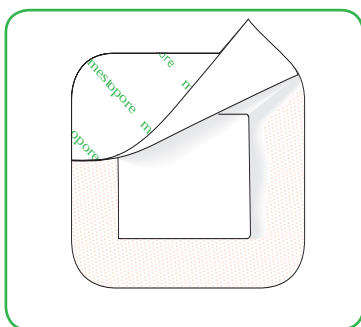

Remove release film from top half of Mestopore.

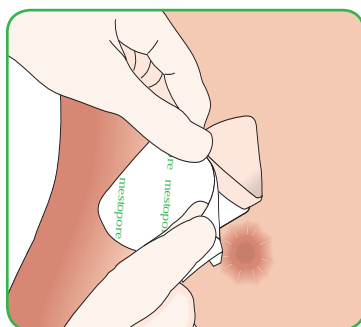

Apply the dressing gently against skin above the stoma.

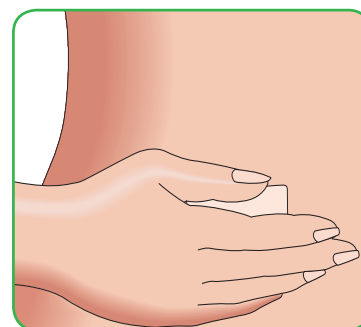

To remove dressing, lift gently and peel back the dressing.

Benefits of Mestopore

- Skin-friendly adhesion
- Non-sensitising
- Adapts well to body contours
- Shower-proof
- No residues
- Easy to use

Note

In case of clinical signs of infection, consult a healthcare professional for adequate infection treatment.



Mestopore is delivered unsterile in a protective shelf container

Art. No	Size cm (inch)	Pad size cm (inch)	Pcs/shelf cont.	Pcs/transp. cont.
665000	9 x 10 (3.5 x 4)	5 x 5.5 (2 x 2.2)	20	160

Mölnlycke Health Care AB (publ)
Box 13080, SE-402 52 Göteborg, Sweden
Phone + 46 31 722 30 00
www.molnlycke.com



MÖLNLYCKE
HEALTH CARE

The symbol and the word mark are both registered trademarks or trademarks pendings of Mölnlycke Health Care.