




Integrated compression zones (Tytex ZoneTech<sup>®</sup>) for the best surgical result

Provides optimum support and skin-friendliness

# Bella

## SEAMLESS POST-OP BRA

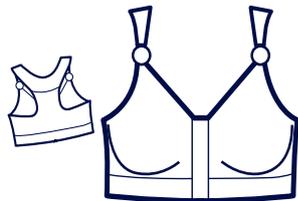

COMPRESSION LEVEL 3 - INTERMEDIATE



Support/  
compression zones





Breathable



4-way stretch



Seamless

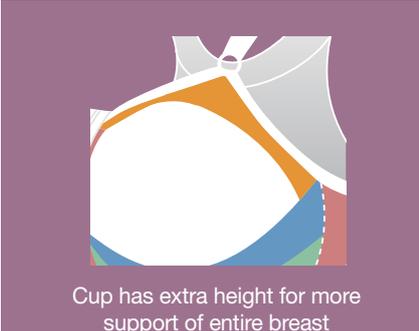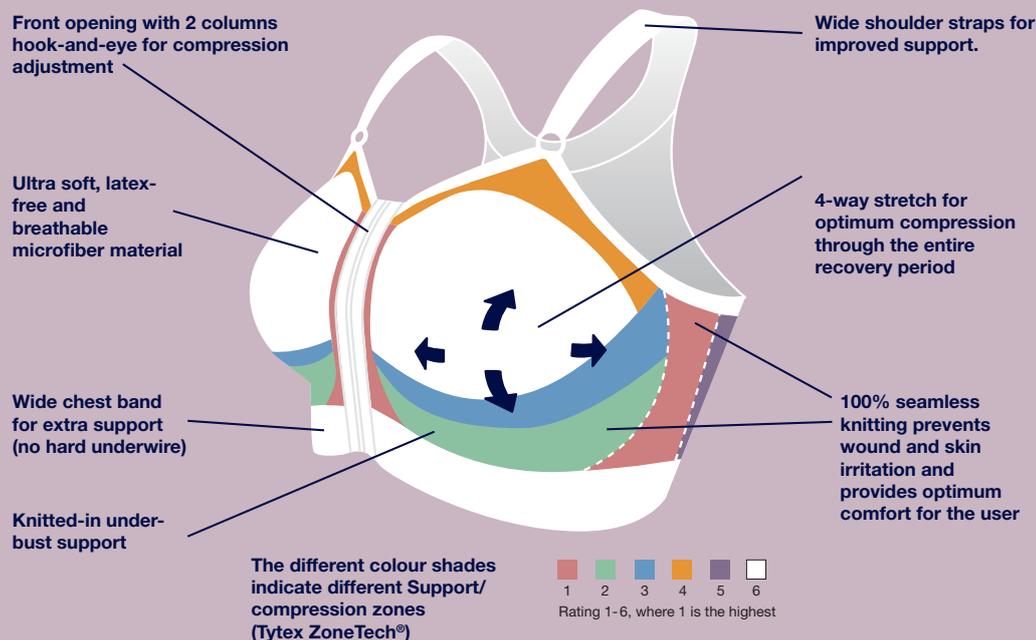

The Bella bra is designed for the time following a surgical procedure to the breast, when the skin is tender and sensitive, and where compression is recommended.

The bra contributes to wound healing and protection of the surgical site, reducing the risk of edema and post-surgical complications.

The 100% seamless knitting increases comfort for the user, prevents pressure points and tissue irritation, resulting in maximum skin-friendliness.

**Indications:**

Medical device for post-surgical use after breast surgery.

**Precautions:**

If you have any questions regarding use and correct fitting, please consult your doctor or nurse. Make sure that any bandages and/or drainage tubes underneath the bra are placed correctly to avoid pressure points and/or obstructed flow from the drainage tubes. For breast augmentations, a stabilizer band for breast implants may also be required. Consult your doctor or surgeon before use. Always follow the surgeon's recommendations. The bra must be removed during radiotherapy treatment, as it contains metal parts. If any unexpected discomfort, wound irritation or similar occurs, please consult your doctor or nurse.

**Recommended sizes**

Band Size	CM	60 cm	65 cm	70 cm	75 cm	80 cm	85 cm	90 cm	95 cm	100 cm	105 cm	110 cm	115 cm
	IN	24 in	26 in	28 in	30 in	32 in	34 in	36 in	38 in	40 in	42 in	44 in	46 in
Cup Size	AA												
	A												
	B		<b>XS</b>	<b>S</b>		<b>M</b>		<b>L</b>		<b>XL</b>		<b>2XL</b>	
	C												
	D												
	E												

\* NOTE: Due to possible post surgical swelling and edema, you may need to go up 1-2 sizes compared to your normal bra size. For more compression go down one size and for less compression go up one size. Important: Always measure yourself to find the right size!

Available colours:



**Composition:**

91 % Polyamide  
9 % Elastane

**Care Handling:**



104°F

Close hook and eye before washing. Wash with similar colours in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.