

GENERAL ENVIRONMENTAL HYGIENE

Actichlor™ and Actichlor™ Plus

Chlorine Disinfectant Range

Actichlor and Actichlor Plus form a comprehensive range of chlorine disinfectant products for all aspects of surface and environmental disinfection.

The Actichlor range offers:

- ▲ Effective chlorine disinfection in both tablet and granule formats
- ▲ Broad spectrum efficacy suitable for a wide variety of applications, including blood spills and sporicidal disinfection
- ▲ Easy to use formats with detailed training materials and guidance available
- ▲ Single-step cleaning and disinfection with Actichlor Plus



Actichlor™ and Actichlor™ Plus

Chlorine Disinfectant Range

Clean and Disinfect with Actichlor™ Plus

The Actichlor Plus tablet combines a chlorine compatible detergent with NaDCC in a single tablet format.

- ▲ Full spectrum of efficacy
- ▲ Excellent cleaning and disinfection performance in one easy step

Disinfect with Actichlor™

Actichlor offers a complete range of chlorine disinfectant products for all aspects of surface and environmental disinfection:

- ▲ Full spectrum of efficacy
- ▲ Wide range of tablet sizes available

MICROBIOLOGICAL PROPERTIES

Disinfectant solutions made with Actichlor or Actichlor Plus are effective against all bacteria, spores, yeasts, moulds and viruses including Hepatitis B and HIV.

Tested to EN 1276, EN 1650 and EN 13704.



Actichlor Granules



Actichlor Plus 1.7g Disinfection Tablets



Actichlor 0.5g Disinfection Tablets



Actichlor 1.7g Disinfection Tablets



Actichlor 2.5g Disinfection Tablets



Actichlor Spill Kit

PRODUCT SPECIFICATIONS

Actichlor Plus Disinfectant Tablets -

Effervescent NaDCC disinfectant tablets with detergent.

Actichlor Disinfectant Tablets -

Effervescent NaDCC disinfectant tablets.

Actichlor Granules - Absorbent NaDCC granules for the disinfection of blood spills.

Actichlor Plus Tablets - Effervescent NaDCC tablets with detergent.

Actichlor Spill Kit - 10 clean up packs, with dilution bottle, disinfection granules and disinfection tablets for blood spills.

TRAINING AND SUPPORT

A dilution bottle is available to assist accurate usage of Actichlor and Actichlor Plus.

The range is also supported by usage posters and a training DVD, available from your Healthcare Manager.



PROVEN.
EFFECTIVE.
EFFICIENT.

APPLICATION AREA

For disinfection of blood spills, general clinical environment.

INSTRUCTIONS FOR USE

To achieve a solution of 1000ppm, dissolve 1 x 1.7g tablet in 1 litre of water; or for a solution of 10,000ppm, 10 x 1.7g tablets in 1 litre of water. Please see dilution tables for additional dilutions.

INGREDIENTS

Each Actichlor and Actichlor Plus tablet contains Sodium Dichloroisocyanurate (NaDCC, troclosene sodium) in an inert effervescent base.

Actichlor Spill Kit - 10 clean up packs, with dilution bottle and disinfection tablets for blood spills.

PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Actichlor Plus 1.7g Disinfectant Tablets	1.7g x 150	3077170
Actichlor 0.5g Disinfectant Tablets	0.5g x 600	3049680
Actichlor 1.7g Disinfectant Tablets	1.7g x 200	3037640
Actichlor 2.5g Disinfectant Tablets	2.5g x 100	3049670
Actichlor 500g Granules	500g	3037680
Actichlor Spill Kit	1	3030520
Actichlor Dilution Bottle	1	408432

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE.

FOR FURTHER INFORMATION PLEASE CALL:
0161 486 7000 (UK) OR
01 468 8456 (IRELAND)

ECOLAB HEALTHCARE

Lotherton Way, Garforth
Leeds LS25 2JY, UK
+44 (0) 2920 854 390
www.ecolab.co.uk
www.ecolab.ie