

# Qufora® IrriSedo MiniGo

For low volume transanal irrigation

## Product components

- Soft pump with a flip lid that connects to the cone
- Rectal cone with a hydrophilic coating that is activated with water
- Accessory: Flex Tube



soft pump



rectal cone



flex tube - accessory

## Product features

### Soft pump with flip lid

- As simple as filling an ordinary bottle, and a click sound confirms when the flip lid is safely closed
- Easy to squeeze soft pump that can be squeezed more than once, and the one-way valve prevents backflow back to the soft pump
- Holds enough water for the irrigation - up to 160\* ml of water can be instilled into the bowels without the need to refill the soft pump

### Smooth, hydrophilic coated cone with a rounded top

- Easy to attach to and remove from the soft pump
- Design to fit with the anatomy of the anal canal
- Ready to use for a smooth insertion and removal, when the hydrophilic coating is activated with water

### Pouch for the single-use cone

- Hygienic handling of the cone
- Handy disposal bag for the cone after irrigation

### Box

- Storage of products between irrigations



\*Individual differences may occur

## Material description

Parts	Material	Softener
Soft pump		
Container	TPE	No softener
Flip lid and belt	PP	No softener
One-way valve in lid	Silicone	No softener
Cone		
Cone	TPE	No softener
Coating	Hydrophilic coating	No softener
Flex Tube		
Tube	TPE	No softener
Lid connector	TPE	No softener
Cone connector	ABS	No softener

## Dimensions and weight

Parts	Dimensions	Weight	
Soft pump	Length	80 mm	41 g
	Width	60 mm	
	Height	95 mm	
	Volume	180 ml	
Cone	Length	59 mm	7,5 g
	Width	39 mm	
	Height	68 mm	
Flex Tube	Total length approx.	680 mm	12 g
	Free length min.	625 mm	

### Recommended usage time

- The soft pump can be used for up to 1 month
- The cone is single-use only
- The Flex Tube is single-use only

### Shelf life

The expiry date is stated on the label.

### Storage

Store in a dry place with a temperature between 0° and 30° C, out of direct sunlight and away from children.

### Disposal

Qufora IrriSedo MiniGo does not require special disposal protocols and may be disposed of in household waste.

### Classification and sterilization

Qufora IrriSedo MiniGo is a Class I Device, non-sterile.

### Certificates

Regulation (EU) 2017/745  
Standards: ISO 13485: 2016

## Design that matters for the environment

- PVC and phthalate-free
- Reduced weight of the soft pump and cone over Qufora® IrriSedo Mini
- Re-designed the box to also serve as storage for the products between irrigations, reducing amount of extra packaging included










### Manufactured by:

Qufora A/S  
Gydevang 28-30  
2450 Allerød  
Denmark


Tel: +45 88 30 05 92  
info@qufora.com  
www.qufora.com

# Qufora® IrriSedo MiniGo

For low volume transanal irrigation

57001-005	Teaching set 8 retail pr. shipper	1 soft pump	5 cones	1 toilet bag
				
57003-015	Base set 8 retail pr. shipper	1 soft pump	15 cones	
				
57003-030	Base set 8 retail pr. shipper	1 soft pump	30 cones	
				

## Accessories:

57007-015	Tube accessory set 8 retail pr. shipper	15 tubes
		

Tillverkare:

MBH-International A/S  
 Gydevang 30  
 DK-3450 Allerød  
 Danmark

**qufora®**

Distribueras av:

OneMed Sverige AB  
 Dockplatsen 12  
 211 19 Malmö  
 E-post  
 kundservice@onemed.com

**ONEMED**