

evercare® inLine Gravity set, Non-Vented, FilterStop, PVC-free REF: 10598, 10599

evercare[®] inLine Gravity Sets are sterile, single use products that are used to conduct intravenous fluids into the vascular system of a patient by connecting to a source of infusion in one end and to an infusion device with vascular access in the other end. The fluid is transferred by the means of gravity and the flow rate is regulated by a roller clamp component. The non-Vented Gravity Sets are sutible for infusion with plastic bags or bottles and no need for venting and to avoid risk of air infusion.

Key Features

- Transparent Soft drip chamber for easy squeezing and filling
- Particle filter at bottom of the drip chamber
- ▶ Hydrophobic filter prevents leakage while priming
- ▶ Rotating luer lock



REF	DESCRIPTION	SIZE - Tube Inner/outer, L(length)	PRIMING VOLUME(ml) Approx	Pack Count dispenser/carton
10598	Gravity set, Non-Vented, FilterStop, PVC-free	Ø 3/4.1 mm, L 180cm	11.7	20 / 80
10599	Gravity set, Non-Vented, FilterStop, PVC-free	Ø 3/4.1 mm, L 320cm	20.8	15 / 60

Sales Unit: Transport Carton

Page 1 2024-06-13

Regulations

The product complies with legal requirements for medical devices. EO sterilized according to valid version of ISO 11135 standard.



The products covered by this data sheet do not contain substances classified as carcinogenic, mutagenic or toxic for reproduction (CMR), including any of the **REACH regulation:** Substances of Very High Concern (SVHC) as listed in the latest available version of the Candidate List published by European Chemicals Agency (ECHA). EN ISO 10993-01 **Biological evaluation:** Compliance with legal Medical Device Regulation (EU) 2017/745, MDR Class IIa requirements: **Compliance with product** EN 8536-8, EN ISO 80369-7 standards: EN ISO 13485 **Quality standards:** Label information and

EN ISO 15223-1, EN ISO 20417

Material

packaging compliance:

Product Part	Raw Material & Characteristics	
Spike cover	PP, transparent	
Non-vented spike	ABS, transparent blue	
Drip chamber	Polyolefin alloy, PVC-free	
Disc with filter	ABS + PTFE, white	
Tube	Polyolefin alloy, PVC-free	
Roller clamp	ABS, roller body blue, roller wheel white	
Rotating luer lock	ABS & PC, transparent	
Purge filter cap	HDPE, light blue	

Page 2 2024-06-13

Warnings

Warnings & Recommendations

The use of this product is restr	cted to a qualified doctor or nurse.
----------------------------------	--------------------------------------

Read the instructions before use. The device should be used in accordance with the instructions.

Do not use for transfusion of blood or blood components.

The device is guaranteed sterile and pyrogen-free if the individual package has not been opened or damaged. Do not use if package is damaged or if protective caps are loose or missing.

Do not over-tighten a luer beyond the friction fit as this may damage both the luer and another connecting device.

Do not use any instruments for clamping the tubing or luer tightening.

Re-use may lead to infection or other illness/injury.

Storage Recommendation	Recommended storage in clean, dry space between 15 - 25 °C. Products should be protected from direct sunlight, other intensive light sources and ozone.
Disposal Recommendation	Dispose in accordance with local regulations.
Shelf Life	5 years

Packing Information

Package	Material	Size (L x W x H) mm
Single Pack	Medical grade paper, PP/PE film	235 x 100
Dispenser Pack	PP	305 x 460 x 0
Transport Carton	TCF quality, Carton	400 x 295 x 287
Packing Tape	PVC-free, Transparent	N/A

Barcodes / UDI-DI

REF	Single Pack	Dispenser	Transport Carton
10598	06438129124200	N/A	06438129524208
10599	06438129124217	N/A	06438129524215

Evercare Medical AB is a partner that enables better lives for patients, facilitates everyday life for healthcare employees and contributes to lower care costs. We strive to reduce the environmental impact and, together with our suppliers, to improve working conditions throughout the value chain.