FLOWART® NEEDLE-FREE VALVE PORT

DESCRIPTION

FlowArt® is a single-use, disposable connector intended for use with a vascular access device for administration of fluids during IV therapy.

The product is a needle-free valve with an integrated silicone seal that protects the patient and healthcare staff from needlestick accidents during intravenous therapy by minimizing needle usage.

FEATURES

- ☑ Microbial barrier swabable silicon stem with a split septum, flat seal to allow enhanced disinfection and drying time.
- Clear transparent housing to allow clear view into the fluid pathway for enhanced flushing.
- ☑ Compatible with TPN solutions, blood and oncology drugs.
- ☑ Withstands pressure injection up to 400 psi at 10 ml/second flowrate.
- Allows multiple accesses.
- Z Conforms to ISO 80369-7.
- ☑ MRI and CT suitable.
- ✓ Shelf Life: 5 Years
- ☑ in compliance with MDD 93/42/EEC. EC Certificate No: 1984-MDD-18-524
- ☑ Photoprotective (-PH) versions are in amber color.
- ☑ Latex-free, PVC-free, DEHP-free

Part	Material
Valve Body	Copolyester
Swabable septum	Silicone

DIAGRAM



PRODUCT DATA

Characteristic	Specification	
Product Code	A1010,	
	A1010-PH,	
	A1010M*	
Product Name	Needle-free Valve Port	
Priming Volume (mL)	0.085	
Displacement (mL)	-0.02 (negative)	
Flow Rate (mL/min @ 1m water head pressure)	312	
Flow Rate (mL/min @ 0.5 bar pressure)	645	
Dimensions: Width x Length x Height (cm)	1.3 x 2.5 x 2.5	
Weight (g)	2.73	

*blue MLL

LABELLING AND PACKAGING

Each unit is labelled on its blister and packaging container. Each container is labelled with the following traceability information: Quantity, Product description, Product date, Lot number

Product	Unit GTIN	Shelf Pack /Box Content (Units)	Box GTIN
A1010	08699443580013	250	18699443580010
A1010-PH	08699443580013	250	18699443580010
A1010M	08699443580013	250	18699443580010

TDS.001 _Rev07_19.08.2024

Asset® Medikal Tasarim San. Tic. A.S. is an ISO 13485 registered company, Certificate No.: M 8072.

FlowArt is a registered trademark of Asset Medikal.

Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. ©2024. All rights reserved.

Asset® Medical, Inc.

Asset® Medikal Tasarim San. Tic. A.S.

5965 Village Way E105-300 San Diego, CA 92130, USA

info@assetmedical.com www.assetmedical.com Ikitelli OSB Mahallesi, 17. Cadde, No:17 Basaksehir, Istanbul - Türkiye

: +90 212 494 27 27

: +90 212 494 34 37

