HIGH LEVEL DESINFIOINTI

ANIOXYDE 1000 LD ja BANDELETES ANIOXYDE 1000 LD (Testiliuskat)

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ECOLAB®
www.ecolab.com

ANIOXYDE 1000 LD

High-level disinfection for thermosensitive instruments and endoscopy devices

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
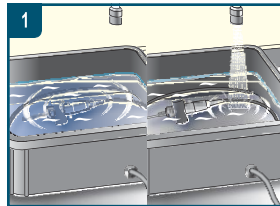
(koeliuskalla havaittu raja-arvo)


KÄYTTÖOHJE

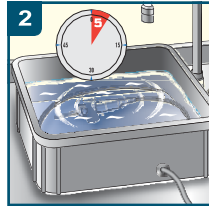
Pre-disinfection step: Clean the medical device with a detergent or a pre-disinfectant product. Rinse thoroughly. For endoscopic equipment: brush channels.

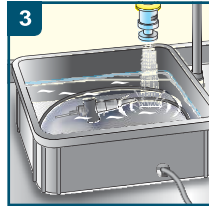Disinfection step: Pour the activated solution (see activation procedure on the label/protocol). Cover the basin. Respect the minimum contact time. For endoscopic equipment: make solution circulate inside channels. Do not add any product into Anioxyde 1000 LD.

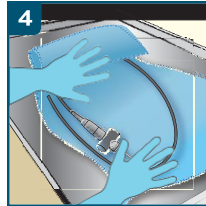Rinse thoroughly with sterile or filtered water (0.2 µm) to eliminate any trace of the product. For endoscopic equipment: rinse the internal and external parts of the medical device.

Dry with a single-use towel. Keep the medical device as aseptically as possible until its next use.

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CLAIM/CONDITIONS	CONTACT TIME
Bactericidal - According to EN 13727/EN 14561, clean conditions	5 min
Yeasticidal - According to EN 13624/EN 14562, clean conditions	5 min
Tuberculocidal - According to EN 14348/EN 14563, clean conditions	5 min
Fungicidal - According to EN 13624/EN 14562, clean conditions	5 min
Mycobactericidal - According to EN 14348/EN 14563, clean conditions	5 min
Virucidal - According to EN 14476/EN 17111, clean conditions	5 min
Adenovirus - According to EN 14476/EN 17111, clean conditions	5 min
Norovirus - According to EN 14476/EN 17111, clean conditions	5 min
Sporicidal against B. Subtilus / C. Difficile: According to EN 17126, clean conditions	5 min

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PRODUCT	PACKAGING UNIT	ORDER CODE
Anioxyde 1000 LD	4 x 5 L canister + integrated activator	XXXXX
Anioxyde 1000 LD	Test strips of Peracetic Acid rate 1000 ppm (50 per box)	XXXXX

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© 2022 Ecolab. All rights reserved. HC-IR-A.OXY1000LD-5039-2-EU-0622
ECOLAB DEUTSCHLAND GmbH