CareTainer® 70 C€



registered medical product



Dimensions 180 x 135 x 22 mm (L x W x D)

Weight / piece 12 g

Product information Weight of the bundle: 15.4 kg

> Dimensions of the bundle: 756 x 396 x 417 mm (L x W x D)

Package contents: 1,200 pieces

Number of packs/pallet*: 18

Number of pieces/pallet*: 21,600 pieces

Dimensions of pallet*: 1,200 x 800 x 2,646 mm (L x W x D)

*Euro pallet – for other pallets, different packaging information may apply

no printing

Colour

white

UDI-DI: 4260676590076

Non contractual document. Changes excepted. Weights & measures are of average data. You can find more information about this product on the following page.



Rue Pierre Clugnet

56350 Allaire

Tél: +33 (0)2 99 71 95 22

An der Kaje 1 26931 Elsfleth **GERMANY** Tel: +49 (0) 4404 9874-0 www.omnipacgroup.com

Omni-Pac Group

Second Avenue, Flixborough Industrial Estate Scunthorpe North Lincolnshire DN15 8SD UNITED KINGDOM Tel: +44 (0)1724 847 207

Ask for our FSC®-certified products.



Version (ENG) 01.07.2021

Supplement to the product data sheet



Medical products

Product description

Composition (materials used):

Moulded fibre products made of recycled paper and fresh fibre or only recycled paper with the addition of process-related and functional additives.

Intended use:

As medical devices for use in the medical field, e. g. in hospitals, old people's homes, nursing homes etc. and doctor's surgeries for the collection of body fluids or to drop instruments according to their use and before their cleaning.

Restriction of use:

Hot and/or frozen substances, alcoholic solutions, microwave, sterilization processes. Do not reuse!

Storage and transport conditions:

We recommend a storing temperature of $15-30^{\circ}$ C and a relative humidity of 50-60% for these products. If stored in a non-air-conditioned environment, please protect the products from humidity as well as direct sunlight and too humid and too dry air (blow pipe).

We recommend a shelf life of five years under the specified conditions.

Other:

Goods are sensitive to mechanical impact and must not be subjected to impacts or high pressure be exposed. Goods must be protected from moisture and humidity. Use outside the intended purpose is the responsibility of the user.

Version (ENG) 04.10.2021



Omni-Pac Group

An der Kaje 1 26931 Elsfleth GERMANY Tel: +49 (0) 4404 9874-0 www.omnipacgroup.com Second Avenue, Flixborough Industrial Estate Scunthorpe North Lincolnshire DN15 8SD UNITED KINGDOM Tel: +44 (0)1724 847 207

