

DuoDERM® Extra Thin



PRODUCT DESCRIPTION¹:

DuoDERM Extra Thin dressings are sterile, thin hydrocolloid dressings consisting of an inner (wound contact) layer of hydrocolloid contained within an adhesive polymer matrix and an outer layer of polyurethane film. The dressing provides a moist wound environment that favours the healing process by aiding autolytic debridement, facilitating the migration of epithelial cells, and allowing non-traumatic removal of the dressing without damaging newly formed tissue. The adhesive layer contains elastomeric polymers which enhance the dressing's ability to contain wound exudate by forming a cohesive gel.

DuoDerm Extra Thin Dressings

Dressing Size cm	CVT number	pieces/ package
7,5 x 7,5 cm	187951	5
10 x 10 cm	187954	5
15 x 15 cm	187956	5
5 x 10 cm	187959	10
10 x 15 cm Oval	651008	5
9 x15 cm Post Op	187920	10
9 x25 cm Post Op	187921	10
9 x35 cm Post Op	187922	10

PRECAUTION AND WARNINGS¹:

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

- 1) Appropriate supportive measures should be taken where indicated (e.g., use of graduated compression bandaging in the management of venous leg ulcers, or pressure relief measures in the management of pressure ulcers).
- 2) The autolytic debriding environment produced by the dressing may result in the wound appearing larger after the first few dressing changes.
- 3) Colonization of chronic wounds is common and is not a contraindication to the use of the dressing. The dressing may be used on infected wounds under medical supervision together with appropriate therapy and frequent monitoring of the wound. In the presence of a clinical anaerobic infection, occlusive therapy is not recommended.
- 4) Rarely, irritation and/or maceration of the skin, hypergranulation and allergic reactions have been reported.
- 5) Frequent dressing changes on wounds with compromised peri-ulcer skin is not recommended.
- 6) DuoDERM Extra Thin has not been studied with drugs other than topical steroids.
- 7) Sensitivity to the components of the steroid may be enhanced by occlusion. If irritation develops, discontinue use and contact a physician.
- 8) Occlusion of topical steroids has the potential to enhance systemic absorption. For a full discussion of adverse effects, precautions, and all local side effects and contraindications, the package insert of the steroid to be used should be consulted.

DIRECTIONS FOR USE¹:

Always check instructions for use please see IFU in product box before clinical use



ADDITIONAL INFORMATION:

Can be used with compression therapy ¹	YES
Exudate management ¹	Dry to lightly exuding wounds
MRI safe	YES
Waterresistent	YES

INDICATIONS FOR USE¹:

Chronic Wounds: Dry to lightly exuding wounds, e.g., leg ulcers, superficial pressure sores.

Acute Wounds: Surgical wounds, e.g., post-operative wounds, minor burns, abrasions or lacerations. May be used as a secondary dressing in combination with other modern wound care products, e.g., alginates, hydrogels.

Dermatological indications: Indicated for use either alone or with topical steroids in the management of psoriasis or other recalcitrant conditions where the enhanced response offered by occlusion is recommended by the physician. When used together with topical medication, this product should be used under the direction of a physician.

DURATION OF USE¹:




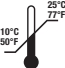
Maximum recommended wear time is up to seven days.



Do not re-use

CONTRAINDICATIONS¹:

Not intended for use on patients with known sensitivity to the dressing or its components. DuoDERM Extra Thin Dressings should not be used with topical steroids for which occlusion is not indicated.

STERILIZATION ¹ :	
Sterile	YES
Method	Gamma Sterilized
CERTIFICATIONS ¹ :	
CE mark	CE 2797
MDR 2017/745	Class IIb
MATERIAL SPECIFICATIONS ¹ :	
List of ingredients	 Polyurethane film, Sodium carboxymethylcellulose, Gelatin, Pectin, Adhesive Polymers.
STORAGE AND HANDLING ¹ :	
Before Use	Keep away from sunlight
  	Keep dry.
	Store at room temperature (10°C - 25°C/50°F - 77°F).
After use	After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

References:

1. DuoDerm Extra Thin Dressing IFU Revised 2021-12
1268507G3

Convatec Limited

First Avenue,
Deeside Industrial Park Deeside,
Flintshire CH5 2NU,
UK



UNOMEDICAL A/S

Aaholmvej 1-3, Osted
4320 LEJRE
DENMARK



Convatec Denmark A/S

CVR 31477093,
Transformervej 14, 1. , 2860 Soborg,
Denmark
Tlf.: +45 48 16 74 74
www.convatec.dk

Convatec Norway AS

Org. nummer: MVA 992 780 118
Visiting address:
Nils Hansens Vei 8, 0667 Oslo
Postadresse:
Postboks 6464 Etterstad, 0605 Oslo
Tlf.: +47 21 09 67 90
www.convatec.no

Convatec (Sweden) AB

Organisationsnummer: 556754-8473,
Box 3096, 16973 Solna, Sweden
Visiting address:
Gustav III:s Boulevard 42, plan 9,
16973 Solna, Sweden
Tlf.: +46 020-21 22 22
www.convatec.se

Convatec Oy

Y-tunnus 1644589-9,
Aviapolis, Karhumäentie 3, 01530 VANTAA,
Finland
P: +358 020 7659 600
www.convatec.fi