




TYTEX[®]
BETTER HEALTHCARE TOGETHER

LeggyFix[®] by Tytex

Leg bag holder

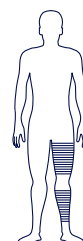

LeggyFix[®] is specially designed to fixate a urine leg bag, without impeding appearance. The weight of a full leg bag is supported by the whole product. This way, the tension required to hold a bag in place is distributed evenly around the leg, rather than being concentrated in two narrow rings at the top and the bottom of the bag.

Features:

- Easy to use
- Safe retention of leg bag for optimum support
- Tension is distributed evenly around the leg
- Provides discretion
- Very soft, stretchy and breathable material, that is comfortable to wear
- Fits left and right leg equally well
- Fits both thigh and calf
- With a slit at the bottom for urine bags with bottom tap opening
- Can hold up to 750 ml/25 oz (leg bags)
- Practical colour code for size indication
- Washable at 60°C/140°F
- OEKO-TEX certified

Use:

For fixation of urinary leg bags. LeggyFix[®] must be folded up, thereby providing a pocket for the urine bag. When LeggyFix[®] is folded, the slit at the bottom will appear, and the valve of urine bags with bottom tap opening can be pulled through.

Composition:

63% Polyester, 37% Elastane.

Washing instructions:




Sizes available:

LeggyFix[®] XS-XL.

Instruction for use:



1. Pull the colour-coded end of the sleeve over the leg with seams on the outside.

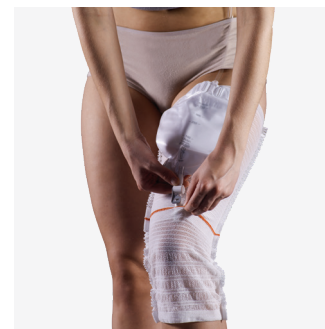

2. Pull the valve of the urine bag through the hole placed at the red line of the LeggyFix.



3. Pull up the lower part of the sleeve over the leg bag.



4. LeggyFix[®] is in place.



Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485