

25XL & 25XL Protected Access

Container arrives pre-assembled

The lid is already attached to the container so no assembly required.



Insert protected access feature

Insert the protected access feature until an audible click can be heard.

Only applicable if using protected access container



Fill in the label before and after use

This ensures a permanent audit trail.



Never fill above the line

Always replace your sharps container when full to avoid the risk of needle stick injuries.



Insert container into trolley

The 25XL container must be used with the 25XL trolley. This guarantees maximum stability. It allows for easy mobility of the container and for sharps to be disposed of at the point of use.

When the container is not in use engage temporary closure



Final closure

The final closure feature must be engaged when the container is full.

To engage, keep the container in the trolley, close the lid to the temporary closure position then firmly apply pressure to all four corners of the lid until four audible clicks can be heard.



FOR MORE INFORMATION:

✉ sharpsafesales@vernagroup.com

🌐 vernacare.com 📞 +44 (0)1495 235800