

User guide

ELERS MEDICAL[®] Privacy Curtains

Introduction

ELERS MEDICAL[®] Curtains are used for providing privacy for patients and staff and to help maintain privacy and dignity for the patient. The curtains are made from polypropylene and available in various sizes and colours. The self auditing curtain label helps the user to implement good changing practises.

- Fr protected and independently tested to BS 5867 Part 2 Type C NFPA 701, EN 13773
- Requiring no washing
- 100% recyclable

How to use

Open the box carefully. Do not use sharp objects to open the box. Each curtain is individually packed in a plastic sleeve. Pull down the plastic sleeve to reveal the self auditing label and write the date of installation on to each label. Remove the plastic sleeve entirely once the curtain has been hung. ELERS MEDICAL Disposable Curtains are designed to be compatible with most commonly used curtain track systems. The curtains are supplied according to individual requirements with or without hooks.

Storage

Curtains can be stored in dry conditions in normal room temperatures. ELERS MEDICAL would recommend the curtains are kept in the box. Store the box upright.

Curtain replacement

ELERS MEDICAL would recommend that curtains are replaced at least once a year, every 6-12 months, unless visibly soiled or contaminated*. The frequency of replacement is dependent on individual hospital or trust guidelines. In the event the curtains are visibly soiled or contaminated, immediate change for those curtains is recommended.

Disposal

The curtain fabric is made from polypropylene (PP) and the hooks and eyelets are made of plastic (PPE). Curtains can be 100% recycled or disposed of as per local hospital protocols.

*ELERS MEDICAL only offers replacement timeframes as recommendations and makes no guarantees regarding the replacement of disposable curtains.