Vastrip® 2+

- GB** Disposable vein stripper
- DE** Einmal-Venenstripper
- FR** Stripper veineux à usage unique
- ES** Fleboextractor desechable
- IT** Stripper monouso
- NL** Venenstripper voor eenmalig gebruik
- FI** Kertakäyttöinen laskimostripperi
- DK** Éngangsvaricestripper
- SE** Varicerstripper för engångsbruk
- NO** Varicestripper til engangsbruk
- GR** Μιας χρήσης εκριζωτής φλεβών
- HU** Eldobható érkihúzó

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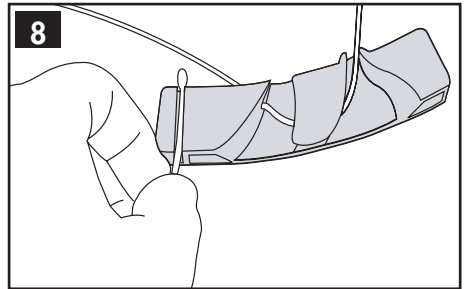
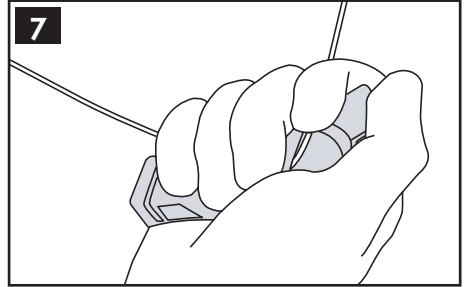
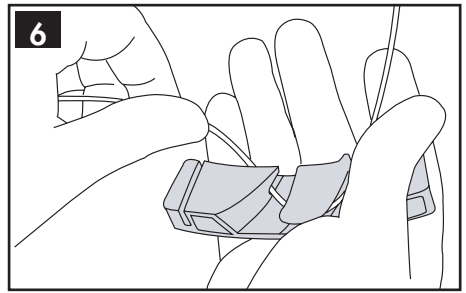
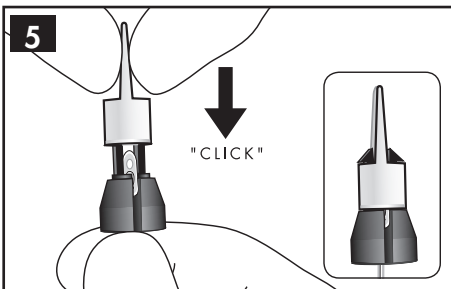
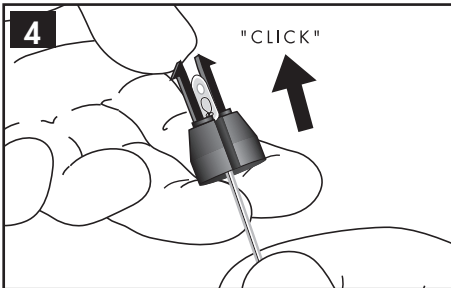
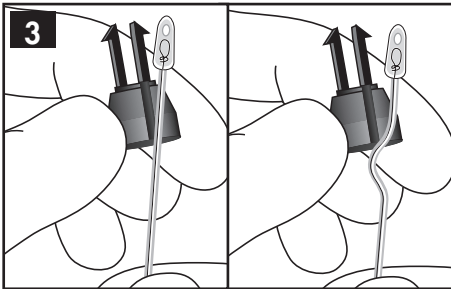
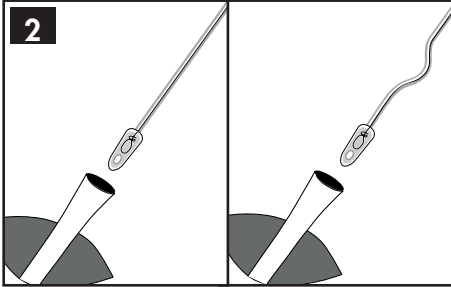
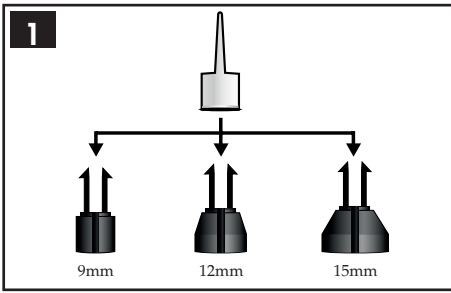
- 1) Vastrip® 2+ is intended for conventional stripping in two directions, or for invagination stripping, in varicose vein treatment in lower extremities in adults. Vastrip 2+ is supplied with olives in three sizes; 9, 12 and 15 mm, and a white lock-cap. The olives fit on either end of the stripper.
- 2) Insert either the straight tip or the spiral tip into the cut vein.
- 3) After insertion, choose an olive of appropriate size. Fasten the olive on the tip of the stripper by inserting the tip (straight or spiral) into the slit of the chosen olive.
- 4) Pull the olive upwards until it has seated firmly as near the tip as possible.
- 5) Secure the olive by positioning the white lock-cap on to the olive.

Important! Ensure that the olive is tightly secured by the white lock-cap.

Vastrip 2+ is a single-use product. If reused Mediplast cannot guarantee the functionality nor the safety of the product. Reuse may lead to infection.

Using the handle

To facilitate stripping, the handle can be used to improve the operator's grip on the stripper.

- 6) Position the stripper in the handle.
- 7) Pull.
- 8) It may be convenient to place the loose end of the stripper in the slot at the left hand side of the handle.

If it is necessary to move the handle further along the stripper:

- Release the stripper from the slot where it enters the handle by tilting the handle downwards from the stripper.
- Slide the handle along the stripper to the required position.
- Re-secure the stripper in the handle.

Ref	Packaging
67321	1x10

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