User manual Face Shield 23670110

The face shield is designed to protect the wearers eyes, nose and mouth from splashes, sprays, and spatter of liquids.

Warning

This product is supplied in multi-pack, in each package there is only one user manual. The user manual shall be made available to all users by posting it where the face shields are distributed or by other means.

The face shield should be checked before each use. Scratched or damaged face shields should be disposed of. It is not possible to repair the face shield.

The headband and foam, when in contact with skin, can cause allergic reactions to susceptible users.

Applicable laws and approvals

The product is designed and tested to fulfil the requirements set forth in PPE EU 2016/425 as a category II PPE and with applicable parts of standard EN166:2001.

Testing and type certification have been made by RISE Research Institutes of Sweden AB, Notified Body no 0402, certificate number C900141, Issue 1, dated 2020-06-24.

Instructions for use

Remove the protection film before first use. Hold the face shield against the face and pull the head band over your head.

Adjust the length of the head band by pulling the strap tab while holding the counterpart of the band with your other hand. Do not overtighten as it will be uncomfortable after some time. You can extend the band if necessary.

When you have the right adjustment, bend the pulling tab backwards to lock the adjustment.

Cleaning

The face shield can be cleaned using a damp cloth with a disinfectant, ethanol or isopropanol is recommended.

NB! Do not dip or soak the foam with alcohol or disinfectants as it will damage the foam.

Storage

Storage should be made in a clean, dry place to avoid dust, scratching, and unnecessary wear. Do not store in direct sunlight or in temperatures above 30 degrees or below 5 degrees Celsius. Expected shelf time in un-opened packaging is more than 10 years.

Recycling

The packing should be recycled as cardboard and the inner-bag as soft plastic (PE). The face shield should be disposed as combustible waste.

Marking

The face shield is marked "BRIGO 1 S - EN166 3 S".

The first digit specifies the optical class according to EN166 (class 1). The last digit (3) means that the product is designed to protect against droplets and splashes of liquid. The S means that the face shield fulfils the increased robustness mechanical requirements of EN166.

Manufacturer

This product is manufactured by Brigo AB, Kryptogatan 20, 431 53 MÖLNDAL, SWEDEN.

Declaration of conformity and product documentation is available on our website www.brigo.com/faceshield

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Ce produit est fabriqué par Brigo AB, Kryptogatan 20, 431 53 MÖLNDAL, SUÈDE.
Dit product is geproduceerd door Brigo AB, Kryptogatan 20, 431 53 MÖLNDAL, ZWEDEN.