CPAP&VPAP Mask and Accessories - CPAP Tubing

REF 70243-150

Date of Manufacture:

Use by:

Storage temp: 15°C ~ 25°C
Storage relative humidity: <85%

Contains : one (1) set

Indication for use: connect with ventilation circuit and equipment

Material: Hytrel, silicone ; Dimension/Length: 22x1500mm ; Cuffs: 22/22mm ID

Instructions for use:

- * Prior to patient connection, insure that all connections are secure and pressurize circuit to confirm there is no leakage and / or occlusion of the circuit.
- * Test interface before use according to the equipment manufacture's recommendations and verify on monitor.
- * Store in a cool dry place.
- * Intended for use by qualified personnel trained.
- * For best performance, use within 5 years after date of manufacture.

Caution:

Leakage, degradation, broken, deformation of this circuit may result in either harmful retention of residual gas or dangerously increase its breathing resistance.

Procedure to delivery the contaminate

If the component contaminated with vomit, blood or secretions during ventilation, please disconnect the device and clear as follows :

- Disassembly the contaminated part
- Rinse the components in water
- If the contaminate still does not clear, replace with another

Disinfection and Sterilization

If the circuit or component contaminated with vomit, blood or secretions during ventilation, please disconnect the device and clear as follows:

- Disassembly the breathing system and wipe all parts with neutral cleaning detergent
- Disinfected by 2% Glutaraldehyde for 10min or autoclaved up to 134°C for 15min
- Make sure there is no breakage or leakage

Manufactured by
HSINER Co.,Ltd.
No. 312, Jhongshan Rd., Shengang Dist.,
Taichung City 429, Taiwan
Tel: 886-4-25152480
Fax: 886-4-25152482
Email: sales@hsiner.com
Web Site: <http://www.hsiner.com>

EU Representative
mdi Europa GmbH
Langenhagener Str. 71,
D-30855 Langenhagen
Tel: +49 511 3908 9530
Fax: +49 511 3908 9539
<http://www.mdi-europa.com>

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