unico graber safety eyewear

- D** Information für den Benutzer
- F** Information pour l'utilisateur
- I** Informazioni per l'utilizzatore
- GB** Information for users
- E** Información para el usuario
- P** Informação para o usuário
- S** Bruksanvisning
- SE** Käyttöohje

- DK** Information til bruger
- NL** Informatie voor de gebruiker
- TR** Kullanıcı talimatları
- HU** Információ a felhasználó számára
- PL** Informacja dla użytkownika
- R** Информация по эксплуатации
- CZ** Informace pro uživatele
- UAE** معلومة للمستخدم

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GB These safety spectacles /goggles have been tested by a Notified Body¹ in accordance with PPE Regulation (EU) 2016/425 and fulfill the requirements of the standard EN 166: 2001. The safety spectacles /goggles may only be used for work in the areas of application indicated, as the safety lenses have to be matched to suit the application. They cannot provide sufficient protection in other areas of application. To clean the safety spectacles /goggles, wash them under running water with washing-up liquid and then dry them with a soft cloth. Automatic cleaning in a cleaning and disinfection unit and subsequent sterilisation is essentially possible. It should be noted, however, that this will diminish the effectiveness of the anti-fog coating. Heavily tinted lenses (e.g. welders' protection filters) may not be used in colourless frames. The period of obsolescence of the PPE and its components, as already described, is three years as from the date of production, as long as it is stored correctly, as well as a further five years in use. Should the frame show any cracks or breaks, the safety spectacles /goggles should no longer be used. Lenses that are scratched or damaged in any other way need to be exchanged. It is therefore mandatory that they are checked regularly. The Unico Graber packaging serves to protect the lenses from mechanical, chemical and thermal influences. For safety spectacles/goggles with filters, please refer to the filter technical documentation for the light transmission curve. The safety spectacles/goggles must not be stored at temperatures that either rise above 50 °C or fall below –10 °C. The numbers preceding the manufacturer's letter code indicate the protection level.

Note: Eye protection equipment for protecting the wearer from high speed particles may constitute a risk to the wearer as a result of the impact, if worn over customary corrective spectacles. Any materials that come into contact with the skin of the wearer may cause allergic reactions in sensitive people. If protection from high speed particles is necessary at extreme temperatures, the eye protection equipment selected must be marked with the letter "T" directly after the letter indicating the impact load permitted, i.e. FT BT. If the letter T does not follow the letter indicating the impact load permitted, the eye protection equipment may only be used at room temperature. Only tested spare parts (accessories) may be used and/or any spare parts whose conformity has expressly been confirmed by Unico Graber AG.

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Declaration of Conformity: www.unicograber.com/Standards and Directives

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