Infant Flow[®] - Low Pressure nCPAP System

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| | | |
|----------|--------------------------|----------|
| 777002XS | Nasal Masks, extra small | 10 st/fp |
| 777002S | Nasal Masks, small | 10 st/fp |
| 777002M | Nasal Masks, medium | 10 st/fp |
| 777002L | Nasal Masks, large | 10 st/fp |
| 777002XL | Nasal Masks, extra large | 10 st/fp |

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| | | |
|----------|---------------------------|----------|
| 777000XS | Nasal Prongs, extra small | 10 st/fp |
| 777000S | Nasal Prongs, small | 10 st/fp |
| 777000M | Nasal Prongs, medium | 10 st/fp |
| 777000L | Nasal Prongs, large | 10 st/fp |
| 777000XL | Nasal Prongs, extra large | 10 st/fp |

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| | | | |
|----------|---------------------------------------|----------|----------|
| 777040XS | Infant Flow LP Headgear, extra small | 17-21 cm | 10 st/fp |
| 777040S | Infant Flow LP Headgear, small | 21-26 cm | 10 st/fp |
| 777040SM | Infant Flow LP Headgear, small/medium | 24-28 cm | 10 st/fp |
| 777040M | Infant Flow LP Headgear, medium | 26-32 cm | 10 st/fp |
| 777040L | Infant Flow LP Headgear, large | 32-37 cm | 10 st/fp |
| 777040XL | Infant Flow LP Headgear, extra large | 37-42 cm | 10 st/fp |

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



EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands

Australian Sponsor:
 Vyair Medical Pty Ltd
 Suite 503, Level 5
 Building C, Foundation Park
 11 Talavera Road
 Macquarie Park, NSW, 2113
 Australia

REF 7772020LP 7772000LP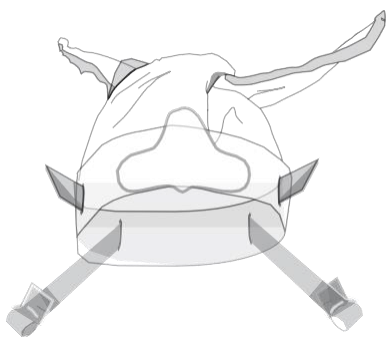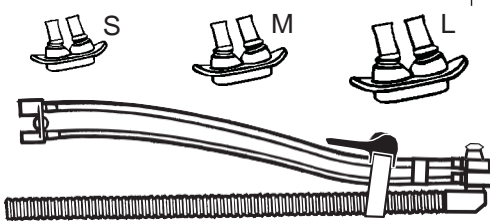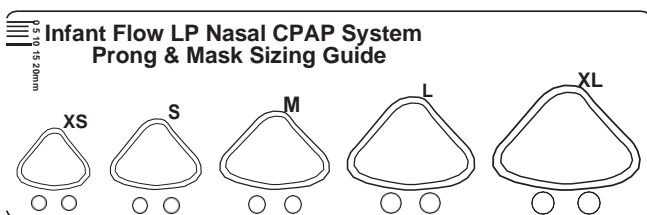

- 000 **REF 777010**
- 00 **REF 777012**
- 0 **REF 777014**
- 1 **REF 777016**
- 2 **REF 777018**
- 3 **REF 777020**
- 4 **REF 777022**
- 5 **REF 777024**
- 6 **REF 777026**
- 7 **REF 777028**
- 8 **REF 777030**
- 9 **REF 777032**

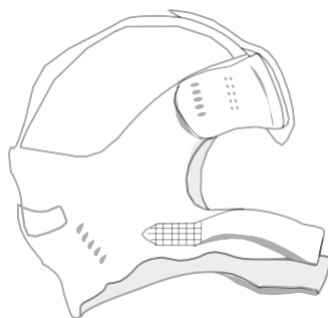- REF 777040XS**
- REF 777040S**
- REF 777040SM**
- REF 777040M**
- REF 777040L**
- REF 777040XL**

| Vd | 1.7ml | 2.8ml | 3.9ml | 5.9ml | 8.1ml |
|------------|-----------------|----------------|----------------|----------------|-----------------|
| REF | 777002XS | 777002S | 777002M | 777002L | 777002XL |
| REF | 777000XS | 777000S | 777000M | 777000L | 777000XL |

DO NOT PLACE
IN TRASH



DO NOT
RE-USE






Infant Flow™ LP System

HEADGEAR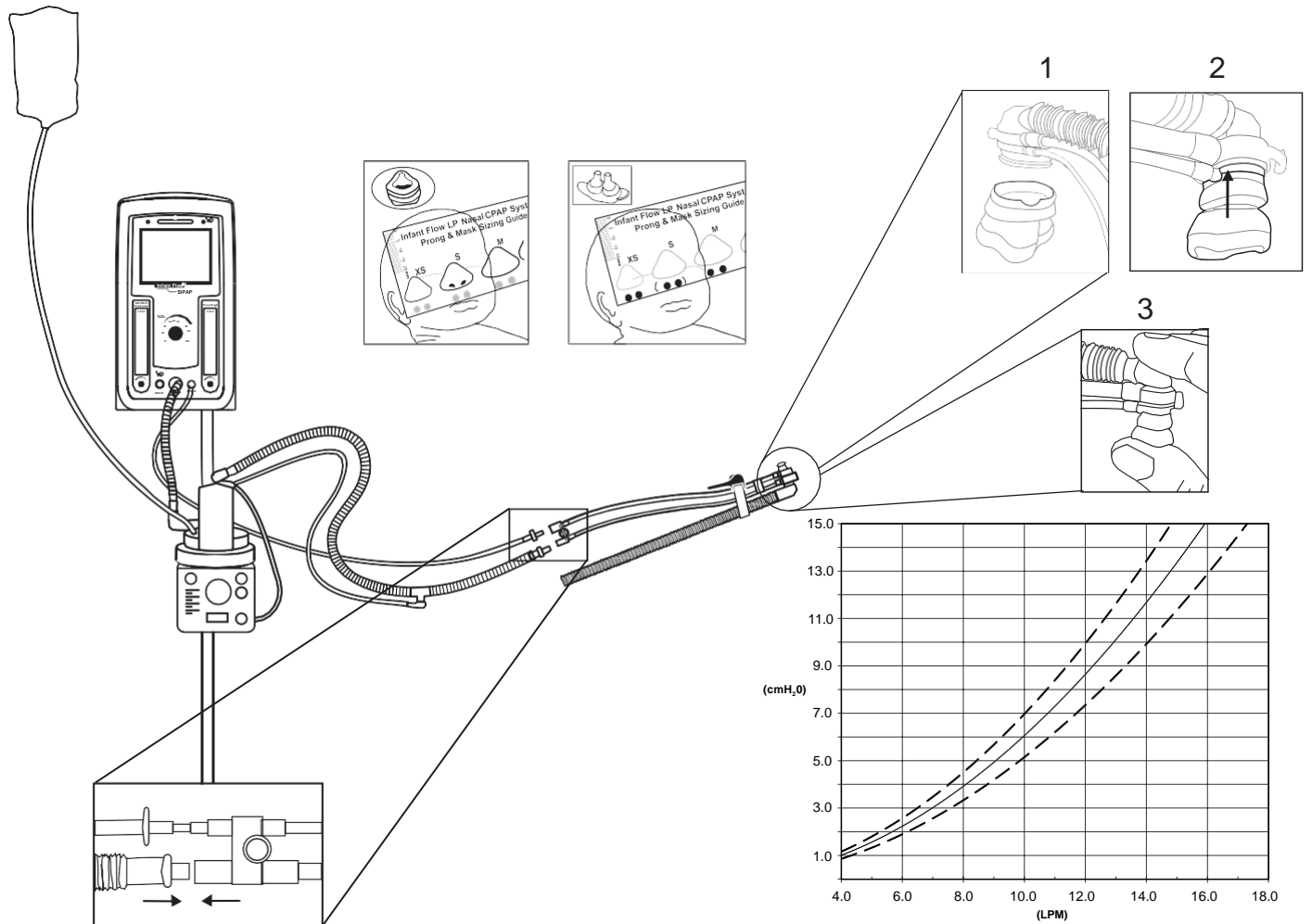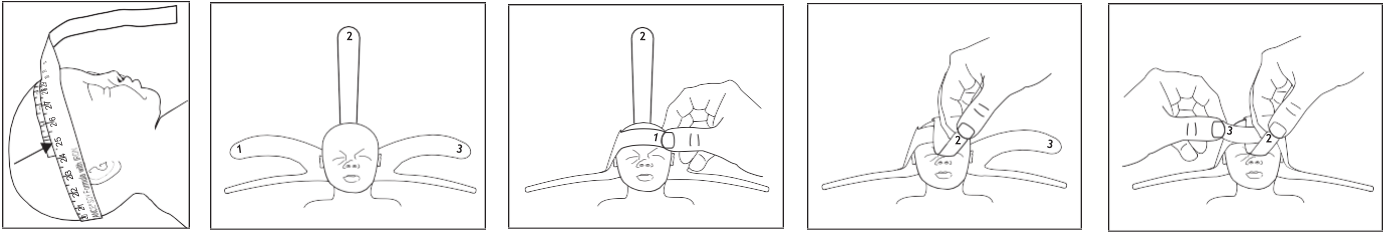

BONNET

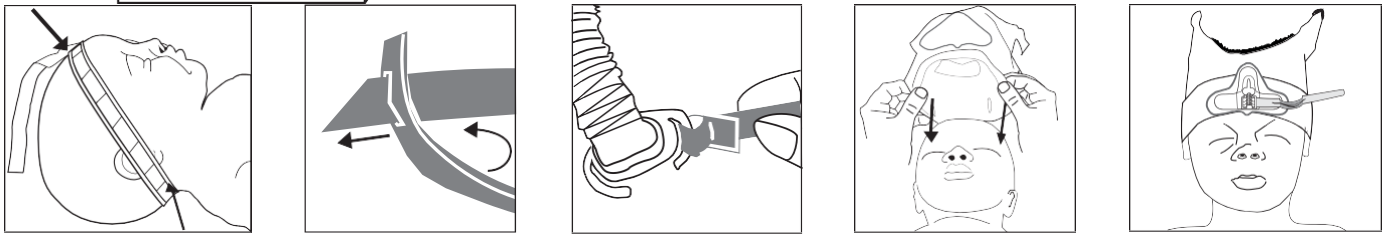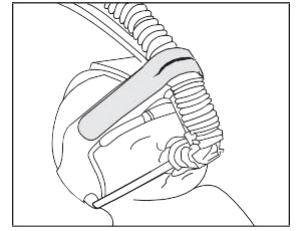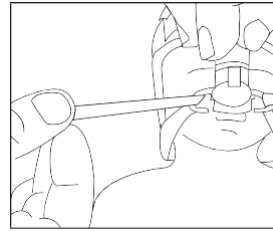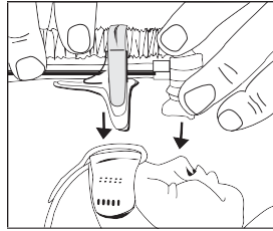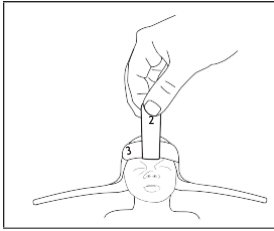
For use with the infant flow siPAP, infant flow, Airlife drivers, bellavista ventilator (with neo option) and the ix5 series ventilators. This device, consisting of a generator, prongs, masks, bonnet and headgear, is intended to provide CPAP to spontaneously breathing infants showing signs of respiratory distress.

⚠Warning:

- Use this product only as directed in the product literature to reduce the risk of nasal irritation, septal distortion, skin irritation, and pressure necrosis.
- To be used by a trained practitioner, under the direct supervision of a qualified physician.
- Only use the Infant Flow LP generator with variable flow nCPAP drivers.
- Do not over tighten the fixation straps.

Notes:

- Select the appropriate size nasal mask to minimize leaks and dead space.
- Select the appropriate size nasal prongs; if between sizes, select the larger size.
- Application of an incorrectly sized prong, mask, bonnet, or headgear will affect stability of the generator. Consider alternating the use of prong and mask interfaces at set intervals to change pressure points on the infant's face.
- Continuously monitor patient's respiratory status, (respiratory rate, heart rate, oxygen saturation).
- Cover both ears evenly; ensure the ears are not folded.
- Adjust the straps to stabilize the generator and maintain a seal at the nose using the least tension possible.
- Recommend humidification be used with nCPAP systems.
- The Infant Flow LP has a built in "pop-off," which is activated if the drive pressure exceeds 60 cmH₂O.
- For expiration dates, reference product labeling.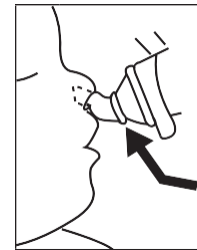

Duration of Use:

- Generator Assembly shall operate as specified for a continuous use of up to 30 days.
- Patient Interface (Prongs and Masks) shall operate as specified for a continuous use up to 14 days.
- Fixation Devices (Headgear and Bonnets) shall operate as specified for a continuous use of up to 2 days or 24 on/off cycles (based on normal patient inspection as frequently as every 2 hours over the 48 hour period).

Final Check and Routine Inspection

Inspect the system after set-up and routinely at least every 3 to 4 hours to ...





- Ensure the patient is receiving the prescribed level of CPAP.
- Ensure the generator is stable, secure, and not pulling upward on the nose.
- Check for deformation or irritation to the nose or surrounding tissue.
- Ensure that the patient's septum is clearly visible when using prongs.
- Ensure that the patient's eyes are clearly visible and that the nares are not blocked, when using masks.
- Inspect the fixation device and straps for proper tension and adjust as needed to maintain a proper fit.
- Monitor the patient for gastric insufflation and abdominal distension.
- Monitor for excessive condensation in circuit and generator.

⊗ For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

Disposal Instruction: Dispose in container that is clearly labeled, leak-proof, and color coded per local standards and state environmental regulations.

Protected by Patents:
 US7578294, ZA200804775
 US7762258, ZA200804774
 US7640934, ZA200804773

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at GMB-RS-Complaint-Intake@vyaire.com and the competent authority of the Member State in which the user and/or patient is established.

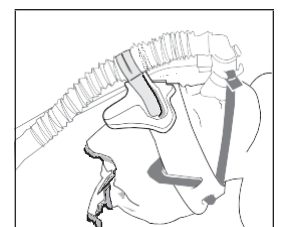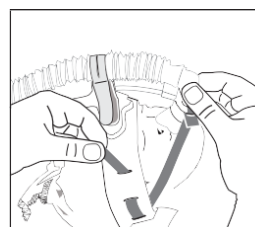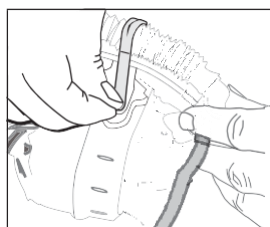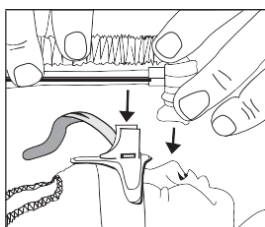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Customer and Clinical Support Product, Accessories, and Parts Ordering

1-833-327-3284
customersupport@vyaire.com

| | |
|---|--|
|  | General warning, as defined in ISO7010-W001 |
|  | Manufacturer Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC, and 98/79/EC |
|  | Authorized representative in the European Community. Indicates the authorized representative in the European Community. |
|  | Device meets European directive 93/42/EEC concerning medical devices. |
|  | Catalogue Number Labels the equipment part number. |
|  | WEEE Labels the device as NOT disposable through the normal waste stream (incineration, for example) |
|  | Caution: Federal law restricts this device to sale by or on the order of a physician. |
|  | Do not re-use. |
|  | Medical Device |
|  | Exp. Date |
|  | Not made with natural rubber latex. |
|  | Consult instructions for use. |
|  | Not made with PHT-DEHP. |

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