# IMMOB K-LIGHT™

## TRAUMA

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Specialstorlekar, specialtillämpningar mm.

CE Registrationcode 344-2757-118173


ESON COMFORT  
www.esoncomfort.se  
"WE INVENT THE FUTURE"

# TT-HEADFIX K-LIGHT™


## HOW TO USE

1. Assemble TT-Headfix K-Light™ over the neck of TraumaTransfer™. Stretch it firmly and mount the velcro on the underside. The two velcro straps on the girdle shall point upwards like in the picture.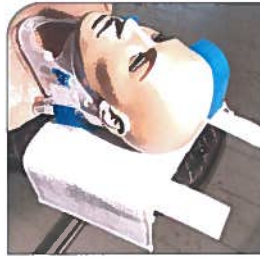

2. Assemble one of the head cushions to the velcro, on top of the girdle. NOTE! One cushion can be mounted before the patient is placed on the TraumaTransfer™. TraumaTransfer™ is ready for use!

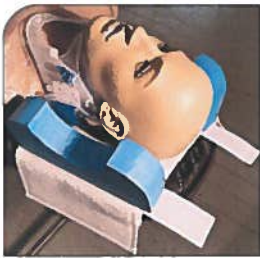3. Center the patient with the head equal to the top of the TraumaTransfer™. After moving the patient to the TraumaTransfer™, assemble the head cushion number two the same way.

4. Now you can secure the head cushions with the two velcro straps. Assemble according to the picture. NOTE! Head cushions ordered separately.

5. Assemble the VFLEXFIX™-straps according to the picture. Start with the chin and end with the forehead.

6. The patients head is now fully immobilized. The picture also shows the velcro straps that secures the head cushions and side flaps of the girdle.

7. The patients head is now in a safe and comfortable position. Please, be aware of the importance of using immobilisation cushions like TT-Pelfix™ or TT-SIP™, to prevent the body from moving on the TraumaTransfer™.

8. This image shows TT-Cover K™™ "the Sahlgrenska version". This version has a single double sided head cushion. The TT-Headfix K-Light can also be ordered as a complete cover, as TT-Cover K-Light™

ESON COMFORT  
Idé-Möbler i Landeryd AB  
Centrumvägen 7  
SE-314 94 LANDERYD

Tel +46 (0)371-40051  
[info@esoncomfort.com](mailto:info@esoncomfort.com)

Copyright 2016 Eson Comfort / Idé-Möbler i Landeryd AB.

All rights reserved

ESON COMFORT  
"WE INVENT THE FUTURE"  
[www.esoncomfort.com](http://www.esoncomfort.com)