

Nova StatStrip™ Glucose and β-Ketone Control Solution

För Nova StatStrip Glucose and β-Ketone Test Strips

Kortbruksbeskrivning

- StatStrip Xpress2 Glukos och Keton sjukhusmätare:
 - För in teststripet i mätaren
- StatStrip Glucose and β-Ketone Hospital Meter
 - Press QC icon on the patient test screen and scan or enter Strip Lot and QC Lot.
 - Insert strip to the meter
- Gently mix the control solution
- Discard first drop of the control solution
- Place a drop of control solution from the bottle at the end of the test strip
- StatStrip Xpress2 Glucose and Ketone Hospital Meter:
 - Compare result to acceptable ranges printed on the control solution label
- StatStrip Glucose and β-Ketone Hospital Meter:
 - Control results are displayed either PASS or FAIL and optionally together with measured value.
 - Press Accept to accept the result.



Note

Use within printed expiry date and within 3 months from first opening.

When first opened mark the vial with expiry date 3 months from the day of opening.

Store in room temperature below 30°C

Please, consult the instruction for use before performing the test.

Ordering number	Ref. no.	Product
154725	46947	NOVA STATSTRIP GLU KET CONTROL LEVEL 1
154766	46948	NOVA STATSTRIP-GLU KET CONTROL LEVEL 2
154723	46949	NOVA STATSTRIP GLU KET CONTROL LEVEL 3