

## NASAL CANNULA FITTING GUIDE

The following is a suggested procedure for applying the Optiflow™ Junior 2 Nasal Cannula (OJR410, OJR412, OJR414, OJR416, OJR418).

### FITTING\*

#### 1 Select cannula size

Select appropriate cannula size; recommended nare occlusion is approximately 50%. The sizing chart on the following page should only be used as a guide.

#### 2 Prepare skin

Prepare the patient's skin according to hospital protocol.

#### 3 Connect to gas source

Connect the cannula to the gas source.  
TIP: Place hand close to nasal prongs to ensure that there is gas flow exiting the prongs.

#### 4 Remove the first F&P Wigglepad™ 2 tabs

Remove the first backing tabs from the F&P Wigglepads™ 2 and avoid touching the adhesive.

#### 5 Insert cannula


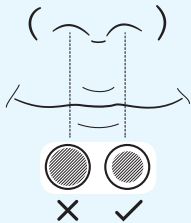



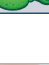
Insert the cannula into the nares. Ensure the cannula bridge rests close to the nose without touching the septum. DO NOT stretch the cannula during application. Stick the F&P Wigglepads™ 2 to the patient's cheeks.

#### 6 Secure cannula

Remove the second backing tabs and stick the F&P Wigglepads™ 2 onto the cheeks.

*\* IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.*

#### 1 SELECT CANNULA SIZE

SIZE RANGE			ENSURE GAP AROUND PRONGS*
XS		OJR410	
S		OJR412	
M		OJR414	
L		OJR416	
XL		OJR418	



### REMOVE CANNULA

#### 1 Place fingertip

Place fingertip on the outside edge of the F&P Wigglepads™ 2 and gently peel the cannula away from the Wigglepads 2.



### F&P WIGGLEPADS™ 2 REMOVAL & REPLACEMENT

#### 1 Lift the edge

Lift the edge of the F&P Wigglepads™ 2. Use a damp cloth to wipe the patient's skin and the underside of the Wigglepads 2 while gently peeling away from the patient's face.



#### 2 Adhere






Adhere the replacement F&P Wigglepads™ 2 to the cannula, remove the first backing tabs and stick the Wigglepads 2 onto the patient's cheeks.



#### 3 Remove

Remove the second backing tabs and stick onto the patient's cheeks.

### SIZING INFORMATION

		WEIGHT GUIDE (KG)								SPARES							
		500g	1	1.5	2	2.5	3	3.5	4								
	<b>OJR410 XS</b>	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>								<b>F&amp;P Wigglepads™ 2</b> WJR110							
	<b>OJR412 S</b>	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>															
		WEIGHT GUIDE (KG)								SPARES							
		1	2	3	4	5	6	7	8	9	10	15	20	25	30		
	<b>OJR414 M</b>	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>								<b>F&amp;P Wigglepads™ 2</b> WJR112							
	<b>OJR416 L</b>	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>															
	<b>OJR418 XL</b>	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>															
		<div><div></div> Expected to fit patient</div> <div><div></div> May fit patient</div>															

\* IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.

#### NOTE:

1. Recommended nare occlusion of 50% should be used to size cannula.
2. Patient weight should only be used as a guide.



To view the  
Optiflow™ Junior 2  
inservice video  
scan this QR code