


Manage
postpartum
hemorrhage.




Bakri®

POSTPARTUM BALLOON WITH RAPID INSTILLATION COMPONENTS

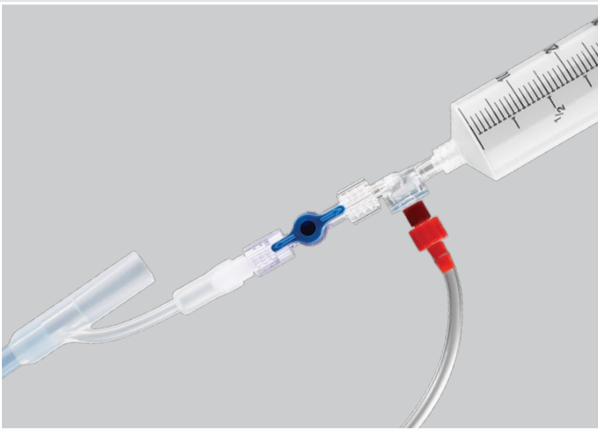

A simple solution for postpartum hemorrhage

Sequential procedure considerations for conservative management of uterotonic-resistant PPH¹

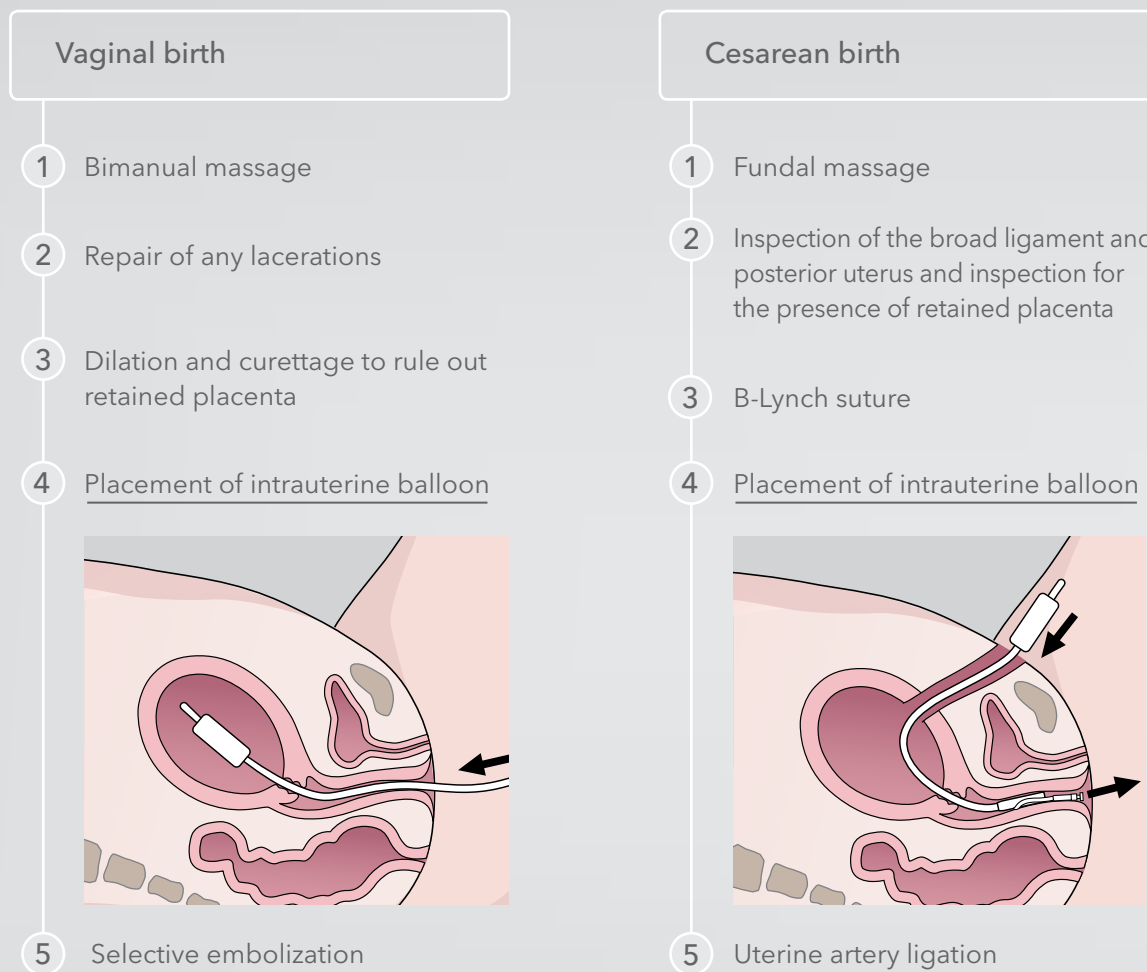

Note: The above flowchart describes sequential treatment considerations for uterotonic-resistant PPH with blood loss under 1,500 mL, after evaluation for uterine inversion and amniotic fluid embolism.

References

1. California Maternal Quality Care Collaborative. Obstetric hemorrhage emergency management plan: table chart format. Version 2.0. Published 2014. Accessed August 31, 2016.



The Bakri Postpartum Balloon with Rapid Instillation Components is used for temporary control or reduction of postpartum uterine bleeding when conservative management of uterine bleeding is warranted.

When inflated with sterile liquid, the Bakri Postpartum Balloon with Rapid Instillation Components applies pressure to the uterine walls that may provide management of postpartum hemorrhage.

- Can be used following vaginal or cesarean delivery
- Includes rapid instillation components to facilitate inflation of the balloon
- May be used with B-Lynch compression sutures if clinically warranted
- Is constructed of latex-free silicone





Bakri®

POSTPARTUM BALLOON WITH RAPID INSTILLATION COMPONENTS

This device is intended to provide temporary control or reduction of postpartum uterine bleeding when conservative management is warranted. The set includes a 60 mL syringe, extension tubing with bag spike, and dual check valve. Please refer to the Instructions for Use for complete information on product usage, indications, and contraindications.

Order Number	Reference Part Number	Catheter Fr	Length cm	Balloon Volume mL
G24237	J-SOSR-100500	24	54	500

Includes a dual check valve and 180 cm long IV tubing with attached bag spike.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Resources

1. Aibar L, Aguilar MT, Puertas A, et al. Bakri balloon for the management of postpartum hemorrhage. *Acta Obstet Gynecol Scand.* 2013;92(4):465-467.
2. Diemert A, Ortmeier G, Hollwitz B, et al. The combination of intrauterine balloon tamponade and the B-Lynch procedure for the treatment of severe postpartum hemorrhage. *Am J Obstet Gynecol.* 2012;206(1):65.e1-e4.
3. Georgiou C. Intraluminal pressure readings during the establishment of a positive 'tamponade test' in the management of postpartum haemorrhage. *BJOG.* 2010;117(3):295-303.
4. Grönvall M, Tikkanen M, Tallberg E, et al. Use of Bakri balloon tamponade in the treatment of postpartum hemorrhage: a series of 50 cases from a tertiary teaching hospital. *Acta Obstet Gynecol Scand.* 2013;92(4):433-438.
5. Kumru P, Demirci O, Erdogan E, et al. The Bakri balloon for the management of postpartum hemorrhage in cases with placenta previa. *Eur J Obstet Gynecol Reprod Biol.* 2013;167(2):167-170.
6. Vinteoux E, Ulrich D, Mousty E, et al. Success factors for Bakri™ balloon usage secondary to uterine atony: a retrospective, multicentre study. *Aust N Z J Obstet Gynaecol.* 2015;55(6):572-577.
7. Vitthala S, Tsoumpou I, Anjum ZK, et al. Use of Bakri balloon in postpartum haemorrhage: a series of 15 cases. *Aust N Z J Obstet Gynaecol.* 2009;49(2):191-194.

Customer Service

EU Website: cookmedical.eu
 EDI: cookmedical.eu/edi
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001633, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com

USA Website: cookmedical.com

EDI: cookmedical.com/edi.do

Americas:

Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335

E-mail: orders@cookmedical.com

Australia:

Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283

E-mail: cau.custserv@cookmedical.com



AI, ESC, IR, OHNS, PI, RH, SUR-A4