

# Arjo Liquids Flusher Detergent

Mild alkaline detergent to be used for automated cleaning of human waste containers. Suitable for use on alkali resistant surfaces such as stainless steel, ceramics, glass and resistant plastics.



## Direction of use

Dosing range 2-4ml/l depending on water hardness.  
Check materials compatibility before use.  
Ensure complete rinsing after use.  
Do not mix with other chemicals.

## Technical data

Specific gravity (20 °C): 1.078 (typical)  
pH as supplied (typical): 12.9  
pH at 0.5% water (typical): 9.8  
Viscosity: 3.7 cSt

## Ingredients

Sequestering agent, Scale inhibitor  
Amphoteric surfactant, Corrosion inhibitor

## Shelf life

24 months in original sealed containers.

## Safety and handling

Refer to Material Safety Data Sheet.

## Storage information

Store in a cool, dry place protected from frost and away from acids and strong oxidizing agents.  
Store upright in original containers.  
Optimum storage temperature 5-25 °C.

## Environmental awareness

Biodegradable formulation and recyclable packaging.

## ORDER INFORMATION

Language	5 I
(DE/NL/FR/IT)	504028901
(EN/PL/CZ/FR)	504028902
(PT/GR/TR/ES)	504028903
(EN/CN/RU)	504028904
(SV/DA/NO/EN)	504028905
(JP/KR/AR/EN)	504028908



Arjo's range of detergents have been developed, tested and validated on Arjo washers to consistently meet ISO 15883 standards for cleaning efficacy of medical devices.