Product specification (English)

Product: HEATING CABINET FOR CEDERROTH EYE WASH
REF: 790400

1. General description and intended use:

A cabinet made of isolating expanded polypropylene (EPP) with a thermostatic heating element.

The cabinet is intended for storage of two bottles of Cederroth Eye Wash (REF 7251) in cold environments, keeping the temperature of the Eye Wash solution at +20°C (±2°C) when the surrounding temperature is between -20°C and +20°C.

The heating cabinet automatically switches between 12V DC and 24V DC (at 230V a transformer is needed). If the equipment is left on for longer times, e.g. in a parked vehicle, a voltage guard register the incoming voltage and automatically switches the equipment off if the voltage drops below a certain lowest limit.

If desirable, it is possible to seal the cabinet door with a thin sealing string. The seal must however break at all times without any additional tooling.

2. Composition:

Heating Cabinet with electrical cord and plug type MW113301	(1 pcs)
Instructions for fixing cabinet on wall	(1 pcs)
Product specification	(1 pcs)

3. Technical description:

Cabinet:	Expanded polypropylene (EPP). Impact resistant. High chemical resistance. Resistant to temperatures up to 130°C. Dimension.: 276 x 315 x 130 mm (w x h x d) Electrical cord 2m with plug for 12V/24V cigarette lighter outlets. When removing the red adapter (bayonet joint) the plug also fits in 12V/24V 12mm outlets.
Heating element:	Aluminium profile with printed circuit card. Thermostat that keeps the temperature at +20°C (±2°C) when surrounding temperature is between -20°C and +20°C External voltage: 12V <u>or</u> 24V DC. Note: 230V with use of transformer only. Power: 15W. Secondary power with transformer min. 17W, 12,5-16VDC <u>or</u> 25-30VDC. Light-emitting diodes with indications for Power ON, 12V, 24V and Low incoming voltage. Approved according to EMC regulations EN 61000-6-3:2001, EN 55022 Class B, EN 61000-6-2:2001, EN 61000-4-2,-3, ISO/TR 7637-1-2 pulse 5

4. Labelling:

On transport packaging:	Cederroth, address, telephone- and fax number	On article:	Cederroth, address, telephone- and fax number
	Name of the product		Reference code
	Reference code		LOT-number
	EAN-code		CE-mark
	LOT-number		

5. Storage conditions:

The heating cabinet should be placed protected from extreme weather conditions (rain, snow etc.).

6. Authorization:

Authorized by: 2007-01-26
 Teija Ålander
 Quality Assurance Manager

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