


Shoulder Drape w/ Drisite® Plus

- The patented Drisite Plus system enhances drape absorption to minimise risk of bacteria transfer and unwanted liquids on the OR floor
- Impermeable materials all over to protect from strikethrough in all surgical situations
- Soft plastic adhesive edge towards wound area gives a secure fit to skin and provides an optimal adhesion zone for extra pouch
- Flexible split sheet solution to be used as it is or with separate fluid collection pouch placed for either supine or sitting position
- Ultra strong material performance in critical area




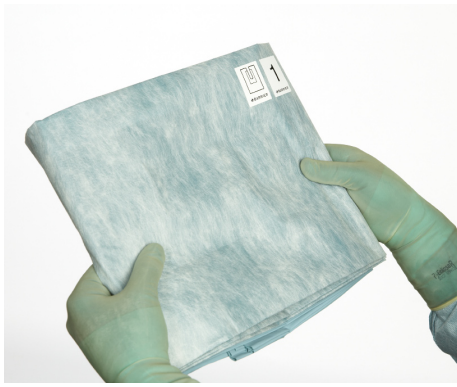

1 Position the first drape towards patient as indicated by the instruction label.




2 Position the split sheet below the arm. Unfold the drape side to side and then towards patient's feet. Remove the release paper. Secure adhesive edge one side at a time. Place the split sheet where the operation area is the widest.

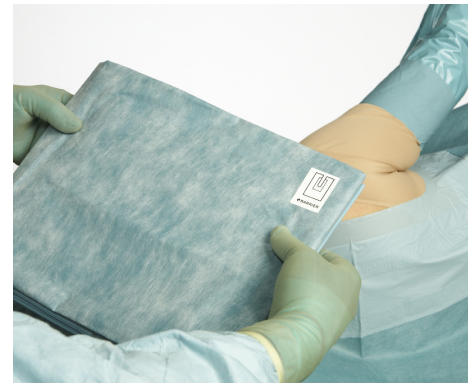

3 Position the second split sheet above the shoulder as indicated by the instruction label.



4 Unfold the drape side to side and then towards patient's head. Remove the release paper.

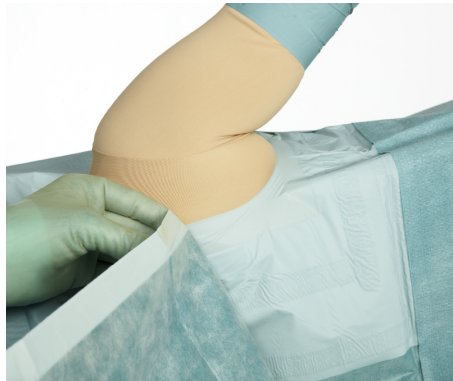

5 Place the adhesive split sheet overlapping the first split sheet for an effective barrier.



6 Unfold the split sheet towards patient's feet.

Sterile codes	
60001	Shoulder Set
996109	1 mayo stand cover reinforced 79×145 cm
–	4 cellulose towel 19×25 cm
70007	1 lower shoulder split sheet 230×295 cm, split 10×70 cm
946111	1 stockinet 22×75 cm
911510	2 op tape 9×49 cm
70005	1 upper shoulder split sheet 240×170 cm, split 15×45 cm
948090	1 table cover reinforced 150×190 cm
ProcedurePak® Components	
70007	BNS Shoulder Split Sheet 230×295 cm, split 10×70 cm, patched
70005	BNS Shoulder Split Sheet 240×170 cm, split 15×45 cm, patched

