


Split sheet w/Drisite® Plus for ENT

- BARRIER® split sheets are purpose-designed featuring only impermeable materials to minimise the risk of strikethrough in all surgical situations
- The new BARRIER surgical drapes feature soft and comfortable materials that conform naturally to the body. This improves drapeability for ENT surgery
- The patented Drisite Plus system enhances drape absorption to minimise risk of bacteria transfer and unwanted liquids on the OR floor

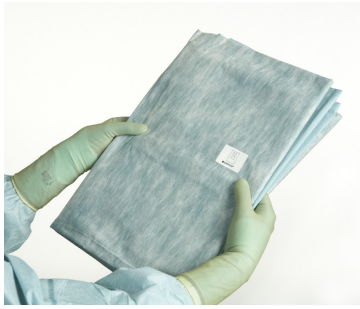



1 Position the split sheet towards patient as indicated by the instruction label.



2 Position the split sheet in the middle of the chest.



3 Unfold the drape side to side.



4 Unfold the split sheet towards patient's feet.



5 Remove the release paper.



6 Secure adhesive edge one side at a time.



7 Position the adhesive op-sheet on the chin.



8 Unfold the drape over patient's head or anesthesia frame.

Sterile codes	
65000	ENT SET
996106	1 mayo stand cover 79×145 cm
-	4 cellulose towel 19×25 cm
911510	2 op-tape 9×49 cm
910519	1 op-towel 75×90 cm
930200	1 adhesive op- towel 90×75 cm
70509	1 split sheet 150×260 cm, split 7×65 cm
914080	1 table cover 150×190 cm

