

# VICRYL\* Rapide (Polyglactin 910)

## SUTURE

### Description:

|                                 |   |
|---------------------------------|---|
| <b>Braided<br/>Coated</b>       | For easy handling and secure knot tying.<br>For smooth passage through tissue and easy knot tie down. |
| <b>Synthetic<br/>Absorbable</b> | For minimal tissue reaction.<br>By hydrolysis. Essentially complete in 42 days.                       |

### Tensile strength: Post implantation

|  | Approximate % original strength remaining |
|--|---|
|--|---|

|            |     |
|------------|-----|
| 5 days     | 50% |
| 10-14 days | 0%  |

VICRYL\* Rapide suture typically falls off 7-10 days post-operative or can be wiped off subsequently with gauze. Normally the removal of the suture is not required.

**Color:** Undyed or violet.




**Range:** 8-0 to 1 (USP). Supplied as needled sutures and ligatures.





**Indications:** VICRYL\* Rapide is intended for use in soft tissue approximation where only short term wound support is required and where the rapid absorption of the suture would be beneficial. Typical areas of use include:


- Skin closure, cuticular or subcuticular
- Episiotomies
- Closure of oral mucosa
- Conjunctival sutures in ophthalmic surgery


Full details in the Instruction for Use included in every package.

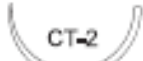

| <b>1/2 Circle Taper Point Needles</b>  |       |              |             |                   |                |        |
|--|-------|--------------|-------------|-------------------|----------------|--------|
| Needle   | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code   |
| <br>RB-1<br>17 mm | 5-0   | ○            | 70          |                   | 36             | V2130H |
|  | 4-0   | ○            | 70          |                   | 36             | V2140H |
|  | 3-0   | ○            | 70          |                   | 36             | V2150H |
| <br>SH-1<br>22 mm | 4-0   | ○            | 70          |                   | 36             | V2180H |
|  | 3-0   | ○            | 70          |                   | 36             | V2190H |
|  | 2-0   | ○            | 70          |                   | 36             | V62H   |
| <br>SH<br>26 mm   | 4-0   | ○            | 70          |                   | 36             | V4150H |
|  | 3-0   | ○            | 70          |                   | 36             | V4180H |
|  | 2-0   | ○            | 70          |                   | 36             | V4170H |
|  | 0     | ○            | 70          |                   | 36             | V66H   |




| <b>1/2 Circle Taper Point Needles (Heavier Wire Diameter)</b>  |       |              |             |                   |                |         |
|--|-------|--------------|-------------|-------------------|----------------|---------|
| Needle   | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code    |
| <br>CT-2<br>26 mm  | 2-0   | ○            | 70          |                   | 36             | V55H    |
|  | 0     | ○            | 70          |                   | 36             | V56H    |
| <br>CT-1<br>36 mm | 2-0   | ○            | 90          |                   | 36             | V9450H  |
|  | 0     | ○            | 90          |                   | 36             | V9480H  |
|  | 1     | ○            | 90          |                   | 36             | V70H    |
| <br>CT<br>40 mm   | 0     | ○            | 90          |                   | 36             | V9580H  |
| <br>CTX<br>48 mm  | 0     | ○            | 90          |                   | 36             | PN1695H |

| <b>1/2 Circle ETHIGUARD* Blunt Point Needles</b>  |       |              |             |                   |                |        |
|---|-------|--------------|-------------|-------------------|----------------|--------|
| Needle  | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code   |
| <br>CTB-1<br>36 mm | 2-0   | ○            | 90          |                   | 36             | W9952H |


| <b>Straight Taper Point Needles</b>  |       |              |             |                   |                |         |
|--|-------|--------------|-------------|-------------------|----------------|---------|
| Needle   | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code    |
| <br>ST-4<br>19 mm | 4-0   | ○            | 70          |                   | 36             | BA1673H |

**VICRYL\* Rapide**  
(Polyglactin 910)


**1/2 Circle TAPERCUT\* Needles**

| Needle  | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code    |
|---|-------|--------------|-------------|-------------------|----------------|---------|
|  V-6<br>17 mm  | 3-0   | ○            | 70          |                   | 12             | OA1674G |
|  V-7<br>26 mm  | 3-0   | ○            | 70          |                   | 12             | OB1674G |
|   | 3-0   | ○            | 70          |                   | 36             | OB1674H |
|  V-34<br>36 mm | 2-0   | ○            | 90          |                   | 36             | V5170H  |
|   | 0     | ○            | 90          |                   | 36             | V5180H  |


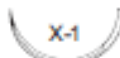
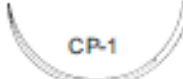

**Compound Curve TAPERCUT\* Needles**

| Needle  | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code   |
|---|-------|--------------|-------------|-------------------|----------------|--------|
|  DA<br>25 mm | 4-0   | WHITE        | 45          |                   | 12             | V2912G |

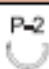


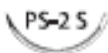
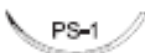
**1/2 Circle Reverse Cutting PRIME\* Needles**

| Needle  | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code    |
|---|-------|--------------|-------------|-------------------|----------------|---------|
|  P-2<br>8 mm | 4-0   | ○            | 45          |                   | 36             | JA1642H |




**1/2 Circle Reverse Cutting Needles**

| Needle  | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code   |
|---|-------|--------------|-------------|-------------------|----------------|--------|
|  J-1<br>17 mm  | 4-0   | ○            | 45          |                   | 12             | 18510G |
|   | 4-0   | ○            | 45          |                   | 36             | V4550H |
|   | 3-0   | ○            | 45          |                   | 36             | V76H   |
|  X-1<br>22 mm  | 3-0   | ○            | 45          |                   | 36             | V7902H |
|   | 2-0   | ○            | 45          |                   | 36             | V81H   |
|  CP-1<br>36 mm | 2-0   | ○            | 90          |                   | 36             | V4721H |
|  CP<br>40 mm   | 0     | ○            | 90          |                   | 36             | V4851H |
|   | 1     | ○            | 90          |                   | 36             | V4861H |



**3/8 Circle Reverse Cutting PRIME\* Needles**


| Needle  | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code      |
|---|-------|--------------|-------------|-------------------|----------------|-----------|
|  P-1<br>11 mm      | 6-0   | ○            | 45          | MULTIPASS*        | 36             | MPVR9913H |
|   | 6-0   | ●            | 45          |                   | 36             | V32H      |
|   | 6-0   | ○            | 45          |                   | 12             | W9913     |
|   | 5-0   | ○            | 45          | MULTIPASS*        | 36             | MPVR9915H |
|   | 5-0   | ○            | 45          |                   | 12             | W9915     |
|  P-3<br>13 mm      | 5-0   | ○            | 45          | MULTIPASS*        | 36             | MPVR4930H |
|   | 5-0   | ○            | 45          |                   | 36             | V4930H    |
|   | 4-0   | ○            | 45          | MULTIPASS*        | 36             | MPVR4940H |
|   | 4-0   | ○            | 45          |                   | 36             | V4940H    |
|  PS-2<br>19 mm     | 4-0   | ○            | 45          | MULTIPASS*        | 36             | MPVR4960H |
|   | 4-0   | ○            | 75          |                   | 12             | W9922     |
|   | 4-0   | ○            | 75          | MULTIPASS*        | 36             | MPVR4260H |
|   | 3-0   | ○            | 45          |                   | 36             | V4970H    |
|   | 3-0   | ○            | 45          | MULTIPASS*        | 36             | MPVR4970H |
|   | 3-0   | ○            | 70          | MULTIPASS*        | 36             | MPVR4270H |
|   | 3-0   | ○            | 75          |                   | 12             | W9923     |
|  PS-2 S<br>19 mm | 4-0   | ○            | 45          | Slimmed           | 36             | V4960H    |
|   | 4-0   | ○            | 70          | Slimmed           | 36             | V4260H    |
|  PS-1<br>24 mm   | 3-0   | ○            | 45          |                   | 36             | V6830H    |
|   | 3-0   | ○            | 70          | MULTIPASS*        | 36             | MPVR9360H |
|   | 3-0   | ○            | 70          |                   | 36             | V9360H    |
|   | 2-0   | ○            | 70          | MULTIPASS*        | 36             | MPVR9380H |
|   | 2-0   | ○            | 70          |                   | 36             | V9380H    |

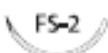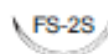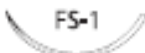



**3/8 Circle Reverse Cutting Needles**

| Needle   | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code    |
|--|-------|--------------|-------------|-------------------|----------------|---------|
|  FS-2<br>19 mm  | 5-0   | ○            | 45          |                   | 36             | V2910H  |
|  | 4-0   | ○            | 45          |                   | 36             | FH1642H |
|  | 3-0   | ○            | 45          |                   | 12             | V2930G  |
|  | 3-0   | ○            | 45          |                   | 36             | V2930H  |
|  FS-2S<br>19 mm | 4-0   | ○            | 45          | Slimmed           | 36             | V2920H  |
|  FS-1<br>24 mm  | 3-0   | ○            | 45          |                   | 36             | V4421H  |
|  | 3-0   | ○            | 70          |                   | 36             | V4420H  |
|  | 2-0   | ○            | 45          |                   | 36             | V4431H  |

**VICRYL\* Rapide**  
(Polyglactin 910)

| ▲ Straight Cutting Needles   |       |              |             |                   |                |         |
|--|-------|--------------|-------------|-------------------|----------------|---------|
| Needle   | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code    |
|  TS 48 mm | 3-0   | ○            | 75          |                   | 36             | VR2239  |
|  | 2-0   | ○            | 75          |                   | 36             | VR2240  |
|  KS 60 mm | 3-0   | ○            | 70          |                   | 36             | AA1673H |

| ▲ 3/8 Circle Conventional Cutting PRIME* Needles   |       |              |             |                   |                |           |
|--|-------|--------------|-------------|-------------------|----------------|-----------|
| Needle   | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code      |
|  PC-3 16 mm | 4-0   | ○            | 70          | MULTIPASS*        | 36             | MPVR9918H |
|  | 3-0   | ○            | 75          |                   | 12             | W9919     |

| ▼ 3/8 Circle MICROPPOINT* Spatula   |       |              |             |                   |                |        |
|---|-------|--------------|-------------|-------------------|----------------|--------|
| Needle  | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code   |
|  GS-12 5,5 mm        | 8-0   | ●            | 30          |                   | 12             | V1220G |
|  GS-9 6,5 mm        | 8-0   | ●            | 30          |                   | 12             | V1210G |
|   | 7-0   | ●            | 30          |                   | 12             | V1211G |
|  GS-9, GS-9 6,5 mm | 8-0   | ●            | 30          |                   | 12             | V39G   |
|   | 7-0   | ●            | 45          |                   | 12             | V38G   |
|  TG140-8 6,5 mm    | 7-0   | ●            | 30          |                   | 12             | W9914  |

| Ligatures |       |              |             |                   |                |        |
|-----------|-------|--------------|-------------|-------------------|----------------|--------|
| Needle    | Gauge | Suture Color | Length (cm) | Other information | Number per box | Code   |
| SUTUPAK*  | 4-0   | ○            | 3x45        |                   | 24             | V8643E |
|           | 2-0   | ○            | 3x45        |                   | 24             | V8645E |
|           | 0     | ○            | 3x45        |                   | 24             | V8646E |
|           | 0     | ○            | 6x45        |                   | 24             | V8980E |