



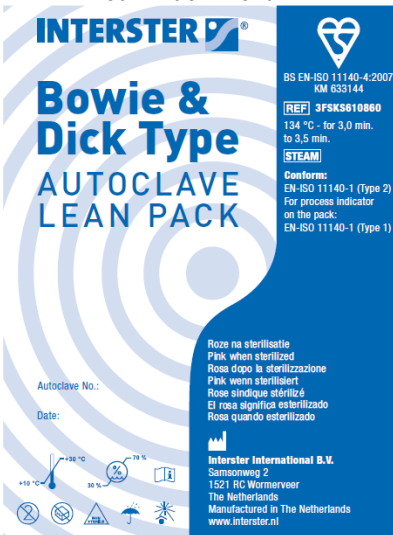

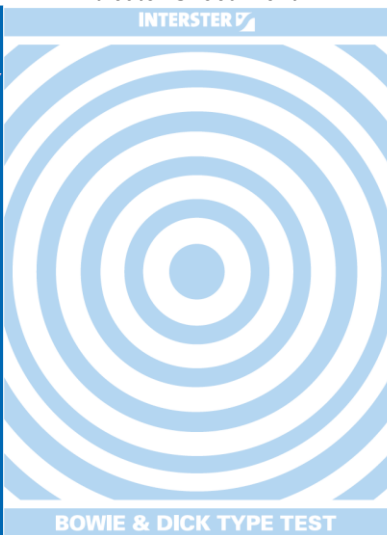


Product: Bowie & Dick Type Autoclave Lean Pack
Article code: 3FSKS610860

TECHNICAL DATA SHEET 1/2				
Product name:	Art. Code:	UDI Code:	Contents:	Size:
Bowie & Dick Type Autoclave Lean Pack 3,0 - 3,5 min / 134°C - 137°C	3FSKS610860	87199791200160	20pcs	<i>Indicator sheet:</i> Length: 100mm Width: 150mm <i>Lean pack:</i> Length: 100mm Width: 75mm Height: 30mm <i>Distribution pack:</i> Length: 315mm Width: 107mm Height: 160mm
Intended use:	The Bowie & Dick Type Autoclave Lean Pack is intended to test daily if the process within the autoclave is conform the requirements.			
Shelf life:	4 years			
Sterile / Non-sterile:	Non-sterile			
Sterilization method:	Steam			
Resterilization:	No			
Latex free:	Yes			
DEHP free:	Yes			
Single use:	Yes			
CE class:	Not applicable			
Standards:	ISO11140-1 (Type-2) & ISO11140-4			
Picture:	<div><div><p><i>'Lean' Pack Front</i></p></div><div><p><i>Indicator Sheet Front</i></p></div><div><p><i>Indicator Sheet Back</i></p></div></div>			
Material:	The Bowie & Dick Type Autoclave Lean Pack is made from a lean pack of 240 g/m ² cardboard which is printed in offset. The indicator sheet is made from 70g/m ² medical paper and is applied by silk screen procedure on one side and offset printed on the back. The indicator sheet is enclosed in 42g/m ² procurator paper. On the front is a Type I process indicator which changes from light blue to pink.			
Type test:	The Bowie & Dick Type Autoclave Lean Pack is a 'type test', which is based on the 'original' Bowie & Dick test using a 7KG pack. The Bowie & Dick Type Autoclave Lean Pack has been tested using a 7KG pack as is mentioned in annex E of the ISO11140-4.			



Product: Bowie & Dick Type Autoclave Lean Pack

Article code: 3FSKS610860

TECHNICAL DATA SHEET 2/2	
Test results:	The Indicator has to change from light blue to even pink or lighter after sterilization.
Product family:	The Bowie & Dick Type Autoclave Lean Pack is part of the control product family.
Related products:	3FSKS610804 Bowie & Dick Type Autoclave Test Pack
Storage conditions:	The Bowie & Dick Type Autoclave Lean Pack should be stored in dry, cool areas away from sunlight with a temperature range of 10-30°C and a relative humidity of 30-70%.
Date: 03-02-2021  B. Randshuizen Product Manager	Date: 03-02-2021  R. Strackx Commercial Director
<p style="text-align: center;">Disclaimer</p> <p>The information supplied in this document is for guidance only and should not be construed as a warranty. All implied warranties are expressly disclaimed, including without limitation any warranty of merchantability and fitness for use. All users of the material are responsible for assuring that it is suitable for their needs, environment and end use. All data is subject to change as Interster International B.V. deems appropriate. Refer to www.interster.nl for contact information.</p>	

This document replaces all preceding Technical Data Sheets